



- Improves mobility and allows for greater independence for the user on the go
- Lightweight design makes rollator easy to use, fold and store for convenient transport
- Padded seat and backrest provide additional comfort
- Ergonomic hand grips and hand brakes allow user to operate with ease
- Complete with pouch for storage of personal items
- 6" wheels allow for ease of movement
- 300 lb weight capacity - EVENLY DISTRIBUTED (350 lb on RJ4318 Wide models)



## Walkabout Lite, Hemi and Wide 4-Wheel Rollator

Walkabout Lite, Hemi and Ancho Rodador de 4 Ruedas  
Items RJ4300, RJ4302 & RJ4318



### User Manual

Important: Read Before Use  
*Instrucciones e Información de la Garantía*  
Importante: Lea Antes de Uso



GF0600057RevF15

## Limited Warranty

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at [www.grahamfield.com](http://www.grahamfield.com).

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for five (5) years and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

## Instructions for Use – Please Read All Instructions Before Use.

Please consult a healthcare professional for the proper adjustment.

### Assembling the Rollator

1. Remove Rollator and other components from carton.
2. Unfold frame; spread front and back legs apart.
3. Push down locking mechanism to secure legs in place.  
**NOTE:** Ensure the side crossbars (locking mechanism) lock down.
4. Insert handgrip handle bars into frame.  
**NOTE:** After installation, make sure brake cable is secured to frame with rollator clip.
5. Secure positioning of handle bars with adjustment knob.
6. Install backrest into frame and secure with push buttons.

### Folding the Rollator

1. If necessary, remove the accessories and lift the seat.
2. Hold one handle and pull up on the Folding Pull Strap that is attached to the crossbar (locking mechanism) or the pouch underneath the seat.

### Using the Hand Brake

Brakes should be used in one of the following methods:

- To Remain Stationary: Hold the hand brake up.
- To Continue Mobility: Release the brake. To unlock, pull up on the top portion of the brake handle.
- To Sit on Seat: Push down on the bottom portion of the brake handle to lock.
- To unlock, pull up on the top portion of the brake handle.  
**CAUTION:** Always lock hand brakes when user is immobile; never sit on rest seat if hand brakes are not locked.

### Adjusting the Height

1. Remove the adjustment knob and bolt by turning counterclockwise.
2. Consult with healthcare professional for proper adjustment.
3. Install the adjustment knob into one of the adjustment holes by turning clockwise.
4. Repeat STEPS 1–3 for the other side. Make sure that the height adjustment is at the same height for both handles to ensure safety and proper use.  
**NOTE:** After adjusting the height, the brakes of the rollator must be checked and, if necessary, adjusted.

## Attaching Basket, Pouch or Tote Bag

### Basket

Install the basket underneath the seat. On some rollator models, the basket is designed to fit underneath the seat or in the front by positioning the hooks on the front seat support frame.

### Pouch and Tote Bag

Hang the tote bag between the rollator frame tubes and secure with snap button flaps.

### Maintenance

- For dust and dirt, wipe Rollator gently with damp, soft cloth.
- Inspect wheels periodically for wear or damage. Use a mild soap and water solution to clean, rinse with clear water, then dry with a soft cloth.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the rollator.
- Before every use, inspect operation of the brakes and have them adjusted if necessary by your GF Health Products, Inc. authorized distributor.
- Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your GF Health Products, Inc. distributor for repair and/or appropriate replacement parts.
- If handgrip or hand brakes are loose, **DO NOT** use. Contact your GF Health Products, Inc. authorized distributor.
- Replace any broken, damaged or worn items immediately.
- For the best service from your rollator, always use replacement parts from GF Health Products, Inc.
- Periodically inspect the casters and caster stems for assembly tightness and verify that the wheels are free of hair or other debris that could interfere with free wheel operation.

Item Number	Weight Capacity
RJ4300s	300 lb
RJ4318s	350 lb
RJ4302s	300 lb
RJ4805s	300 lb
RJ4400s	500 lb
RJ4405s	500 lb
RJ4402s	500 lb
609201s	300 lb



## Safety Guidelines – Please Read Before Use

1. Intended usage: The rollator has been designed to assist people when walking.
2. Each individual should ALWAYS consult with their physician or other health care professional to determine proper adjustment and usage.
3. The brakes MUST be in the locked position before using the seat.
4. All wheels MUST be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
5. This product has a maximum user weight limitation of 300 lbs (350 lb on RJ4318 Wide models). ALWAYS observe the user weight limitation on the hangtag of this product and on the chart referenced at left.
6. Ensure handgrips DO NOT twist on rollator handle - otherwise damage or injury may occur.
7. After installation and before use, ensure that all attaching hardware is securely tightened.
8. ALWAYS use caution when using the rollator on wet and slippery surfaces. DO NOT use the rollator on stairs, steep inclines or uneven terrain.
9. DO NOT hang anything from the frame of the rollator. Items should be placed in the pouch.
10. The rollator basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (5 kg). Items placed in either the basket or the bag should not protrude from the basket or bag.
11. The backrest should ALWAYS be in place and is NOT designed to support the total body weight of the user.
12. Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use EXTREME caution when reaching for any object.
13. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
14. Inspect rollator weekly or on a set routine preventative maintenance schedule for normal wear and tear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.
15. If the hand brakes fail to stop the Rollator, contact your GF Health Products, Inc. authorized distributor immediately for adjustment or replacement.

**WARNING:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.

**CAUTION:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in personal injury or product/property damage.

**NOTE:** Provides application recommendations or other useful information to ensure that you receive the greatest benefit from this product.

### WARNING

**WARNING:** DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

**WARNING:** DO NOT use the rollator if there is any damage to the frame.

**WARNING:** DO NOT use the rollator as a wheelchair or transport device. Rollators are not intended to be propelled while seated.

**WARNING:** If components are damaged or missing, contact your distributor immediately. DO NOT substitute parts.

**WARNING:** Before use, ensure that all assembled parts are secure and all hardware is tightened.

**WARNING:** Care should be taken to ensure that all hand and height adjustments are secure, and that wheels and moving objects are in good working order before using this rollator.

**WARNING:** When using the rollator in a stationary position, the hand brakes MUST be locked.

**WARNING:** ALWAYS observe the user weight limitation.

**WARNING:** Notice for California Customers- **California Proposition 65 WARNING:** This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

**WARNING:** DO NOT attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the rollator and may cause the rollator to tip, resulting in injury or damage.