

WalkingBelt

SystemRoMedic™

User manual – English



REF

6041

6042



SWL: 50 kg/110 lbs

WalkingBelt is used as support for gait training with children with impaired strength and stability. WalkingBelt provides support around the torso and is held in place with a padded leg harness. WalkingBelt has two padded shoulder straps, which are held by the caregiver to provide additional support and stability during gait training. It is not to be used as an assistive device for lifting. WalkingBelt has a soft polyester lining that feels comfortable against the skin. The material has an anti-slip function.



Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.



Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

Applying WalkingBelt



Place WalkingBelt around the user's waist and place the arms through the shoulder straps. Fasten and tighten the strap.

Adjust the shoulder straps to the appropriate length. The caregiver should keep her/his arms relatively straight during gait training.

Place the leg harness under the user, fasten and tighten. The leg harness should be tight enough to provide good support, but not so tight that it causes discomfort.

It may be necessary to tighten the waist strap and leg harness once the user has risen to a standing position.



Using WalkingBelt



Adjust the length of the shoulder straps to enable a comfortable working stance for the caregiver and good support for the child.

Stay close to the child to give support and assurance. Be attentive to the child's signals. Discontinue training at the least sign of discomfort.

Do not use the WalkingBelt as an assistive device for lifting!

Removing WalkingBelt

Pull down the shoulder straps. Disconnect the clasps on the leg harness and waist strap. Remove the belt.

Material

Polyamide, polyester

Care of the product

Read the product label.



Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Expected lifetime

Up to 10 years of normal use

Product information

Art. no.	Productname	Size	Measurements	Rec. user weight
6041	WalkingBelt	XXS	600 mm/23,6"	0-25 kg/ 0-55 lbs
6042	WalkingBelt	XS	700 mm/27,5"	20-35 kg/44-77 lbs