

THANK YOU...

Thank you for purchasing the SuperPole™ Ultra. We are confident that you will find the unique design and durable construction of this product to be of great assistance in your daily routine.

To fully enjoy your SuperPole™ Ultra, please take a few moments to read these instructions (and the SuperPole™ Instructions DC100). You should store these instructions in a safe place for future reference. SuperPole™ Ultra is covered under SuperPole™ warranty (see DC100 - SuperPole™ Instructions).

⚠ SAFETY CONSIDERATIONS

CAUTION: It is your responsibility to see that your SuperPole™ System and SuperPole™ Ultra are properly assembled, installed, and cared for. Failure to follow instructions could result in serious injury.

SUPERPOLE™ ULTRA INSTALLATION

The SuperPole™ Ultra components (Ultra top plate, Ultra pole and extension tube) should not be used with the components of any other floor to ceiling poles. These parts have been labelled for your convenience. Do not use with the following HealthCraft accessories with the SuperPole™ Ultra or injury may occur: SuperBar™, Safety Island™, SuperTrapeze™, Uni-Fit™, and Angled Ceiling Plate.

ASSEMBLY INSTRUCTIONS

The SuperPole™ Ultra replaces the standard SuperPole™ in the SuperPole™ System instructions (DC100). Follow the instructions below to assemble the SuperPole™ Ultra, and then proceed with the instructions of the pole.

G1. Unpack the top plate from the rectangular box, the ultra pole (41) and the extension tube assembly (42) from their shipping tubes.

G2. Insert the end of the ultra pole with the o-rings into the extension tube assembly as shown in **Figure 1a**. The fit should be snug. If necessary tap the poles together using a hammer and a block of wood. Do not hammer directly on the pole. Instead, use the block of wood between the extension screw and the hammer to prevent damage.

G3. Continue to tap the poles together so until the ultra pole is completely inserted into the extension tube assembly. There should be no gap between the bottom of the pole and the top shoulder of the joiner piece, as shown in **Figure 1b**. Any remaining gap could result in a loss of compression force in the installed pole causing possible movement of the pole, and injury.

G4. The ultra pole assembly is now complete. Using this assembly instead of the standard SuperPole™, complete steps A2 to A11 on page 4 of the SuperPole™ Instructions (DC100).

Your SuperPole™ installation is now complete.

Figure 1a.


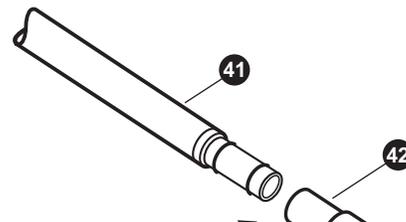


Figure 1b.



SP-ULTRA: SUPERPOLE™ ULTRA

SP403HDU
Ultra Top Plate

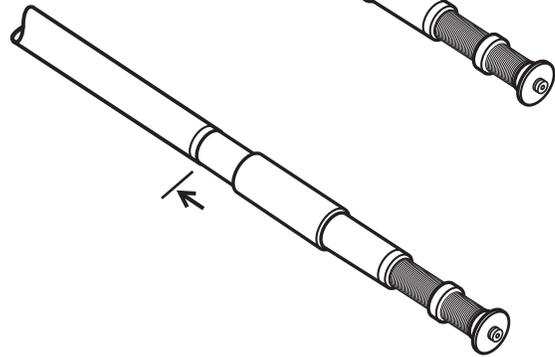

SP400HDU
Assembly,
Ultra Pole +
Joiner with Grip

SP153
Extension Tube
Assembly, Ultra

SP402
Base Plate
Assembly



SP400HDU: ASSEMBLY, ULTRA POLE + JOINER DETAIL

SP152
Weldment,
Pole with Joiner

HW411
O-ring



SP153: EXTENSION TUBE ASSEMBLY DETAIL

SP151
Custom Cut
Extension Tube

SP114
Plastic Cover Sleeve

SP105HDU
Threaded Collar Ultra

SP172
Jam Nut

SP107
Extension Screw

SP145
Plastic Washer

