

Homecraft Heavy Duty Drop-Arm Commode

0816-11-110



Before Assembly

Remove contents from package, making sure that all components are included and that no parts have been damaged during handling and shipping. Contact Patterson Medical if you find damage.

Familiarize yourself with the parts.

Read the following instructions and make sure you understand them. For clarification of any step, consult a healthcare professional.

Assembly Instructions

Attach pieces as follows: insert leg frames on the main frame; seat and seat cover on the back support; and back support on the main frame.

Determine the leg height that would ensure safety and comfort of the user. To achieve the chosen height: depress the spring button on leg, slide it to the corresponding hole for desired position and release the button. While adjusting the other three legs, make sure the buttons are set at the same height. At this point and before each use, make sure that buttons protrude all the way through the holes.

As a final check, try to move the legs up and down; if they do move, the buttons are not locked.

Set the commode on a level spot on the floor, where it will be stable and safe for the user. When

the commode is positioned between the shower and outside of the shower, take extra care that the commode is level. Any instability puts the user at risk of sliding off the commode and falling.

Drop Arm Instructions

The arm supports should only be used as assistance for the user—they DO NOT support the full body weight. They should be dropped down before the user transfers on and off the commode. To lower, push down the lever at front of armrest. Drop it and attach to back leg tube. Do the reverse to lift and use the push button to secure the arm. Make sure the armrests are secure before using them.

Precautions

Supervision or assistance should be provided for users who are weak or tend to be unbalanced.

Before each use, ensure that the spring buttons protrude all the way through the appropriate holes so they are locked in place.

Confirm that legs are set securely on the floor, and that the seat is level.

Test for stability before use. Do not exceed the user weight capacity of 500 lbs.

Care

On a regular basis or after each use when being utilized as a portable commode, clean the bucket

with a disinfectant and a soft cloth. Do not use abrasive, scouring pads or solvents including lacquer thinner, turpentine or acetone. An everyday cleaning product or window cleaner can be used on the frame.

Periodically inspect the rubber tips on the legs. If tips are torn or missing, replace them immediately.

Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.