

# general purpose slings



Our General Purpose slings have a high weight capacity and a generous overall width to offer a stretched seating position that is well-suited for lifting heavier users in all situations. The integrated head support allows the patient to be lifted even from a floor.

### Promotes Infection Control

These single patient use slings offer the convenient option of disposal after use combined with the added benefit of hygiene control. Since the sling can only be used by one patient at a time, facility infection control protocols are easily supported.

### Two Convenient Options

The General Purpose sling is available with either reinforced loops that ensure durability and ease of use, or with clips for easy and secure attachment. Either option is an ideal solution for sudden positioning change requirements. These slings are available in adult sizes Small to Extra Large and can carry a maximum capacity of 318kg/700lb.

### Optimal Features

These slings feature divided legs and a high back for added support. They are made of a strong but soft polyester/cotton blend and are ideal for a seated, horizontal or semi-recumbent transfer. The unique material wicks away fluids and can be left behind a patient. Model #NSB-104 can be used with a spreader bar. Model #NSB-105 features loops that can be attached to a hanger bar with 2-point, 3-point or 4-point suspension.

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.: 21180.\*



### Leg Padding

Soft padded supports for legs. Ideal for users with sensitive skin or pain. Synthetic sheepskin. Part no.: 21146-S, M, L



### Hanger Bar

For NSB-105. For 2-, 3-, or 4-point suspension. Part no. 57551, 57664 (2- or 3-point), 55619 (4-point - attach to leg hook on lift)



### Spreader Bar

For NSB-104. Powered Spreader Bar Part no.: Small 92441, Medium 92442, Large 92443

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for short period of time.



### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Use to/from the floor

Suitable for transfers from the floor.



### Single Patient Use

Sling is intended for use by a single patient at a time. Dispose after use.

## Care and Features




### Clip or Loop Sling

Two convenient attachment options - clips or loop slings.




### Reinforced Straps

Colour coded reinforced straps for easy determination of required positioning.

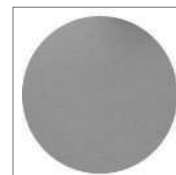

### Polyester/Cotton


Material allows fluids to wick away from patient. Do not launder. Single Use.

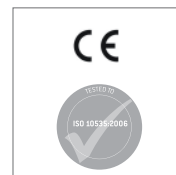

Tested to all relevant standards, including ISO 10535, hoists for the transfer of disabled persons.

## Technical Specifications

Part Number	Size	Colour	A	B	C
NSB-104C-S	Small	S	30cm / 12"	84cm / 33"	78cm / 31"
NSB-104C-M	Medium	M	43cm / 17"	88cm / 35"	97cm / 38"
NSB-104C-L	Large	L	53cm / 21"	103cm / 41"	114cm / 45"
NSB-104C-XL	Extra Large	XL	68cm / 27"	114cm / 45"	133cm / 53"
NSB-105-S NSB-105V-S <sup>1</sup>	Small	S	30cm / 12"	84cm / 33"	78cm / 31"
NSB-105-M NSB-105V-M <sup>1</sup>	Medium	M	43cm / 17"	88cm / 35"	97cm / 38"
NSB-105-L NSB-105V-L <sup>1</sup>	Large	L	53cm / 21"	103cm / 41"	114cm / 45"
NSB-105-XL NSB-105V-XL <sup>1</sup>	Extra Large	XL	68cm / 27"	114cm / 45"	133cm / 53"
Maximum User Weight	318kg / 700 lb				

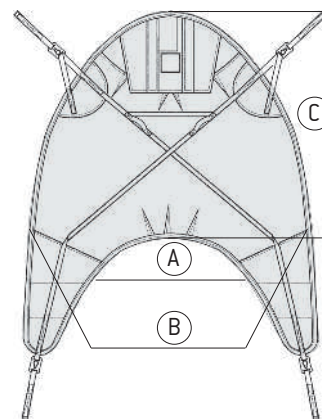

Learn More About This Product



\* Available in assorted lengths: 10cm/3.9" (21180-10), 15cm/5.9" (21180-15), 20cm/7.8" (21180-20), 25cm/9.8" (21180-25), 35cm/13.7" (21180-35)

<sup>1</sup> Only available in the United States

This product is manufactured in accordance with all relevant ISO standards.

