

Supreme™ Instructions

For PVC Reclining Bath Chair

Model Numbers:

BCCS-8510S (up to 100 lb. weight capacity)

BCCS-8610M (up to 130 lb. weight capacity)

BCCS-8710L (up to 180 lb. weight capacity)



These products covered by U.S. Patents:
D606,387; D606,388; D606,389; D592,492; 3,795,218

ATTENTION

Your Supreme Bath Chair has been built to the highest standards of quality to ensure years of superior service. Please take the time to read the informative instruction guide and pay particular attention to the safety information provided.

Here are some precautions you should take to reduce the risk of injury:

1. Read and understand ALL instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. To reduce the risk of injury or drowning, NEVER leave the user unsupervised or unattended while using this product.
4. Check regularly that all tubing, bolts, buckles, and other parts are tight at ALL times.
5. Check regularly that all PVC tubing and connectors are free from cracks or distortion. At the first sign of cracks or distortion, discontinue use immediately.
6. SAVE THESE INSTRUCTIONS.

Thank you for choosing Inspired by Drive.

Before You Begin

The following are assembly instructions for each of the sizes of Inspired by Drive Supreme PVC bath chairs. Follow ONLY the instructions that are specific to your particular model.

Models BCCS-8510S - Supreme Bath Chairs (Small)

The small Supreme bath chairs are shipped with the frame and seat unattached.

1. Reach inside the seat cover by opening the top flap of the mesh cover.
2. From the inside of the seat, insert the bolts through the pre-drilled holes in the plastic tubing, the through the holes in the plastic frame. The cap nuts should then be screwed to the bolt ends protruding on the outside of the frame.
3. Insert the tube at the rear of the seat back into the frame's larger tube by pressing the metal release button.
4. Additionally, insert the tube under the seat front into the frame's larger tube by pressing the metal release button.

Models BCCS-8610M - Supreme Bath Chairs (Medium)

The medium Supreme bath chairs are shipped with the seat already attached to the frame.

1. Insert the tube at the rear of the seat into the frame's larger tube by pressing the metal release button.
2. Additionally, insert the tube under the seat front into the frame's larger tube by pressing the metal release button.

Models BCCS-8710L - Supreme Bath Chairs (Large)

The large Supreme bath chair is shipped with the seat already bolted to the frame. Simply insert the tube at rear of the seat into the frame's larger tube by pressing on the metal release button.

Setting the Angle of Recline

1. Push the release button on the upright post. Slide the inner post up or down, until the bottom locks into the desired position.

Setting the Seat Angle

1. Push the release button on the post under the seat. Slide the inner post up or down until the button locks into the desired position (please note that large chair has two posts).

CE

BS Partnership
41 Dan-y-Coed Road
Cardiff
CF23 6N
UK

Inspired
by *drive*

Every Inspired by Drive product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.