
safety precautions

1. Consult your healthcare professional before using the device to ensure user safety, security and comfort
2. The device is only to be used by one user at a time
3. Do not stand on the device, and ensure the device is used on a flat surface
4. Do not exceed the maximum weight limit
5. Periodically check the device for signs of wear or accidental damage, for example loose fasteners, damaged framework / plastic parts or torn and worn upholstery. If problems are found, do not use until the device has been repaired.
6. The device should not be used whilst unattended by a competent adult.

cleaning procedure

The device is cleaned down with a damp cloth using mild detergent and/or disinfectant. For example, a mild chloride disinfectant (solution of 1000 parts per million) can be used. Excess water is then removed from the device using a towel or a compressed air tool. The device is then placed in a suitable environment to fully dry.

Limited 1 Year Warranty

Your Inspired by Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.



2017 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare



First Class Chair

Item # FC 2000N & FC4000N



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

first class chair

installation

1. Remove contents from carton.
2. Turn seat upside down and open knobs (A) by turning counter clockwise (figure 1).
3. Slide back support bracket (B) into red seat assembly on the underside of the the seat.
4. Tighten knobs (A) to secure back.

adjustments

Seat Depth

Loosen knobs (A), pull/push back to the desired position and tighten knobs to secure in place.

Seat Height

Loosen knobs (C) on each leg, pull/push inner leg to desired position and tighten knobs to secure in place. A 15° anterior or posterior tilt can be achieved by setting front and back legs at different heights. (figure 2)

Armrest Height

Loosen bolts (D) and adjust height as appropriate and tighten bolts (D). (figure 3) Armrests can be flipped up. (figure 4)

Lap Belt

Adjust belt length through buckles.

Lock Arms

To lock arms and prevent it from flipping up turn knob (E) (figure 5) clockwise until it locks the arm.

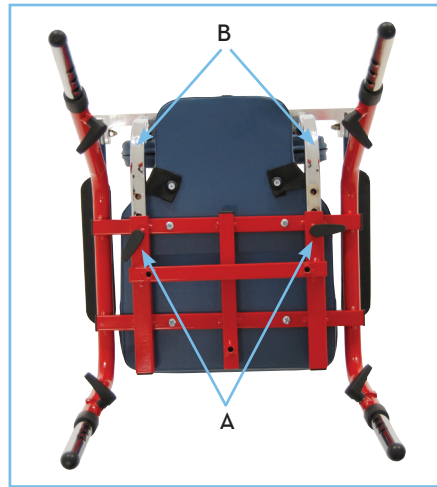


figure 1

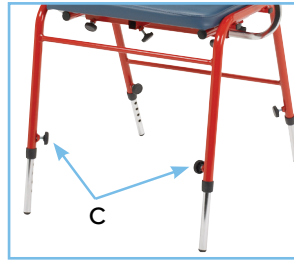


figure 2



figure 3



figure 4



figure 5

Use knob E to lock or unlock armrests



Item # FC 2000N

accessories (sold separately)

Description	Item #
Tray	FC 2024 & FC 4024
Footrest	FC 2028 & FC 4018
Headrest	FC 8000N
Hip Guides	FC 8027N
Legs with Casters	FC 2029N & FC 4029N
Leg Extensions	FC 8050N
Anti Tippers	FC 2030N & FC 4030N
Support Kit	FC 8025N & FC 8025LN
Trunk Harness- small	VTH 001N
Trunk Harness-large	VYH 001 LN