



Special solutions

# Immedia SlingOn


## A TRANSFER AID FOR APPLICATION OF LIFTING SLING AND POSITIONING

Immedia SlingOn is primarily designed for easy application of a hoist sling in bed without the need for logrolling or moving the user. SlingOn consists of two separate ultra low friction nylon slide sheets held together by snap buttons in the corners and it has handles along the sides. SlingOn is an ideal aid for users that needs frequent hoisting, heavy users, or users that should not be logrolled for application of a sling. In addition it can be used for all positioning in bed.

SlingOn is placed under the user using a simple folding technique, whether it is for the application of sling or for positioning. The hoist sling is then inserted between the two layers of SlingOn.

The ultra low friction facilitates sliding the sling in place underneath the user, and adjustment if necessary. When the hoist sling is in place, remove the top part of SlingOn.

*Note: The user is always lifted in the sling, never in SlingOn. Always remove the top part of SlingOn before hoisting.*

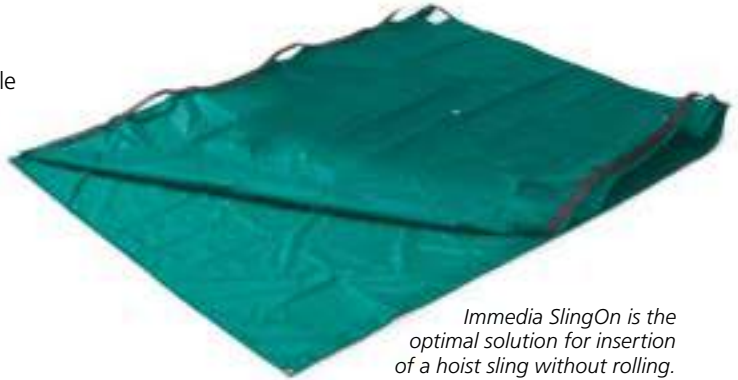

*Immedia SlingOn is the optimal solution for insertion of a hoist sling without rolling.*



*When working with bariatric patients and narrow surfaces, logrolling patients with a hoist sling can be challenging. The Immedia SlingOn allows you to use hoist slings without logrolling the patient.*

**RECOMMENDED  
USE!**



1. Roll up the Immedia SlingOn from the short side in 20-30 cm steps. Roll out the SlingOn underneath the user in stages. It is important to pull the SlingOn simultaneously.

2. When the Immedia SlingOn is in place, apply the hoist sling between the two layers of the SlingOn. The hoist sling is applied in the same way as the Immedia SlingOn. Pull from the head end towards the seat in small steps. When the hoist sling is in place, remove the top part of the SlingOn. The user can now be hoisted.

### TIP!

Use the Immedia SlingOn as traditional slide sheets. The two parts are equipped with handles to make repositioning easy.

The Immedia SlingOn can also be applied from knee level and towards the head end.



### INTENDED USE

- Application of a hoist sling without rolling
- Minimise friction under pressure points
- Turning
- Positioning
- Move higher up in bed

DESCRIPTION	WIDTH X LENGTH		ITEM NO.	SPECIFICATIONS
<b>Immedia SlingOn</b>	<b>CM</b>	<b>INCH</b>		MATERIAL: Nylon STRAP: Polyester Press buttons
~ Large	120 x 150	47.2 x 59.1	IM437	
~ Small	80 x 100	31.5 x 39.4	IM437/S	
~ Medium	80 x 170	31.5 x 66.9	IM438	
<b>WASHING INSTRUCTIONS</b>				
Can be wiped with disinfectant Do not use fabric softener or bleach.				