



# Illustrations

Illustration 1

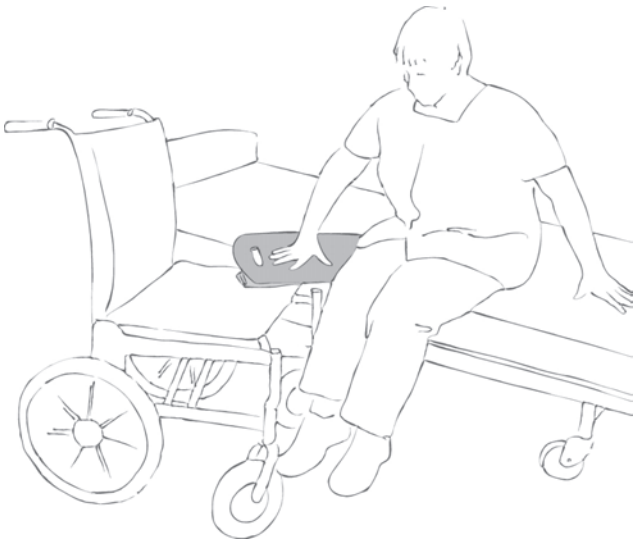


Illustration 2



Illustration 3



# Contents

	<b>Heading</b>	<b>Page</b>
1.	General/Features.....	4
2.	Practical handling.....	5
3.	Handling/Materials/Cleaning .....	6
4.	Size/Combinations .....	7

# General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.**

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



The product can be scrapped in accordance with national regulations



Wipe off



Do not bleach



Read the User Manual



Manufacturer



Tumbledry



Risk of sliding down



Batch no./Lot no.



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

# Features

The E-Board can be used to transfer clients from wheelchair to bed, bath/commode, car seat, etc., and vice versa.

# Use

---

## Positioning

---

Position the wheelchair at an angle of about 45° to the bed. Position the E-Board so that the grey non-slip strips on the backside lie securely against the wheelchair seat and the surface of the opposite surface.

Positioning is easier if the client either leans against the direction of movement or shuffles him/herself sideways on the mattress until he/she is seated on the glide board (Illustration 1).

---

## Use

---

The client can now slide/"shuffle" slowly over into the wheelchair by pushing him/herself away from the mattress (Illustration 2).

Depending on the client's resources, the carer can guide the client, supporting the upper body, possibly using a Support Belt, Smartmove or Sling.

---

## Tips

---

If a slope is established (e.g. if the bed is slightly higher than the wheelchair), the client will slide more easily from one to the other.

In connection with manual handling, it will often be an advantage to use a Pediturn turnplate for the feet and a Support belt.

To remove the E-Board, the client moves his/her weight away from the glide board, lifts and then pulls it free (Illustration 3).

# Use

## Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use correct manual handling techniques.  
Recommended free space: 15-20 cm.

Remember to use the client's resources.

Remember to apply brakes to lock wheelchair and bed.  
The client must have good balance.

There is a risk that the client may slide down.

Never leave the client alone on the edge of the bed.



Never use the E-Board under wet conditions as the underside of the board may become slippery in contact with water.

Never leave the E-Board on the floor.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

# Materials

The E-Board is made of polyethylene.

# Cleaning

The product should be checked regularly, preferably each time it is used.



**Never try to repair a product yourself. Never use a defective product.**

The E-Board can be wiped with soapy water or disinfectant.



## Articles

Article no.	Description	Size (cm)
IM404	E-Board	B33 x L60 x H0.5
IM405	E-Board	B33 x L75 x H0.6
IM408	E-Board	B25 x L60 x H0.5
IM410	E-Board	B20 x L45 x H0.5

## Combinations

Article no.	Description	Size (cm)
IM99836GT	Pediturn, turnplate for floor use	Ø40
IM99700	Pediturn, hard turnplate, for floor use	Ø38
IM8017015	Support belt, 11 handles, X-large	W15 x L170
IM8015013	Support belt, 8 handles, large	W13 x L150
IM8013013	Support belt, 6 handles, medium	W13 x L130
IM8010513	Support belt, 5 handles, small	W13 x L105
IM8010509	Support belt, 4 handles, small	W9 x L105
IM428	Sling, 2x4 handles	W20 x L185
IM425	SmartMove, 2 handles	W21 x L60