

# MiniBoard

Manual English

 etac®



# Illustrations

Illustration 1


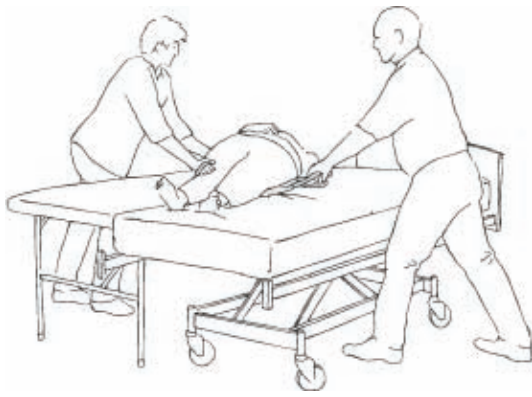


Illustration 2

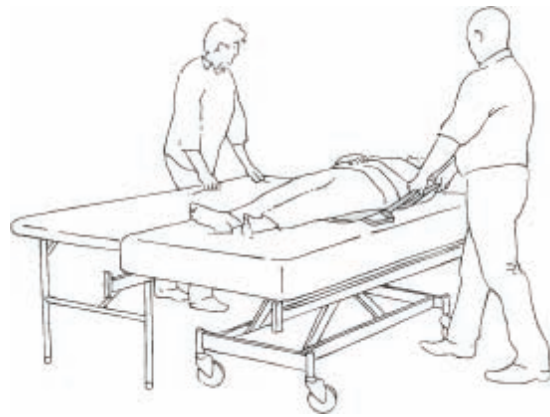

# Contents

	<b>Heading</b>	<b>Page</b>
1.	General/Features.....	4
2.	Practical handling.....	5
3.	Handling/Materials/Cleaning .....	6
4.	Size/Combinations .....	7

# General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the client" in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.



**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the client or carer.**

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

The product can be scrapped in accordance with national regulations.



Washing



Iron



Never leave on the floor



Max. weight



Handwash



Do not dryclean



Warning



The product can be scrapped in accordance with national regulations



Wipe off



Do not bleach



Read the User Manual



Manufacturer



Tumbledry



Risk of sliding down



Batch no./Lot no.



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

# Features

MiniBoard can be used in manual handling situations, such as turning a client in bed, moving a client higher up in bed, from bed to bed, and when lifting a client from the floor. MiniBoard is available in two models: one with 7 handles and one with 11 handles. Both models are equipped with an extra handle at the head end providing ideal support for the client's head.

# Use

---

## Positioning

---

To position the MiniBoard under the client: Roll the client slightly against the direction of movement. Then fold the functional handles alongside the client in under the MiniBoard, and slide the MiniBoard in under the client. Roll the client back and then roll him/her gently in the opposite direction and fold out the functional handles.

---

## Use

---

The carer now grasps the functional handles and can move the client in the bed, from one bed to another, from lying to seated and then standing, etc. (Illustrations 1 and 2).

If MiniBoard is used to lift a client from the floor, carers should assess working conditions and space available vis-à-vis correct manual handling techniques and the total weight that can be lifted responsibly.

MiniBoard can be easily removed by moving the functional handles to one side in under the MiniBoard, rolling the client gently to this side, and then pulling MiniBoard out on the opposite side.

# Use

## Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling.

Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Remember to use the client's resources.

MiniBoard is not suitable for lifting with a lift.

There is a risk that the client may slide down. Never leave the client alone on the edge of the bed.



Never leave the client on the MiniBoard unless there is a guard rail.

Never leave MiniBoard on the floor.



To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

If in any doubt - please contact Etac.

# Materials

Top side: 100% nylon.

Inner surface: synthetic board (removable).

Handles: polyester.

## Articles

Article no.	Description	Size (cm)
IM8012454	MiniBoard, 7 handles	W54 x L124
IM8020054	MiniBoard, 9 handles	W54 x L200

## Cleaning

The product should be checked regularly, preferably each time it is used.

Check that seams and fabric are not damaged.



**Never try to repair a product yourself. Never use a defective product.**

### **Washing instructions:**

MiniBoard can be wiped off with disinfectant.  
Remove board before washing.  
Do not use fabric softener.

