


Liko Slings - Basic Models

Quick Reference Guide

English

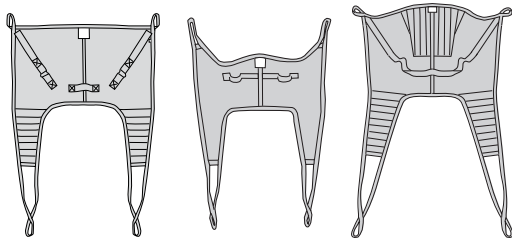

Mod. 00, 02


Mod. 10, 11

Mod. 20, 21, 26


Safety Instructions

- Read the instruction guides for the lift and lifting accessories before use.
- Ensure that the sling selected meets the patient's needs with reference to model, size, fabric and design.
- For safety and hygiene reasons, use individual slings.
- Plan the lifting operation carefully.
- Never leave a patient unattended during a lifting situation.
- Before the patient is lifted from the underlying surface, but after the straps have been fully extended, make sure the straps are properly connected to the sling bar.
- Maximum load, See the sling's product label. To retain max. load it is essential to use lift/accessories rated for the same max. load or greater.
- Check the sling regularly, especially after washing.


Choosing Size

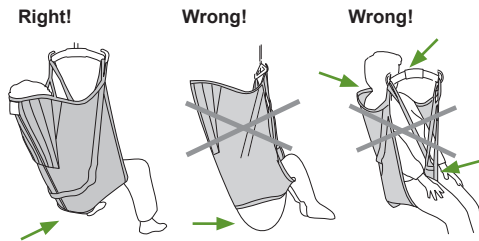

Lifting from a Sitting Position

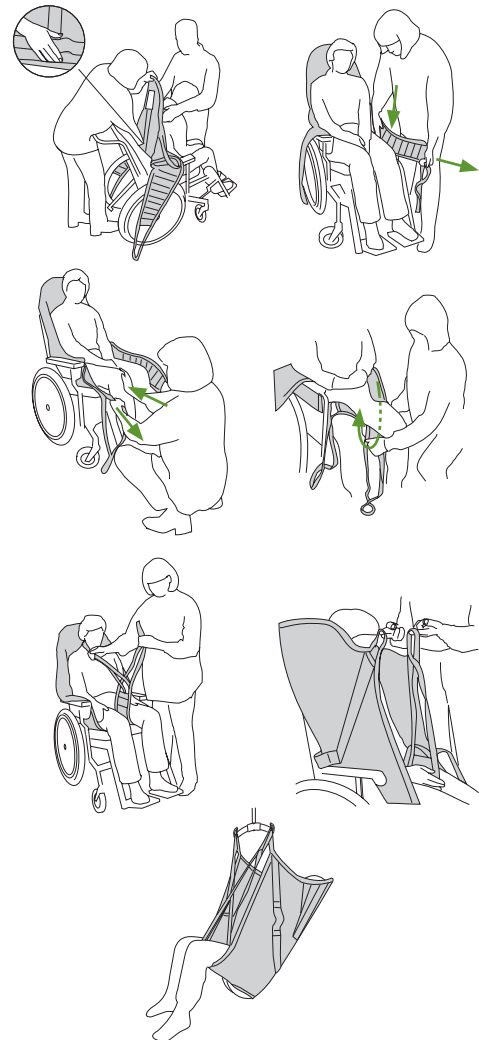


Lifting to a Sitting Position

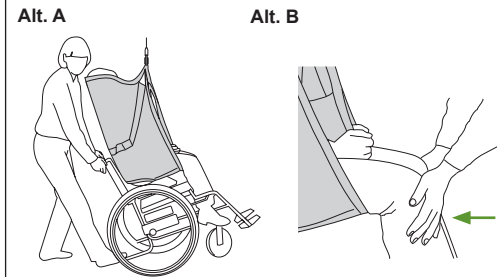


⚠ Do not use the handles to lift!

Lifting from the Bed

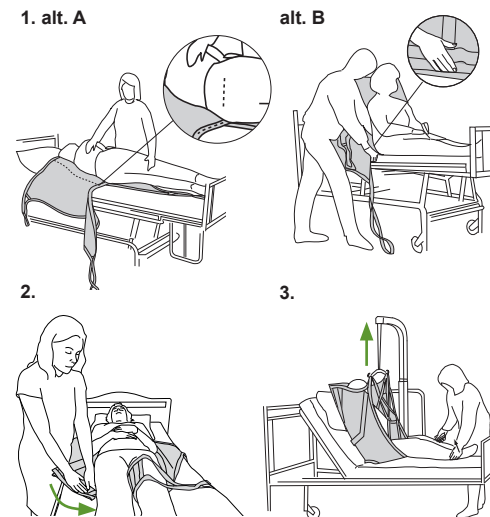

Lifting from the Floor

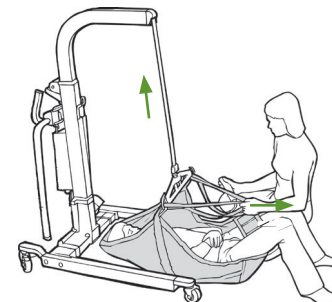

⚠ The mobile lift's wheels should be locked.

After the Transfer

The sling is preferably removed after the transfer, in other case we recommend using a sling in net polyester.

Washing Instructions

Read the washing instructions on the slings product label before washing!

Ergonomics

- Plan the lifting operation. Work calmly and methodically.
- Don't strain your back! Work in a walking stance with your back straight.
- Use the bed's raising and lowering functions.
- Reduce the lever effect. Work close to the patient and the lift.

© Copyright Liko AB



A Hill-Rom Company