

See exploded view
Base frame

Z:\SW-Tek\Elexo XXL 5300\

Signed	ES	Date	10-10-2013
Revised	1 /KA	Date	05-01-2015
Revised	- /	Date	-


Copyright reserved
by law

unit measure-
ment: mm

Overview parts Elexo XXL

Lopital Nederland bv

Type nr: 51005300

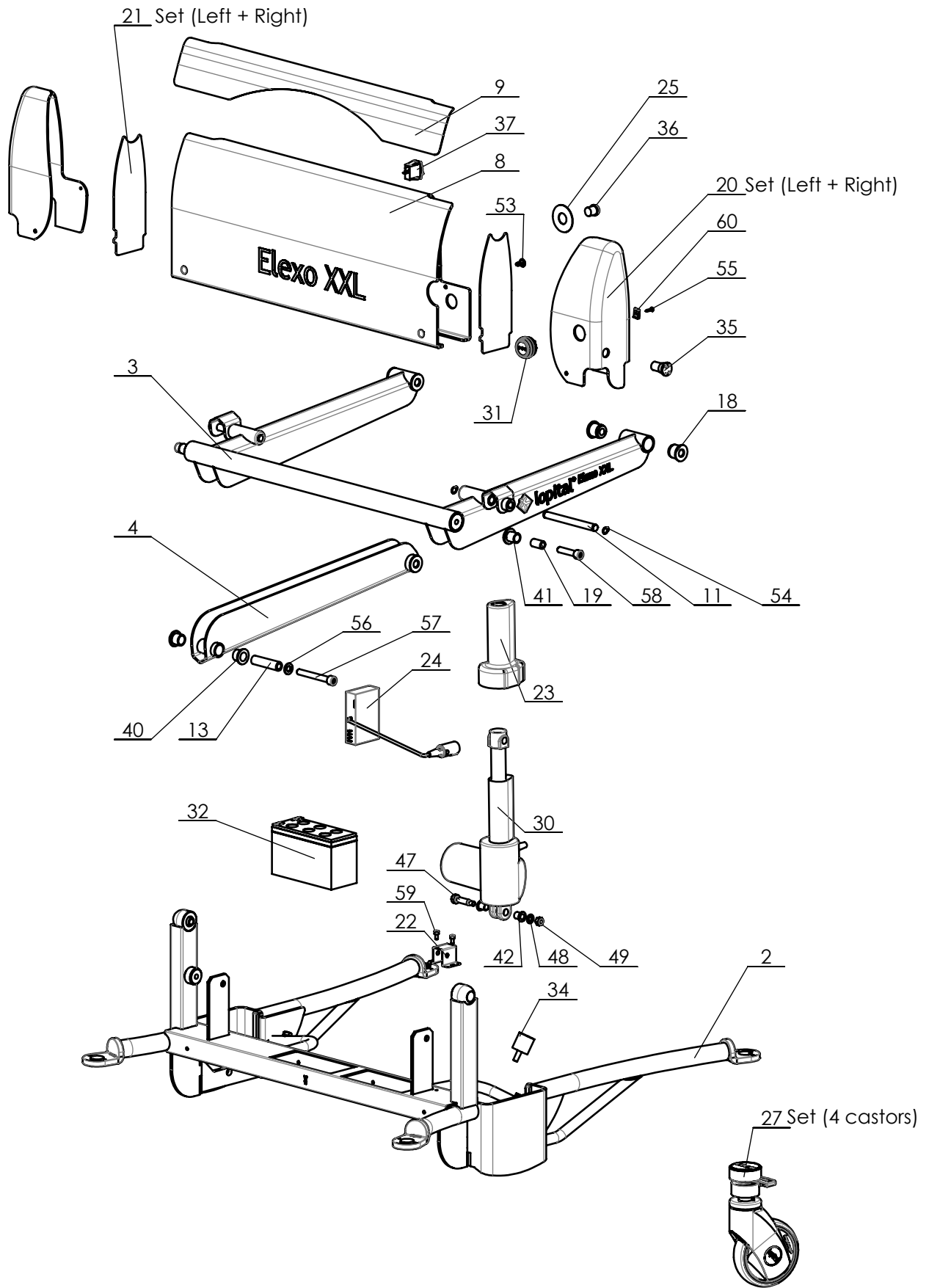

Z:\SW-Tek\Elexo XXL 5300\

Signed	ES	Date	10-10-2013
Revised	1 /KA	Date	05-01-2015
Revised	- /	Date	-

Overview parts Elexo XXL base frame

Copyright reserved by law

unit measurement: mm

Lopital Nederland bv

Type nr: 51005300

Elexo XXL 51005300

Part nr.	QTY.	Art.nr.	Part
1	1	70550002	Seat frame
2	1	70550001	Frame
3	1	70550003	Main lever
4	2	70550004	Control lever
5	1	70530005	Armrest (L+R)
6	2	75530017	Rosette
7	1	70530006	Footrest
8	1	70530018	Battery cover
9	1	70530019	sticker motorcover
10	4	70530007	Hinge pin L=100
11	2	70530016	Hinge pin L=127
12	2	70530027	Hinge cover blue
13	2	70530024	Bush L=63 mm
14	1	70530032	Footplate (L+R)
15	1	70530008	Seat(L+R)
16	1	70530009	Backrest
17	1	70530012	Calf plate
18	8	70560041	Bush
19	2	70519923	Bush
20	1	70550011	Accu-eindkap (set)
21	2	70550019	Seperation plate
22	1	70550021	Bracket coupling
23	2	70550020	Cover for TA1
24	1	70550013	Uni-lop Control
25	1	70550043	Sticker ELD
26	4	70519931	Leg support clamp
27	4	70530017	Castor XXL
28	1	69501104	Froli bad pan + lid
29	1	70550012	Charger incl. plug
30	2	70550006	Actuator TA1
31	1	70550018	Chassispart charger
32	2	70560004	Battery Set
33	1	70550007	Manual control
34	2	70914104	Vibration damper
35	1	70210517	Emergency stop
36	1	70550014	Emergency lowering button
37	1	70550045	Switch On/Off
38	2	70530029	Pull knob
39	2		Tube cap
40	4		Slide bearing
41	2		Slide bearing

Signed	ES	Date	10-10-2013
Revised	1 /KA	Date	05-01-2015
Revised	- /	Date	-

Overview parts Elexo XXL

Copyright reserved
by law


unit measure-
ment: mm

Lopital Nederland bv

Type nr: 51005300

Elexo XXL 51005300

Part nr.	QTY.	Art.nr.	Part
42	4		Slide bearing
43	4		Washer
44	2		Countersunk screw
45	4		Tube clamp
46	4		Cylinder head screw
47	2		Socket head screw
48	2		Washer
49	2		Lock nut
50	12		Capring
51	12		Chipboard screw
52	12		Screw cap
53	4		Push in Rivet
54	12		Retaining ring
55	2		Chipboard screw
56	2		Washer
57	2		Cylinder head screw
58	2		Cylinder head screw
59	2		Cylinder head screw
60	2		Snapnut
61	4		Cylinder head screw
62	4		Hex bolt

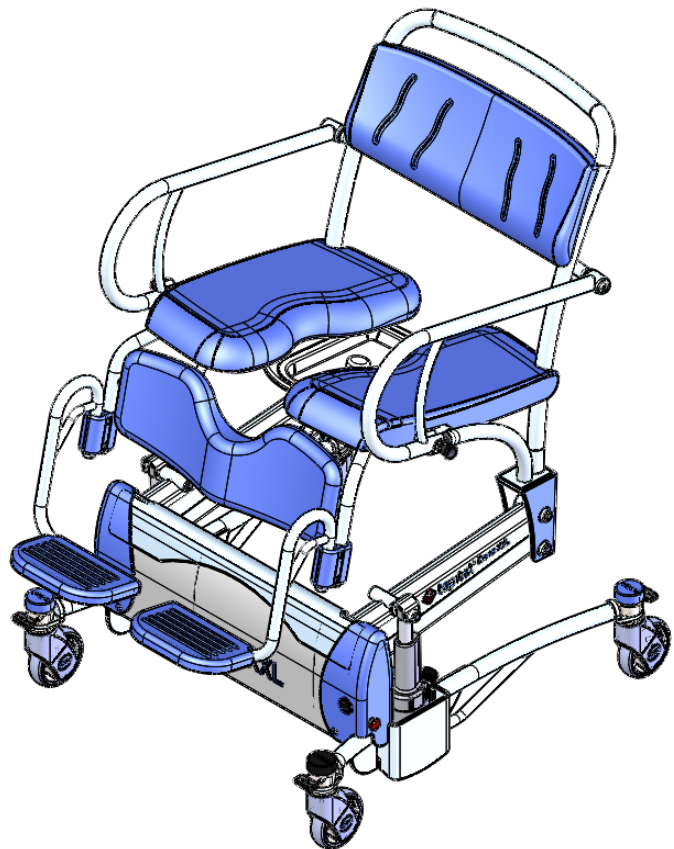

Z:\SW-Tek\ Elexo XXL 5300\

Signed	ES	Date	10-10-2013
Revised	1 /KA	Date	05-01-2015
Revised	- /	Date	-

Overview parts Elexo XXL

Copyright reserved
by law

unit measure-
ment: mm

Lopital Nederland bv

Type nr: 51005300