


# Marina Electric Shower Trolley

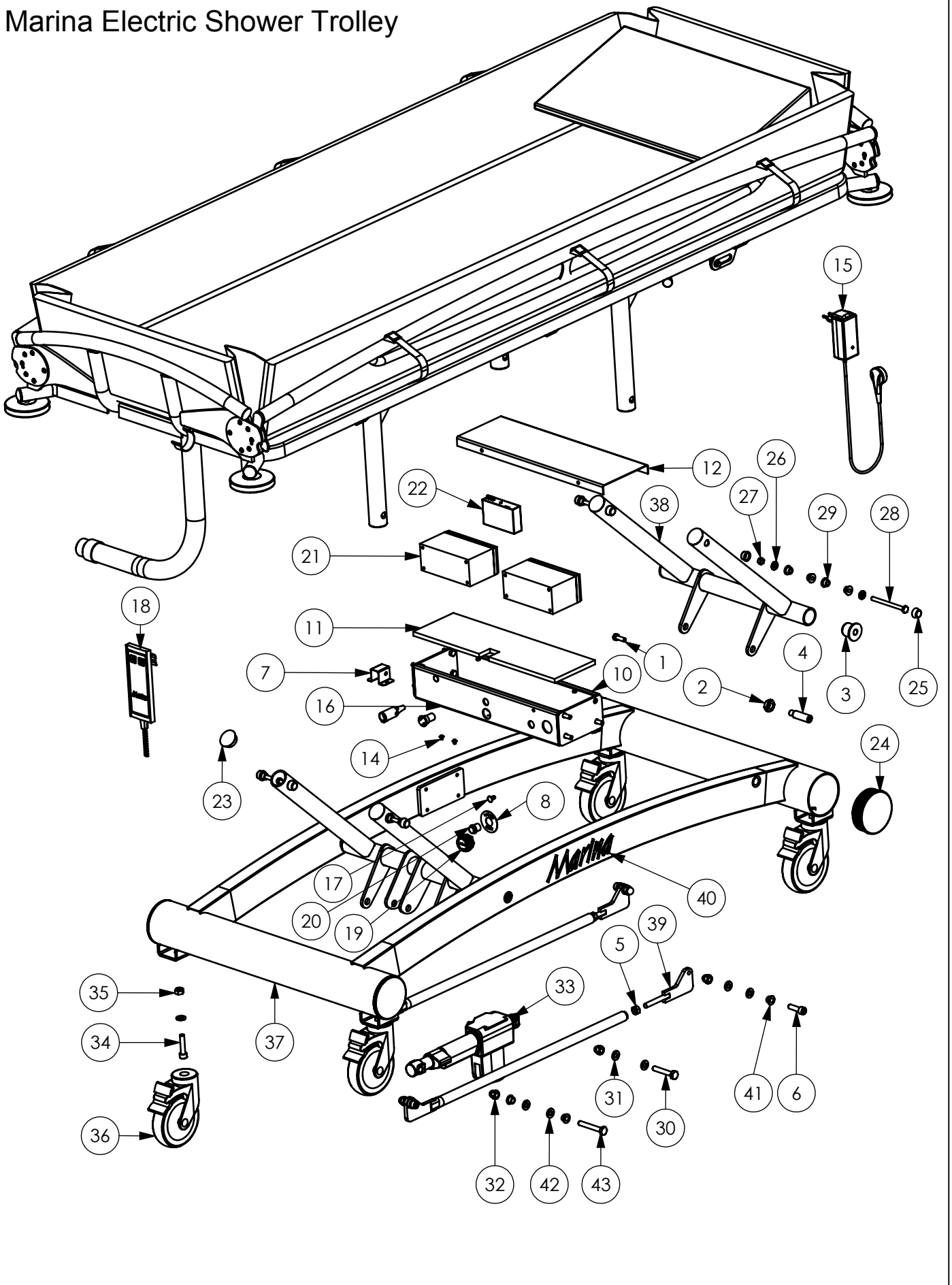

Z:\SW-Tek\Marina 2300\2300 70x200\

Signed: ES	Datum: 02 - 2011
Revised: 3 /ES	Datum: 06-01-2013
Revised: 2 /ES	Datum: 09 - 2011
Copyright reserved by law	Unit of measurement: mm

## Overview Parts

Lopital Nederland bv

Type nr: 61002350

## Marina Electric 61002350


Stuknr.	Art.nr.	Elektrisch/ QTY.	Name	Material	Opm./ Tek.nr.
1		8	Hex Bolt	A2	M8x25 DIN933
2		4	low hex nut	A2	M16 DIN439B
3	70230008	4	jack lower	POM	2300-17.1
4	70307001	4	Custom screw	A4	3070-18
5		2	Hex Nut	A2	M10 DIN 934
6		4	Cil.kop screw bzk	A2	M10x35 DIN912
7	70550021	1	holder clutch	RVS 304	5500-23
8	70550043	1	Sticker ELD		5500-93
9	70235007	1	cablaset Complete		-
10	70235002	1	electronics box		2350-12.5
11		1	padding electronics box	V0	-
12	70235003	1	Cover electronics box	St 37	2350-14.2
13	70235005	1	Sticker charger	-	2350-15
14		2	low round head screw BZK	A2	M5x6 ISO7380
15	70550012	1	charger incl. MAG		-
16	70210517	1	emergency stop	Kunststof	-
17		4	Press stud	PA	Druknagel
18	70235010	1	Hand Control	Kunststof	-
19	70550018	1	Socket charger	Kunststof	-
20		1	Emergency lowering button		-
21	70560004	2	Batteries	Div.	Accu 12V 7Ah
22	70550013	1	Control Box	ABS + Div	-
23	VP 6900040	4	Cap ø40 s=3/5	PE grijs	Insteekdop rond 40 grijs
24		4	Cap ø100	Kunststof	Insteekdop - 100 mm
25		8	cover	PA (grijs)	Afdekkap M8
26		8	cap ring	PA	Kapring M8
27		4	locknut	A2	M8 DIN985
28		4	Hex Bolt	A2	M8x100 DIN931
29		16	Bearing with collar	PA	08x14x10
30		1	Hex Bolt	A2	M10x60 DIN931
31		6	washer	A2	M10 DIN125
32		6	Cap nut M10	A2	M10 DIN1587
33		1	TA7 Actuator	Div.	-
34		4	Cil.kop screwbzk	A2	M10x45 DIN912
35		4	locknut	A2	M10 DIN985
36	70519928	4	Wheel 125 w/brake	Kunststof	-
37	70230001	1	chassis	Alu gec.	2300-11
38	70230003	1 + 1	jack set	Alu	2300-12.2
39	70230005	1 + 1	Tie Rod Set	St + RVS gec.	2300-15
40	70230022	2	Sticker Marina		2300-46
41		6	Bearing with collar	PA	10x14x10
42		10	washer	PA	M10 DIN125 PA
43		1	Hex Bolt	A2	M10x70 DIN931

Signed: ES	Datum: 02 - 2011
Revised: 3 /ES	Datum: 06-10-2013
Revised: 2 /ES	Datum: 09 - 2011

# Overview Parts

copyright reserved by law	Unit of measurement: mm	<b>Lopital Nederland bv</b>	Type nr: 61002350-SL-1
---------------------------	-------------------------	-----------------------------	------------------------

# Marina Electric

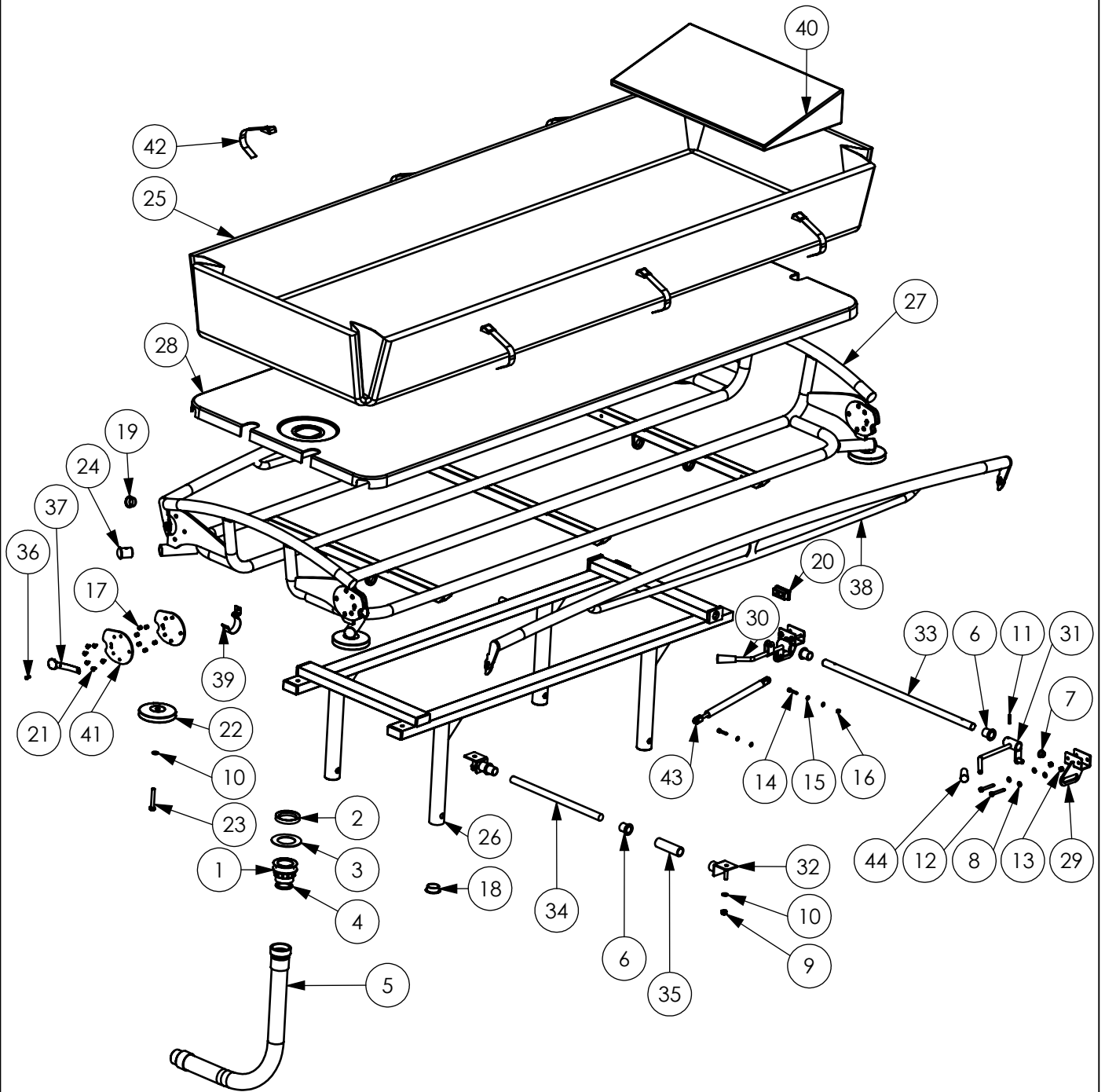

Z:\SW-Tek\Marina 2300\2300 70x200\

Signed: ES	Datum: 02 - 2011
Revised: 3 /ES	Datum: 06-01-2012
Revised: 2 /ES	Datum: 10 - 2011

## Overview Parts

Copyright reserved by law  
Unit of measurement: mm

Lopital Nederland bv

Type nr: 61002300/2350

# Marina Electric

Uppers 61002300 en -2350

Stuknr.	Art.nr.	Uppers/QTY.	Name	Material	Opm./ Tek.nr.
1	70210055	1	outlet	PP	2100-67a
2	70210056	1	clamping ring	POM	2100-66.1
3	70210043	1	clamping disc	Vikupor (wit)	2100-68
4	70210058	1	grommet	Rubber	-
5	70210054	1	Hose (complete)	PVC	-
6		4	Bearing with collar	PA	18x22x25
7		2	Bearing with collar	PA	10x14x10
8		8	washer	A2	M6 DIN 125 A2
9		2	locknut	A2	M8 DIN985
10		6	washer	A2	M8 DIN125 A2
11		2	cylindrical pin	A1	5x24 DIN 7
12		4	Hex Bolt Locknut	A2	M6x45 DIN931
13		4	locknut	A2	M6 DIN985
14		2	Cil.kop screw bzk	A2	M5x25 DIN912
15		4	washer	A2	M5 DIN 125 A2
16		1	locknut	A2	M5 DIN985 A2
17	70220027	24	Stainless Spacer	RVS 304	2100-4902
18	VP 6900040	4	Cap ø40 s=3/5	PE grijs	Insteekdop rond 40 grijs
19	VP 6900030	4	Capø30	PE	Insteekdop rond 30
20	VP 6235025	6	Cap 50x25 l	PE grijs	Insteekdop vierkant 50x25
21		48	low round head screw	A2	M5x8 ISO7380
22	70220041	4	Bumper	PVC + PP	-
23		4	Hex Bolt	A2	M8x55 DIN933
24	70220040	4	Plug	PVC grijs	2200-34.1
25	70220016	1	mattress	PERL violet	-
26	70230006	1	Subframe	Alu gec.	2300-21.4
27	70230007	1	Bovendeel	Alu gec.	2300-31.2
28	70220014	1	Top Plate	ABS	2300-32.2
29	70230012	2	U-bracket		2300-33
30	70230014	1	Tilt Lever right	St ELVZ	2300-34
31	70230015	1	Tilt Lever left	St ELVZ	2300-35
32	70230013	2	axle holder		2300-36
33	70230016	1	Long axis tilt adj.	St 37	2300-38
34	70230017	1	Short axis tilt adj.	St 37	2300-39
35	70230018	1	Bus	PVC	2300-40
36	70210039	2	locking part	RVS 304	2100-58.2
37	70210041	2	Siderail relase lever	RVS	-
38	70220004	2	side bar	Alu gec.	2200-14.1
39	70220028	1	hose Hook cushion	RVS Verenstaal	2200-30.1
40	70210048	1	Head Pillow	PVC	2200-32
41	70220023	8	Locking plate	Aluminium	2200-38
42		6	Strap with buckle		-
43	70210040	1	Gas Spring	St / RVS	150mm 200N BSP
44	70210053	2	cap	Kunststof	-

Z:\SW-Tek\Marina 2300\2300 70x200

Signed: ES	Datum: 02 - 2011
Revised: 3 /ES	Datum: 06-01-2013
Revised: 2 /ES	Datum: 10 - 2011
Copyright reserved by law	Unit of measurement: mm

## Overview Parts

Lopital Nederland bv

Type nr: 61002300/2350-SL