

The Roosevelt

**“EZ-Up”
Head Rest
Instructions**



Table of Contents

Warnings	3
Head Rest Details	4
Cap Details	5
Installing the Head Rest	6
Head Rest Installation (Lower Two Sets of Slots)	8
Head Rest Installation (Upper Two Sets of Slots)	9
Head Rest Installation (Middle Set of Slots)	10
Securing the Cap on Your Child's Head	11
Removing and Re-installing Your Child in the "EZ-Up" Head Rest	12
Care and Cleaning	13
Fabric Care	13
Warranty	14

WARNING!
**FAILURE TO COMPLY WITH THE
FOLLOWING WARNINGS CAN RESULT
IN DEATH OR SERIOUS INJURY TO
YOUR CHILD:**

- This product does not replace adult supervision for a child who is at risk of breathing difficulty if his/her head falls forward.
- **Your child’s head may slip out of this cap!**
- This product is only intended to **help** you keep your child’s head in an upright position.
- This product is for comfort **only**.
- Adult supervision is required 100% of the time when using the *Roosevelt*.
- These are supplemental instructions which **only** describe how to use the “EZ-Up” Head Rest. Please read and follow all instructions on this child restraint and in the written instructions located in the pocket on the side of the restraint.
- The “EZ-Up” Head Rest described in this instruction booklet is only to be used with the *Roosevelt* child safety seat manufactured by Merritt Manufacturing, Inc. **Do not use with any other child restraint system.**

Head Rest Details



Cap Details



Velcro® Adjustment Tab



Velcro® (Designed to Attach to Head Rest)

Installing the Head Rest


Note: The lower straps on the head rest must always be inserted through the same slots as the harness straps (See Step 2 below).




1. Install the harness straps into the appropriate harness slots as specified on Pages 18 and 19 in the “Instructions for Use” manual, which can be found in the pocket on the right hand side of the Roosevelt.



2. Insert both of the head rest's lower attachment straps (over the harness straps) through the set of slots selected in Step 1 (Figure A).
3. Pull the lower attachment straps through to the back side of the restraint (Figure B).

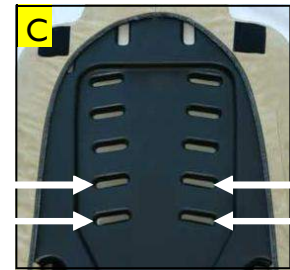

4. If the lower attachment straps were installed in one of the lower two sets of slots (Figure C), then proceed to Step 5. Otherwise (Figure D), skip to Step 6.
5. Skip two sets of harness slots; and insert both of the head rest's upper attachment straps through the third set of slots (Figure E) and pull them through to the back of the restraint.

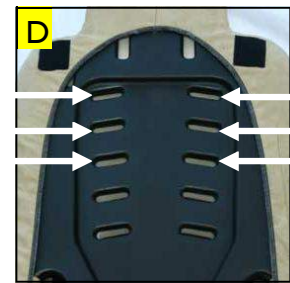

Installing the Head Rest, Continued


There should be two sets of empty slots between the upper and lower head rest straps.

6. If the lower attachment straps were installed in one of the upper three sets of slots (Figure D, opposite page), then lay the headrest's upper attachment straps over the top of the restraint (Figure F).
7. Velcro[®] one of the upper attachment straps to its corresponding lower attachment strap on the back side of the restraint (Figure G).
8. Secure the end of the upper attachment strap to the Velcro[®] patch located on the backside of the cover to keep it from interfering with the harness (Figure H).
9. Repeat Steps 7 and 8 for the other set of attachment straps (Figure I).

Note: Figures H and I depict the back of the restraint when the lower straps are installed in the middle slots. Refer to the photos on Pages 8 - 10 for clarification of the various head rest installations in alternate harness slots.



Head Rest Installation with Lower Straps in One of the Lower Two Sets of Slots

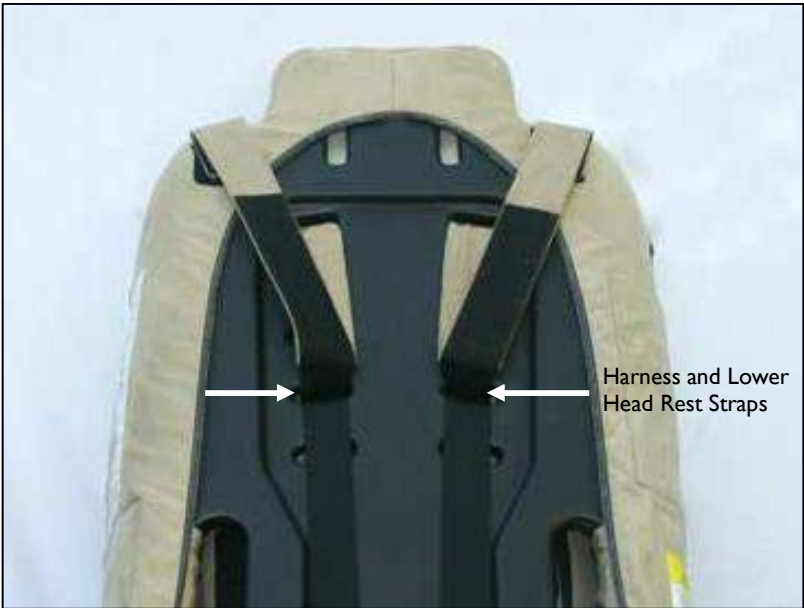

Head Rest Installation with Lower Straps in One of the Upper Two Sets of Slots




Head Rest Installation with Lower Straps in the Middle Set of Slots



Securing the Cap on Your Child's Head

Note: The Roosevelt must be installed in your vehicle according to the “Instructions for Use” manual located in the pocket on the right hand side of the Roosevelt.

1. If necessary, gather your child's hair to ensure that it does not interact with the Velcro[®] on the back of the cap.
2. If attached, unhook the Velcro[®] adjustment tab (Figure A).
3. Place the cap on your child's head, with the Velcro[®] on the outside of the cap and positioned at the back of her head (Figure B).
4. Tighten the cap for a snug fit using the Velcro[®] adjustment tab (Figure C).
5. Secure your child in the Roosevelt according to Pages 44 and 45 in the “Instructions for Use” manual.
6. Place your child's head against the head rest so that the mating Velcro[®] surfaces (on the head rest and cap) contact and adhere to each other (Figure D).

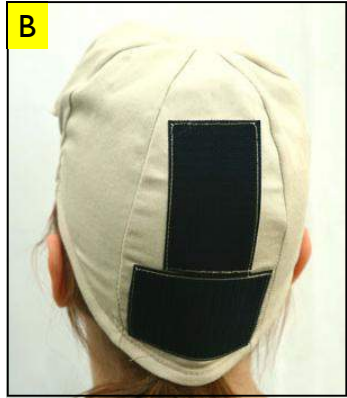

Removing and Re-installing Your Child in the “EZ-UP” Head Rest

Removing Your Child

1. Release the Velcro[®] adjustment tab to let your child’s head out of the cap (Figure A).
2. Remove your child’s head from the cap (Figure B), without removing the cap from the head rest (Figure C).
3. Remove your child from the Roosevelt by performing Steps 5-7 (Pages 44 and 45 in the “Instructions for Use” manual located in the pocket on the right hand side of the Roosevelt) in reverse.



Re-Installing Your Child

1. Secure your child in the Roosevelt according to Pages 44 and 45 in the “Instructions for Use” manual.
2. Place your child’s head in the cap, without removing the cap from the head rest (Figure B).
3. Tighten the cap for a snug fit using the Velcro[®] adjustment tab (Figure A).



Care and Cleaning

Fabric Care

1. After removal, clean the head rest and/or cap separate from any other clothing per Steps 2-4.
2. As much as possible, Velcro[®] the mating tabs on the head rest together so that the Velcro[®] does not fray the fabric during cleaning.
3. Machine wash on gentle cycle using mild soap (detergent) with the “wash” and “rinse” cycles set on “cold”.
4. Machine dry in dryer on “no heat” or “fluff dry” cycle. Add a dry bath towel (an older one that is unlikely to “bleed” on the cover) to expedite the drying process.
5. Do not iron the head rest.

Limited Warranty

Merritt Manufacturing, Inc. (Merritt) provides the following limited warranty on the Roosevelt:

Merritt warrants to the original purchaser of the Roosevelt that, if properly used strictly in accordance with the instructions and specifications included in the original packaging, it shall be free from defects in materials or workmanship for a period of one (1) year from the date of original purchase. Proof of purchase is required, including, but not limited to furnishing to Merritt proof that the owner registration card was returned to Merritt within thirty days of the original purchase or furnishing to Merritt proof of purchase that identifies the claimant as the original purchaser. If any component covered by this warranty fails for reasons covered by this warranty, Merritt will, at its option, provide the original purchaser with repair or replacement components for the covered product or component. This limited warranty is available only for purchasers in the United States and Canada. Merritt reserves the right to discontinue or change fabrics, parts, models or products, or to make substitutions. Please note that the upholstery fabric is subject to some degree of wear as a result of normal use. This kind of normal wear is not covered under warranty.

To make a claim under this warranty, you must contact Merritt via telephone at (317) 409-0148 or write to the company using the address on the back cover of this booklet. PLEASE COMPLETE AND MAIL THE OWNER REGISTRATION CARD WITHIN THIRTY DAYS OF PURCHASE TO PRESERVE YOUR WARRANTY RIGHTS.

WARRANTY LIMITATIONS

This warranty does not include damages due to external sources, including, but not limited to accident, negligence, misuse, neglect, alteration, repair, improper installation, improper testing or use that is not in conformance with the instructions and specifications in the instruction booklet.

LIMITATION OF LIABILITY AND DAMAGES

THE WARRANTY AND REMEDIES SET FORTH ABOVE ARE THE SOLE AND EXCLUSIVE REMEDIES AND ARE PROVIDED IN LIEU OF ALL OTHERS, ORAL OR WRITTEN, EXPRESS OR IMPLIED. IN NO EVENT WILL MERRITT, OR THE RETAILER/DISTRIBUTOR SELLING THIS PRODUCT, BE LIABLE TO THE PURCHASER, OR ANY USER OF THIS RESTRAINT OR OTHER THIRD PARTY, FOR ANY DAMAGES; INCLUDING LOST PROFITS, LOST BUSINESS, INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL DAMAGES OR PUNITIVE DAMAGES. IN NO EVENT SHALL MERRITT OR ITS AFFILIATED COMPANIES BE LIABLE OR RESPONSIBLE FOR ANY

Limited Warranty, Continued

AMOUNT OF DAMAGES ABOVE THE AGGREGATE DOLLAR AMOUNT PAID BY THE CUSTOMER FOR THE PURCHASE OF THE PRODUCT COVERED BY THIS WARRANTY.

LIMITATIONS OF WARRANTIES AND OTHER WARRANTY TERMS AND STATES' LAW

MERRITT FULLY DISCLAIMS AND MAKES NO IMPLIED WARRANTIES OF ANY KIND, INCLUDING, BUT NOT LIMITED TO IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR USE, AND MAKES NO EXPRESS WARRANTIES BEYOND THOSE STATED HERE. NOTWITHSTANDING THE FOREGOING, IF ANY IMPLIED WARRANTIES ARE DEEMED TO BE GIVEN BECAUSE THE APPLICABLE LAW PROHIBITS THEIR EXCLUSION, THEN ANY SUCH IMPLIED WARRANTIES, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED TO THE DURATION AND TERMS OF THE EXPRESS WRITTEN WARRANTY. SOME STATES DO NOT ALLOW THE LIMITATION OF IMPLIED WARRANTIES, LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS OR THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES. THEREFORE, IN THOSE STATES, THE LIMITATIONS LISTED ABOVE MAY NOT APPLY TO YOU. THIS WARRANTY AFFORDS YOU WITH SPECIFIC LEGAL RIGHTS AND YOU MAY HAVE ADDITIONAL RIGHTS, DEPENDING ON THE STATE IN WHICH YOU RESIDE.

BINDING ARBITRATION

Any dispute, controversy or claim (whether in contract, tort or otherwise, whether pre-existing, present or future, and including statutory, common law, intentional torts and equitable claims) against Merritt, its agents, employees, successors, assigns or affiliates (collectively for purposes of this warranty, "Merritt"), arising from or relating to this warranty and agreement, its interpretation, or the breach, termination or validity thereof, the relationships which result from this warranty and agreement (including, to the full extent permitted by applicable law, relationships with third parties who are not parties to this warranty and agreement), SHALL BE RESOLVED EXCLUSIVELY AND FINALLY BY BINDING ARBITRATION ADMINISTERED BY THE AMERICAN ARBITRATION ASSOCIATION under its code of procedure then in effect. Any award of the arbitrator shall be final and binding on each of the parties and may be entered as a judgment in any court of competent jurisdiction.