

# The Roosevelt

## Chest Pad



## Important Notes

**NOTE:** This chest pad is designed to help prevent the occupant from escaping from the Roosevelt.

**NOTE:** Never leave an occupant unattended in the Roosevelt!



**NOTE:** This chest pad replaces the chest clip. One of them must be used at all times.

**NOTE:** This Chest Pad (Figure A) must only be used with the “Roosevelt”, manufactured by Merritt Manufacturing, Inc.



## Fitting the Chest Pad to the Occupant

1. Position the chest pad on a flat surface with the back side facing up, as shown in Figure B.
2. Open the top Velcro tab to expose the top of the zipper (Figure C).
3. Lift both of the Velcro tabs on the end of the zipper (Figure D) and place the end of the zipper up out of the way (Figure E).

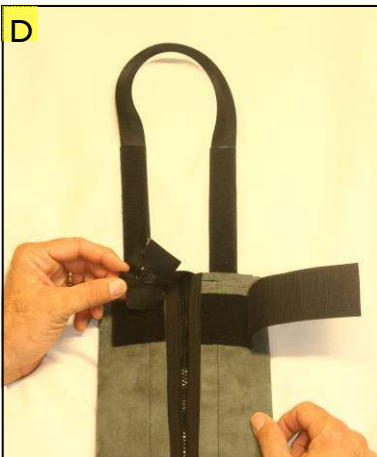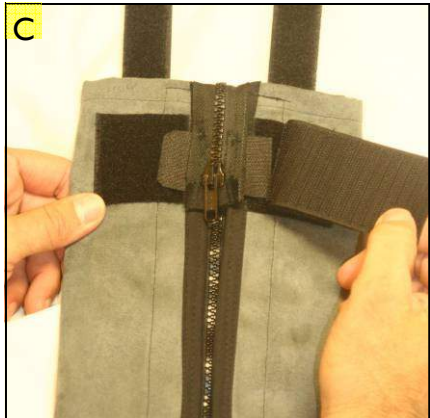

## Fitting the Chest Pad to the Occupant

4. Position the chest pad on a flat surface with the front side facing up, as shown in Figure F.
5. Completely unzip the zipper that connects the two halves of the chest pad (Figures G and H).





## Fitting the Chest Pad to the Occupant

6. Place the neck strap over the occupant's neck, allowing the chest pad to hang over his/her chest (Figure I).
7. Determine the appropriate length off the neck straps so that the top of the chest pad is positioned level with the occupant's arm pits.
8. Perform the steps on Pages 6 - 9 to adjust the neck straps to the correct length (referring back to the steps on this page as necessary).





## Fitting the Chest Pad to the Occupant, Cont.

1. Position the chest pad on a flat surface with the back side facing up, as shown in Figure A.
2. Grasp a corner of the Velcro flap on the left (Figure A) and lift until the entire side of the flap is detached from the velcro (Figure B).

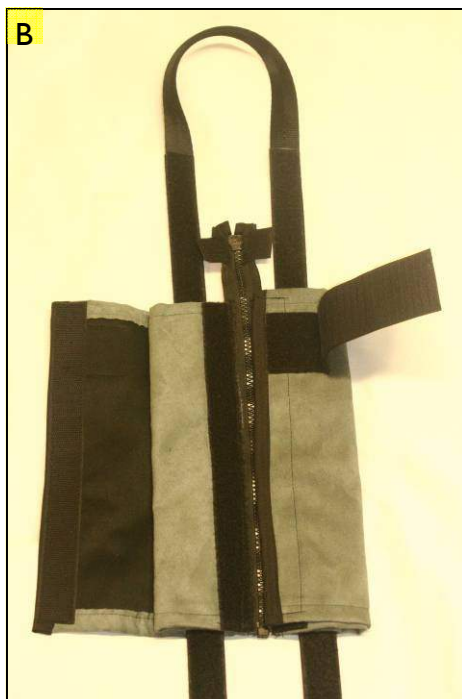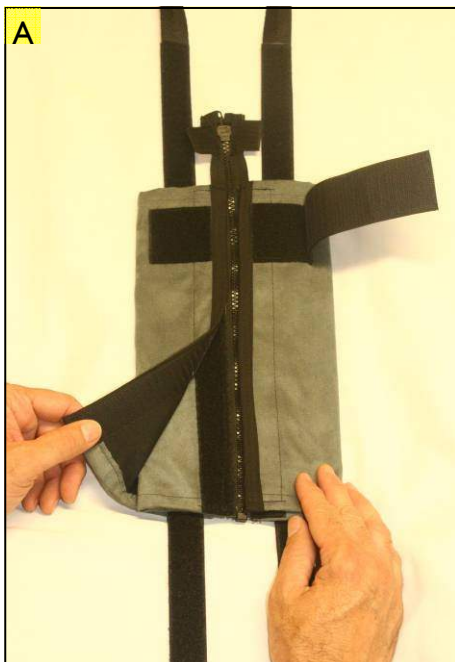




## Fitting the Chest Pad to the Occupant, Cont.

3. Turn the chest pad over so that the front side of the chest pad is facing up (Figure C).
4. Grasp a corner of the large flap at the bottom on the right (Figure D) and lift until the entire side of the flap is detached from the velcro (Figure E).

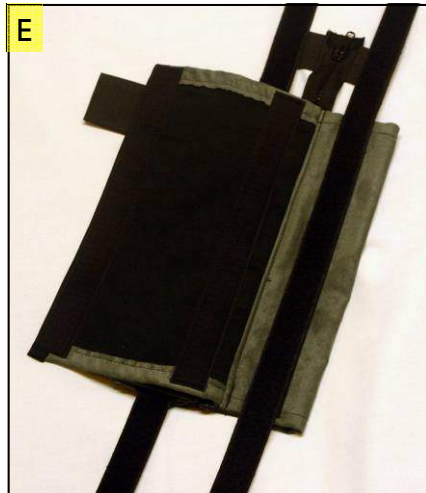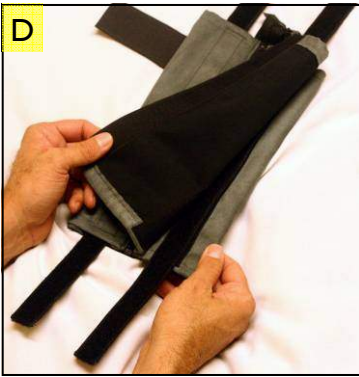

## Fitting the Chest Pad to the Occupant, Cont.


5. Grasp the right side of the neck strap and lift it all the way up and off the chest pad (Figures F and G)
6. Velcro the right side of the neck strap back to the chest pad (Figures H and I) so that its length (after both sides of the neck strap are adjusted) will place the top of the chest pad at the occupant's arm pit level (Refer to the steps on Page 5) .

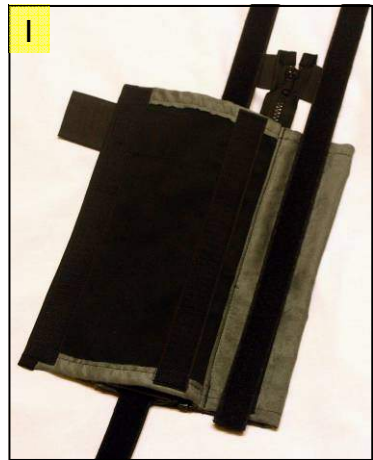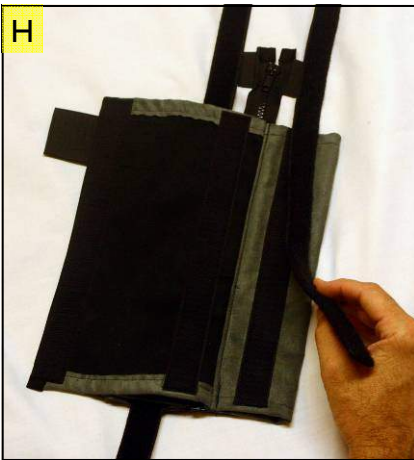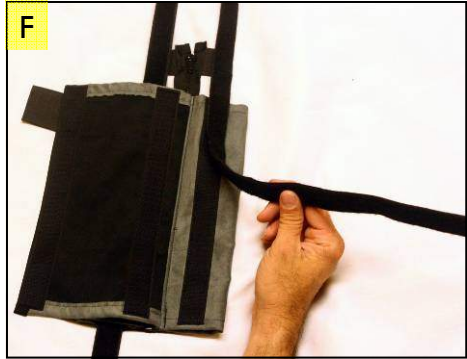



## Fitting the Chest Pad to the Occupant, Cont.

7. Velcro the large flap back to the right side of the neck strap (Figure J).
8. Turn the chest pad over so that the back side is facing up (Figure K).
9. Velcro the flap to the back side of the chest pad (Figure L).
10. Repeat Steps 1 –9 for the left side of the neck strap

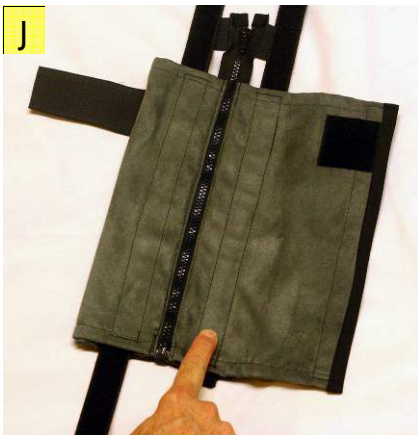

## Removal of the Existing Chest Clip

**NOTE:** The following steps describe how to remove the existing chest clip.

1. Loosen the harness by lifting the harness adjuster ( See arrow, Figure A) and pulling the top of both harness straps forward (Figure A).
2. Remove both neck pads (Figure B).
3. From the back of the restraint, remove one of the harness straps from the splitter plate (Figure C) and pull it through to the front of the restraint (Figure D).
4. Repeat Step 3 for the other harness strap.



## Removal of the Existing Chest Clip

5. Grasp one of the straps at the center of the chest clip and pull the top portion of the strap out of the chest clip (Figures E and F).
6. Grasp the same strap below the chest clip and pull down until it is completely removed from the chest clip (Figure G).
7. Repeat Steps 5 and 6 for the other harness strap.
8. Store the chest clip in a safe place.



## Installation of the Chest Pad

1. From the bottom of the chest pad, insert the right side harness strap all the way through the middle slot of the right side of the chest pad (Figures A and B).
2. Repeat Step 2 with the left side harness strap and the middle slot of the left side of the chest pad.
3. After insertion of both harness straps the chest pad should be positioned as shown in Figure C, with the harness straps positioned between the occupant and the chest pad's neck strap.




## Installation of the Chest Pad

4. From the front side of the restraint, insert one of the harness straps through the appropriate slot (Figure D), according to pages 18 and 19 in the “Instructions for Use” Manual, and pull it through to the back side of the restraint (Figure E).
5. Slide the loop of the harness strap over the splitter plate (Figure E).
6. Repeat Steps 4 and 5 for the other harness strap.



## Installation of Occupant with Chest Pad

1. Secure your child in the restraint according to Steps 1 - 6 on Pages 44 and 45 in the “Instructions for Use” Manual.
2. Pull the two halves of the chest pad together (Figure A), and zip them together all the way to the top (Figures B and C).

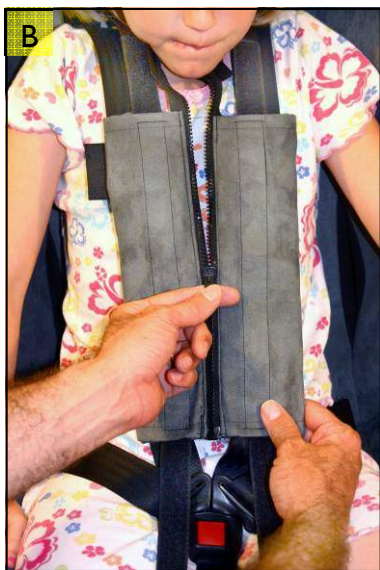

## Installation of Occupant with Chest Pad, Continued

3. Wrap the top of the zipper around the top of the chest pad and Velcro the two small flaps on the top of the zipper to the back side of the chest pad (Figures C (previous page) and D).
4. Cover the top of the zipper with the top velcro tab (Figure E and F).



## Installation of Occupant with Chest Pad, Continued

5. Continue to secure your child in the restraint according to Steps 8 - 11 (ignore Step 10) on Page 45 in the “Instructions for Use” Manual.

**NOTE:** Removal of occupant is the opposite of installation.





## Limited Warranty

Merritt Manufacturing, Inc. (Merritt) provides the following limited warranty on the Roosevelt:

Merritt warrants to the original purchaser of the Roosevelt that, if properly used strictly in accordance with the instructions and specifications included in the original packaging, it shall be free from defects in materials or workmanship for a period of one (1) year from the date of original purchase. Proof of purchase is required, including, but not limited to furnishing to Merritt proof that the owner registration card was returned to Merritt within thirty days of the original purchase or furnishing to Merritt proof of purchase that identifies the claimant as the original purchaser. If any component covered by this warranty fails for reasons covered by this warranty, Merritt will, at its option, provide the original purchaser with repair or replacement components for the covered product or component. This limited warranty is available only for purchasers in the United States and Canada. Merritt reserves the right to discontinue or change fabrics, parts, models or products, or to make substitutions. Please note that the upholstery fabric is subject to some degree of wear as a result of normal use. This kind of normal wear is not covered under warranty.

To make a claim under this warranty, you must contact Merritt via telephone at (317) 409-0148 or write to the company using the address on the back cover of this booklet. PLEASE COMPLETE AND MAIL THE OWNER REGISTRATION CARD WITHIN THIRTY DAYS OF PURCHASE TO PRESERVE YOUR WARRANTY RIGHTS.

### WARRANTY LIMITATIONS

This warranty does not include damages due to external sources, including, but not limited to accident, negligence, misuse, neglect, alteration, repair, improper installation, improper testing or use that is not in conformance with the instructions and specifications in the instruction booklet.

### LIMITATION OF LIABILITY AND DAMAGES

THE WARRANTY AND REMEDIES SET FORTH ABOVE ARE THE SOLE AND EXCLUSIVE REMEDIES AND ARE PROVIDED IN LIEU OF ALL OTHERS, ORAL OR WRITTEN, EXPRESS OR IMPLIED. IN NO EVENT WILL MERRITT, OR THE RETAILER/DISTRIBUTOR SELLING THIS PRODUCT, BE LIABLE TO THE PURCHASER, OR ANY USER OF THIS RESTRAINT OR OTHER THIRD PARTY, FOR ANY DAMAGES; INCLUDING LOST PROFITS, LOST BUSINESS, INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL DAMAGES OR PUNITIVE DAMAGES. IN NO EVENT SHALL MERRITT OR ITS AFFILIATED COMPANIES BE LIABLE OR RESPONSIBLE FOR ANY

## Limited Warranty, Continued

AMOUNT OF DAMAGES ABOVE THE AGGREGATE DOLLAR AMOUNT PAID BY THE CUSTOMER FOR THE PURCHASE OF THE PRODUCT COVERED BY THIS WARRANTY.

### LIMITATIONS OF WARRANTIES AND OTHER WARRANTY TERMS AND STATES' LAW

MERRITT FULLY DISCLAIMS AND MAKES NO IMPLIED WARRANTIES OF ANY KIND, INCLUDING, BUT NOT LIMITED TO IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR USE, AND MAKES NO EXPRESS WARRANTIES BEYOND THOSE STATED HERE. NOTWITHSTANDING THE FOREGOING, IF ANY IMPLIED WARRANTIES ARE DEEMED TO BE GIVEN BECAUSE THE APPLICABLE LAW PROHIBITS THEIR EXCLUSION, THEN ANY SUCH IMPLIED WARRANTIES, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED TO THE DURATION AND TERMS OF THE EXPRESS WRITTEN WARRANTY. SOME STATES DO NOT ALLOW THE LIMITATION OF IMPLIED WARRANTIES, LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS OR THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES. THEREFORE, IN THOSE STATES, THE LIMITATIONS LISTED ABOVE MAY NOT APPLY TO YOU. THIS WARRANTY AFFORDS YOU WITH SPECIFIC LEGAL RIGHTS AND YOU MAY HAVE ADDITIONAL RIGHTS, DEPENDING ON THE STATE IN WHICH YOU RESIDE.

### BINDING ARBITRATION

Any dispute, controversy or claim (whether in contract, tort or otherwise, whether pre-existing, present or future, and including statutory, common law, intentional torts and equitable claims) against Merritt, its agents, employees, successors, assigns or affiliates (collectively for purposes of this warranty, "Merritt"), arising from or relating to this warranty and agreement, its interpretation, or the breach, termination or validity thereof, the relationships which result from this warranty and agreement (including, to the full extent permitted by applicable law, relationships with third parties who are not parties to this warranty and agreement), SHALL BE RESOLVED EXCLUSIVELY AND FINALLY BY BINDING ARBITRATION ADMINISTERED BY THE AMERICAN ARBITRATION ASSOCIATION under its code of procedure then in effect. Any award of the arbitrator shall be final and binding on each of the parties and may be entered as a judgment in any court of competent jurisdiction. Information for submitting claims with the American Arbitration Association may be obtained at