



Product	Shower and changing table 1000, fixed height	
Item number	<p>R8407000 VVS: 777545380 EAN: 5708590294827 HMI: 88217 NBS: N13:40,375 FDA: INT/D124320</p>	
Color	White	
Product description	<p>Shower and changing table 1000, fixed height. Foldable. Incl. safety rail. Outer measurements 29.4" x 70.9". Excl. mounting kit. See Technical specifications (pdf) for information on mounting kits suitable for various wall types. <i>Replaces R8405.</i></p>	
Adjustability	Fixed mounting	
Adjustment height		
Product capacity	<p>Maximum load: 330 lbs. When the table is loaded with 330 lbs, each mounting bolt supports 100 lbs.</p>	
Materials	<p>The table is made from practical and durable PVC canvas which is phthalate free. The frame is powder-coated steel.</p>	
Ergonomy	<p>Pressalit offers a range of high quality, fixed or power adjusted showering and changing tables designed to provide a stable platform suitable for changing adults and children in a variety of settings: schools, day care centers, institutions, hospitals and private homes.</p>	
Features	<p>The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space in front of the unit. Splash curtain (optional) for the changing table protects the caregiver against splashing.</p>	
Warranty	<p>Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.</p>	





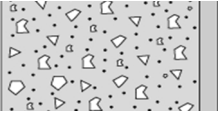

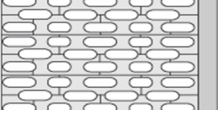
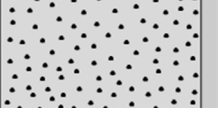
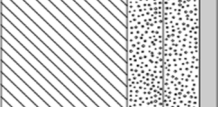
Product	Shower and changing table 1000, fixed height
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	<p>This product complies with the requirements of Council Directive 93/42/EEC concerning medical devices, as last amended by Council Directive 2007/47/EC of 5 September 2007.</p>





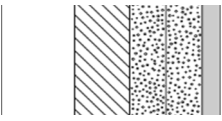
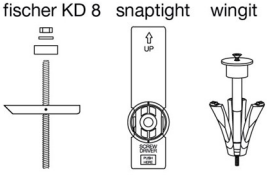
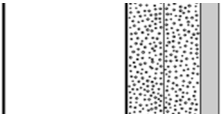
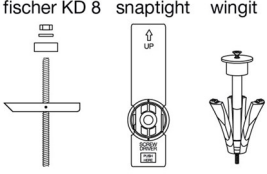
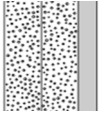

Product	Shower and changing table 1000, fixed height
----------------	--

Mounting in different wall types

	Concrete >B25 V8611 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. To be ordered separately.
	Solid Brick >= Mz12 V8611 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. To be ordered separately.
	Solid Sand Lime Brick >= Hlz12 V8611 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. To be ordered separately.
	Aerated Concrete P4 V8661 - Mounting kit, 8 bolts M5 x 50 - plugs FTP K 6. To be ordered separately.
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8611 - Mounting kit, 8 bolts Ø5 x 50 - plugs SX8. NB! Plugs are not needed. To be ordered separately.





Product	Shower and changing table 1000, fixed height	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (9 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>	<p>fischer KD 8 snaptight wingit</p> 
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</p> <p>Wall fasteners (9 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>	<p>fischer KD 8 snaptight wingit</p> 
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

