



Product	Shower and changing table 1000, electrically height adjustable	
Item number	R8403000-93	
	EAN:5708590317991	
	FDA:INQ/D124319	
Color	White	
Product description	<p>Shower and changing table 1000, powered, height adjustable 15.8"; from 18.9" - 34.7", with wired hand control Foldable. Incl. safety rail. Outer measurements 30.1" x 70.9" Excl. mounting kit. See Technical specifications (pdf) for information on mounting kits suitable for various wall types. <i>Replaces R8401.</i></p>	
Adjustability	Height adjustable	
Adjustment height	400 mm	
Product capacity	<p>Maximum load: 330 lbs. When the table is loaded with 330 lbs, each mounting bolt supports 44 lbs. <i>This product has been tested statically with a load of 830 lbs as the IEC 60601-standard requires testing with a safety factor.</i></p>	
Speed, upwards	<p>Speed up: 15 mm/s Speed down: 20 mm/s</p>	
Noise level	48 dB	
Materials	The table is made from practical and durable PVC canvas which is phthalate free. The frame is powder-coated steel. Material: Steel, PVC.	
Ergonomy	<p>Pressalit offers a range of high quality, fixed or power adjusted Showering and changing tables designed to provide a stable platform suitable for changing adults and children in a variety of settings: schools, day care centers, institutions, hospitals and private homes. Caregivers can easily set the height of the table, enabling them to obtain the correct working position. This ensures safe showering and changing for the user as well as minimizing the risk of back injury for the caregiver.</p>	
Features	<p>Electric with wired hand control – the red buttons clearly indicate “Up” and “Down”. The wired hand control comes with a holder for fitting to the wall. The integrated pneumatic cylinders enable the table to be folded out of the way when not in use to maximize floor space in front of the unit. Splash curtain (optional) for the changing table protects the caregiver against splashing.</p>	
Mains connection (V)	120	










Product	Shower and changing table 1000, electrically height adjustable
Actuator (motor)	LA31 Careline, (Linak), 4000 N, IPX6
Control box	CB9 Careline, (Linak), IPX6
Wired remote control	HB3 (Linak), IP66
Power consumption (maximum load)	Power consumption by maximum load: 70 W Power consumption by standby: 2 W
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.
Cleaning	<p>This product should be cleaned with hot water or a mild cleaning solution. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p>DISINFECTANTS: CaviCide</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
Declaration of conformity	This product complies with the requirements of Council Directive 93/42/EEC concerning medical devices, as last amended by Council Directive 2007/47/EC of 5 September 2007. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and the Council Directive 2011/65/EC on RoHS.










Product	Shower and changing table 1000, electrically height adjustable
----------------	--

Mounting in different wall types

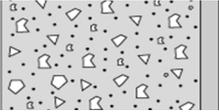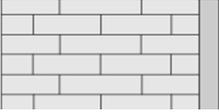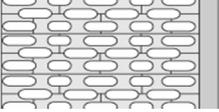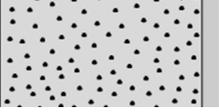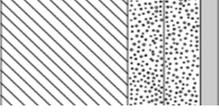	Concrete >B25 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.
	Solid Brick >= Mz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.
	Solid Sand Lime Brick >= Hlz12 V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. To be ordered separately.
	Aerated Concrete P4 V8658 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs GB14. To be ordered separately.
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood V8608 - Mounting kit, 6 lag bolts Ø10 x 70 - plugs SX12. NB! Plugs are not needed. To be ordered separately.



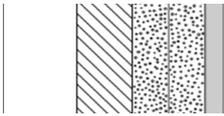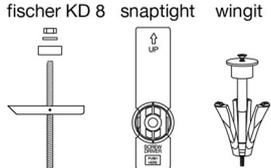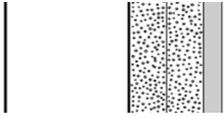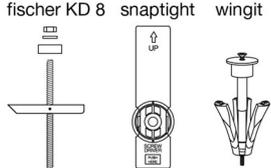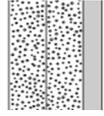

<p>Product</p>	<p>Shower and changing table 1000, electrically height adjustable</p>	
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</p> <p>Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>	<p>fischer KD 8 snaptight wingit</p> 
	<p>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (.08 inch)</p> <p>Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. SnapTight or WingIt anchors or similar.</p>	<p>fischer KD 8 snaptight wingit</p> 
	<p>Drywall (2 x 12,5 mm) water-resistant</p> <p>Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.</p>	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

