

# OWNER'S MANUAL


*PRODUCT PHOTO  
PARTS LIST  
ASSEMBLY INSTRUCTIONS  
MAINTENANCE  
WARRANTY*

## Prime Engineering Up-Rite Stander

Manufactured By  
**Prime Engineering**  
A Division of Axiom Industries Industries, Inc.

# Up-Rite Stander

*A Product designed by Therapists for Therapists*

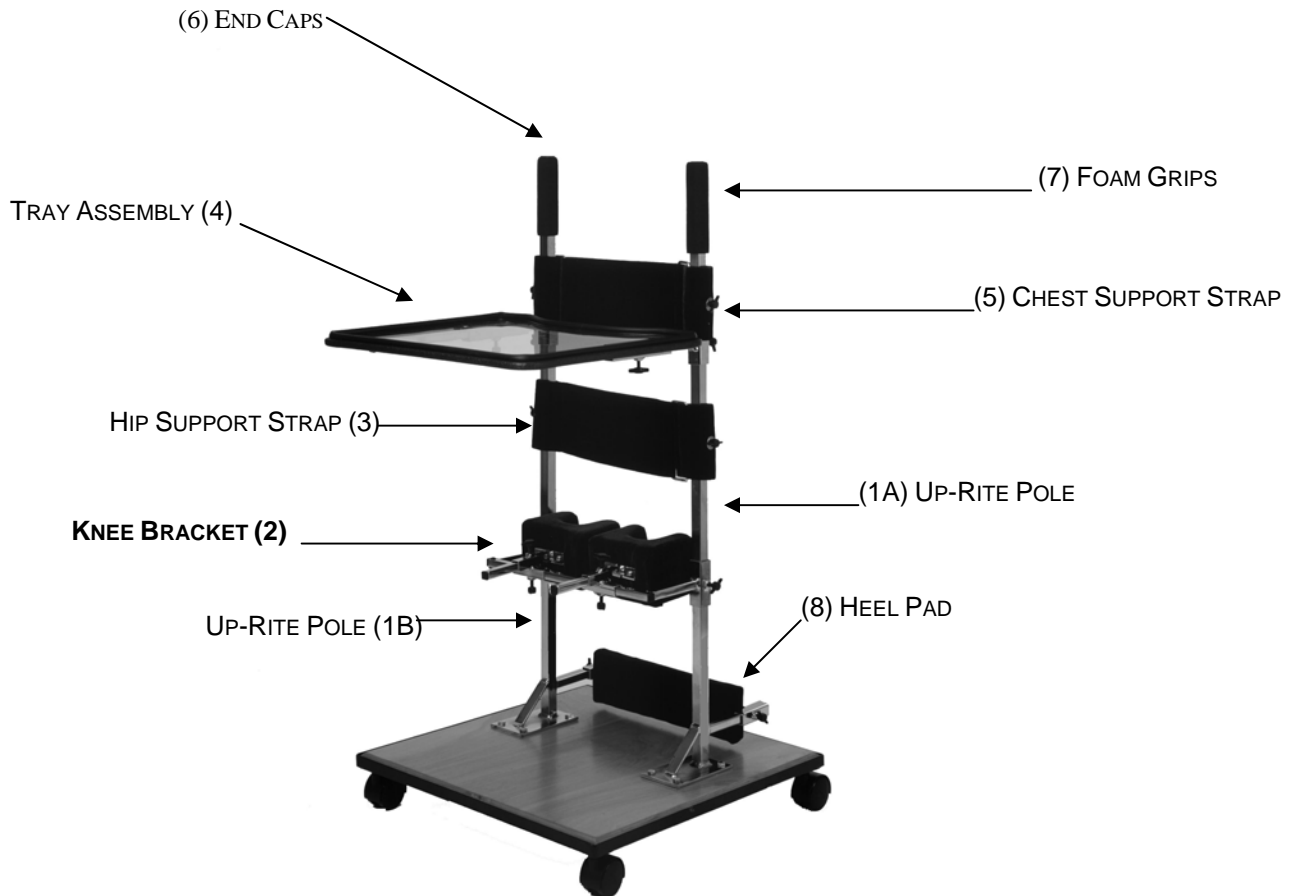

\* Photo shown with optional – WHEELED CASTER BASE

## PRODUCT DIMENSIONS:

Up-Rite I	Up-Rite II	Up-Rite III
Base Size: 20" wide X 20" long	Base Size: 24" wide X 24" long	Base Size: 32" wide X 32" long
Tray Size: 18" wide x 14.5 deep	Tray Size: 21" wide x 17.5 deep	Tray size: 24" wide x 21 deep
Patient Height: up to 36"	Patient Height: up to 48"	Patient Height: up to 60"

Note: Child height should not exceed height of stander.

Congratulation's on your selection of the **Up-Rite Stander** for your standing needs.

The **Up-Rite Stander** is a therapist designed standing system for children requiring good midline positioning for proper weight bearing and activities involving the upper extremities.

Carefully read all instructions and notes before assembling or using your **Up-Rite Stander**.

Stand with us for good positioning, posture and increased function...every day...you deserve the best health standing can offer!

This owner's manual is organized into the following sections:

1. Product Photo and Specifications.
2. Parts List
3. Assembly Instructions
4. Maintenance and Warranty

We've designed the **Up-Rite Stander** for easy assembly. Each Up-Rite Stander is pre-assembled at Prime Engineering to insure that everything works properly. If, however, you have any problems or require for assistance, please give us a call.

## Parts List

The Up-Rite Stander comes partially assembled.

Package contains:

- 2 - Up-Rite Poles
- 1 - Hip Support Strap
- 1 - Chest Support Strap
- 2 - Foam Grips
- 1 - Tray Assembly
- 2 - Knee Pads with Brackets
- 1 - Heel Pad
- 1 pkg. - T-Knobs

When ordered Up-Rite III carton B contains:

- Two - Up-Rite Poles

## ASSEMBLY INSTRUCTIONS

Set-up and assembly on your Up-Rite Stander is easy when you follow these directions:

- 1) Attach the Up-Rite pole (1A) and (1B) to the base. The poles are correct when heel pad brackets face the back of the base.
- 2) The kneepad brackets would be on top of the bar on the Up-Rite II (medium) and Up-Rite III (large) stander. There will be two thumb-locking screws on the top and one on the bottom of the knee pad brackets. The kneepads should be placed in these brackets. Staples in the kneepads should be facing down.
- 3) The Hip Support Strap (2) loops through the D-Ring then Velcro towards the back of the base to the clients left.
- 4) The tray assembly (4) should be placed on the poles with the ribbed side up and toward the front of the stander. Thumb-locking screws underneath the tray will adjust the tray's depth.
- 5) The Chest Support Strap (5) has a double adjustability. The front part of the strap is threaded through D-Ring and Velcro's to the client's right. The back strap goes through the D-Ring and attaches to the client's left.
- 6) End Caps (6) should be placed on the top of the Up-Rite poles.
- 7) Foam Grips (7) slide easily over the end caps. A silicone lubricant can be used to assist in process, if needed.
- 8) Slip heel pad (8) onto the back brackets of the Up-Rite poles. The staples on the back of the heel pad should be facing the wood base.

## General Maintenance

1. Periodically insure nuts and bolts are secured properly.
2. Keep moving parts clean for free movement. The Up-Rite Stander does not require lubrication.
3. Use soft cloth when cleaning tabletop.
4. All fabrics should be hand washed with mild soap and water, then allowed to air dry.

### CAUTION

CHILDREN SHOULD NOT BE LEFT UNATTENDED IN THE STANDER.

Accessory base with caster should be braced against solid surface before engaging the locks.

---

**UP-RITE I, II, III**  
**Limited Warranty**  
Effective February 1, 2002

Warranty coverage is extended to the original purchaser for as long as the original purchaser owns the *Prime Engineering* system. *This warranty is in effect for the Up-Rite (the product) standing frames. Prime Engineering, Inc. ("the Company") shall repair, or replace the Product that fails due to defect in material, or workmanship per the following schedule:*

**Frames:** Lifetime to the original purchaser subject to the following terms, limitations and conditions.

**Straps:** One year to the original purchaser subject to the following terms, limitations and conditions.

**Other Components:** Upholstery, plastic parts, painted surfaces, rubber parts, bearings, casters and other parts not specifically identified above, all of which are warranted against defects in material and workmanship for three months from the original purchase date, subject to the following terms and conditions, and limitations.

**Warranty Conditions.** This warranty shall be in effect from the original purchase date to the consumer. The warranty registration form should be fully completed by the original retail purchaser / user or their agent, and received by *Prime Engineering* at its address set forth on the registration form no later than three months after the original purchase date. Any defective Product shall be delivered to *Prime Engineering*, or its local authorized dealer, at purchaser's expense for warranty inspection. Return freight charges shall be prepaid by *Prime Engineering*.

**Limitations and Exclusions.** The warranty provided above shall not apply to (a) a serial numbered Product if the serial number has been removed or defaced; (b) any Product subjected to negligence, abuse, misuse, improper operation, maintenance, or storage, or damages beyond *Prime Engineering's* control, as determined by *Prime Engineering*; (c) defects arising from purchaser's failure to follow instructions. *Any Changes or Additions* to the Product of unauthorized parts not manufactured or furnished by *Prime Engineering*. *The repair or replacement by Prime Engineering for a defective Part or Product shall be purchaser's sole and exclusive remedy under the warranty.* Repair labor/service charges are not covered by this warranty and are not the responsibility of *Prime Engineering*.

**The foregoing Warranty is exclusive and in lieu of all other warranties, express, implied, or statutory, including warranties of merchantability and fitness for a particular purpose and design. In no event shall *Prime Engineering* be liable for any consequential or incidental damages.**

Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitations of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state.

**Prime Engineering**  
A division of Axiom Industries, Inc.

