

INSTRUCTION MANUAL
Supine Stander
Size Small—SS1



*Read and follow all instructions for assembly and use before
using this product*

Warnings

The use and adjustment of this stander must only be with the recommendation and guidance of a qualified therapist or physician. Adult supervision is required at all times. Never leave a child unattended while the product is in use.

Prospect Designs, Inc will not be responsible for damage or injury resulting from any modification to this equipment by anyone other than the manufacturer.

Prospect Designs, Inc will not be responsible for damage or injury resulting from misuse or neglect of this equipment.

USE

The Supine stander is designed to provide the child with ability to experience weight bearing in an upright or forward leaning position while the combination of adjustments and supports hold the child in good postural alignment.

Never leave a child unattended.

CARE

Regular inspections should be done to check for loose or missing hardware or any signs of cracked or broken parts. Contact the manufacturer immediately if you suspect an unsafe condition.

All components may be cleaned with a mild soapy water or non-abrasive commercial cleaner approved for use on vinyl and wood. Care should be taken to not use excessive amounts of water and wood parts should be wiped dry after cleaning.

WARRANTY

All Prospect Designs, Inc products are guaranteed to be free from defects in materials and workmanship for the life of the product. Should any part break or fail Prospect Designs, Inc. will repair or replace it at no charge to the original owner

FITTING AND ADJUSTING THE PROSPECT DESIGNS SUPINE STANDER Model SS1

Angle Adjustment


The angle of the supine stander is adjustable from fully upright (90°) to nearly horizontal. The fully reclined position is typically used for positioning a child in the stander since all supports and straps can be adjusted and secured without having to fight gravity to keep the child in good alignment. Once the child is positioned the stander can be raised up to the desired angle for the child's abilities.

Each child is different so this and every adjustment should be determined by a therapist or physician.

The stander angle is controlled by the two **angle adjustment knobs** on the back support legs. Loosen the knobs and the stander is easily moved to the desired angle. In order to recline the stander to its fullest the base of the stander must also be adjusted to its greatest length. The base length is controlled by the two **base adjustment knobs**. Loosen the knobs and slide the base longer or shorter as needed.



Stander in upright position

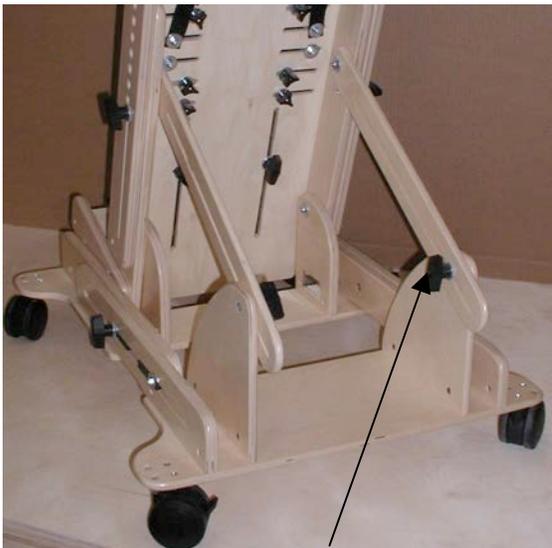


Angle adjustment knob




Stander in fully reclined position

Height Adjustment

The height of supine stander is controlled by the two **height adjustment knobs**. Loosen these knobs and raise the entire supine board assembly to the desired height. The most typical configuration for the stander is to set the height of the main back board so the top is level with or just below the child's shoulder height. The headrest would then extend above the back.

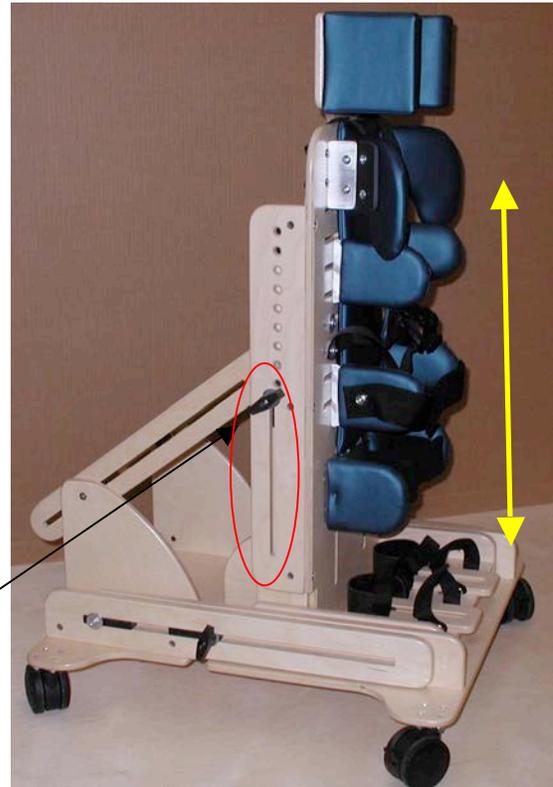

Stander height at shortest adjustment

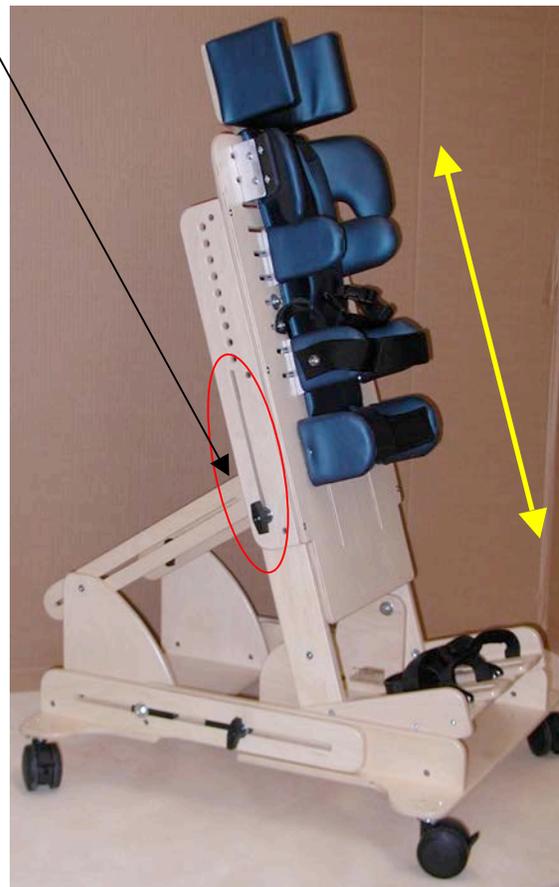

Stander height at tallest adjustment

Support Pad Adjustments

KNEE SUPPORT ADJUSTMENT

The behind the knee pads with padded straps are adjustable up and down and side to side. To make either adjustment you must loosen the knobs on the back of the stander. The mounting bolts in the pads are offset to one side so that when the pad is rotated 180° it moves left or right.

LATERAL PAD ADJUSTMENT

The lateral support pads (hip and trunk) are adjustable for height and width. All adjustments are made by loosening or removing the adjustment knobs on the back of the stander.




Knee pads with straps spaced apart



Knee pads with straps close together

Lateral pad adjustment knobs

Knee pad adjustment knobs

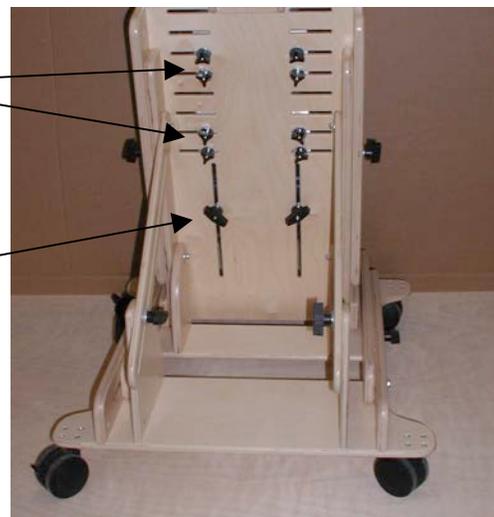

SUPINE STANDER ACCESSORIES

Lateral Knee Supports

The optional lateral knee supports are adjustable for height and angle. They may be mounted on either side of the knee depending on where the extra support is needed. They are adjusted by loosening the same knobs on the back of the stander that control the standard behind the knee pads.



Lateral knee supports mounted outside of knee



Lateral knee supports mounted inside of knee

SUPINE STANDER ACCESSORIES

Chest Harness

The Supine Stander chest harness is attached to the stander with bolts mounted through a pair of the horizontal adjustment slots on the stander back. The shoulder straps with the velcro and D-ring closure are mounted on the back of the stander usually one slot below the trunk support lateral pad. The lower straps are mounted to the front of the stander usually one slot below the shoulder straps. This placement may vary with the needs of the child. A Phillips screw driver and 7/16" wrench are needed to change the strap attachments.

The lower straps are supplied with "slip lok" fasteners which help keep these straps set to an appropriate length once adjusted. These straps are not generally adjusted during regular use. They should be set so that the harness does not ride up above the child's sternum so as to create a risk of strangulation. The shoulder straps then can be pulled snug to the shoulders each time the child is placed in the stander.

Each child is different so this and every adjustment should be determined by a therapist or physician.

Lower straps with Slip Lok fastener



Shoulder strap mounting bolt

