



Large Supine Stander (E430)

Supine Standers


The ability to stand changes how people see and experience the world – literally. In a classroom setting, our Supine Standers allow for greater social interaction and involvement in a wide range of therapeutic or recreational activities. Simply being able to interact with peers at eye level can make any activity more rewarding.

As flexible and adaptable a stander as you'll find anywhere, Supine Standers make standing comfortable for clients with a wide range of capabilities. Each stander is designed to provide excellent support and security as clients improve their skills. With posterior and lateral support, our fully adjustable Supine Standers provide maximum stability while allowing partial weight-bearing and freedom of movement – ideal for individuals who have difficulty controlling the head or shoulders.



“Superbly adjustable and stable, the Supine Stander is ideal for the classroom. The vertical height adjustment makes transfers easier and provides versatility, adapting to the needs of many different students. And in the stander they're more alert and active and improve head control.”

Larry Ezzard, CRTS
Founder and President of Mobility Designs
Atlanta, GA

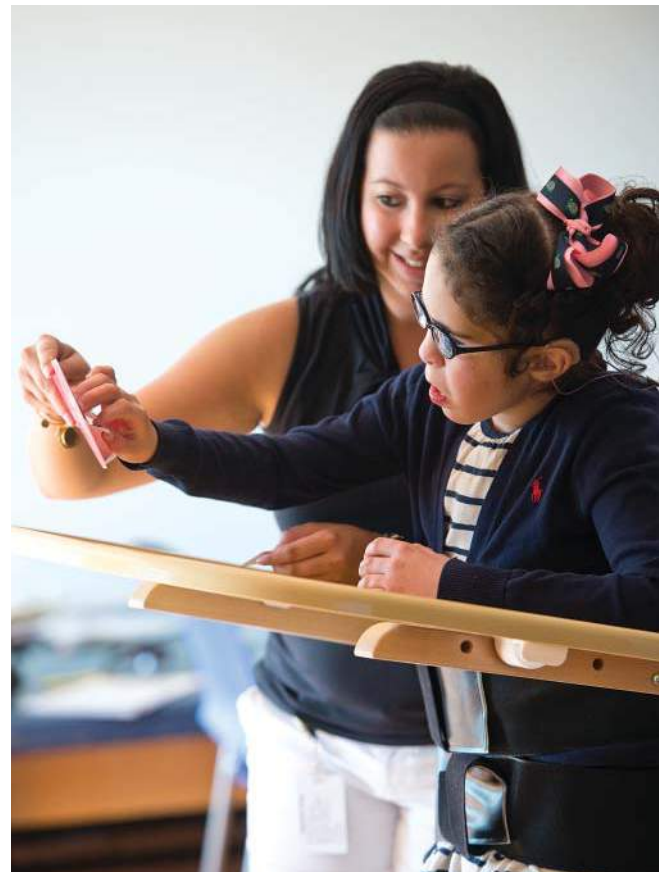



Supported by the hand Anchor, this child is able to participate in activities she enjoys.



This child has advanced so far that the headrest can now be flipped back out of the way.



Use the frame's marked calibrations to record and maintain each user's optimal position.

Standing changes how we see the world.

The Supine Stander offers standing opportunities for medically involved patients, allowing greater social interaction and involvement in a wide range of therapeutic or recreational activities. Ideal for individuals who have been bedridden for a long time or who have significant musculoskeletal weakness, the angle adjustments on the Supine Stander allows a user to be safely assisted into the standing position. At the same time, weight-bearing through the lower extremities can be initiated.

Why use a Supine Stander?

- Standing option for medically involved clients:** The firm main board of the Supine Stander allows users with significant musculoskeletal weakness, secondary to prolonged immobilization, the opportunity to be positioned upright. Stability is provided posteriorly making it a comfortable option for users with tracheotomy.
- For gradual progression to an upright position:** The angle-adjustable stander is effective for assisting individuals who have been bedridden for a long time to a standing position. Ranging from horizontal to upright standing, the Supine Stander is ideal for people with orthostatic hypotension to lessen the adverse haemodynamic response from a rapid change in position.
- Upright interaction:** With the ability to reach 85° of upright positioning, users can be at eye-level with their peers, participating in activities and initiating communication.
- For improving head control:** The head support on the stander can be flipped down to allow the user opportunities to initiate and maintain head control.
- Easier transfers:** The large Supine Stander provides horizontal adjustments of the main board from 21"–30", making transfers to the stander easier from a wheelchair or bed.



The Supine Stander stays secure at any angle. Turn the crank to gradually increase weight bearing.



The Large Supine Stander makes transfers easy. The manual crank raises and lowers the stander horizontally from 21" (wheelchair height) to 30".

Dependable adjustments and easier transfers



The large Supine Stander (E430) provides 9" vertical adjustment in the horizontal position.



The laterals adjust easily even with the client in the Supine Stander.



Double-locking casters prevent any roll or swivel during transfers.

Accessories



Tray

The tray has a smooth cleanable working surface with rounded corners. Angle of tray can be changed by adjusting the armrests.
Inside dimensions: 26" x 15"



Additional laterals

Additional laterals help align the trunk and are fully adjustable.



Sandals and Wedges

Sandals and wedges raise heels or toes. Configure them yourself with the unique "stack and latch" feature. For different leg lengths combine two wedges under one sandal.

Small: 3" x 7", med: 4" x 9", large: 5" x 10½"



Adjustable abduction wedge

Adjustable abduction wedge can be changed from 3½"–11" in width depending on how much abduction is desired.



Round abductor

The round abductor helps maintain 4" of comfortable knee separation.





Collar

Collar adds 2" of separation to the round abductor.

(Requires round abductor)



Hand Anchor

Hand Anchor helps position hands firmly anywhere on the tray. With a secure anchor point for one arm, the user can perform tasks more easily with the other arm and hand.



Rifton Anchor family

For more hand/arm support options see pp. 98–101.



Features of the Supine Stander

Teach head control

by flipping down the top of the stander.

Encourage upper body control

by moving the armrests up and down the frame or by tilting them at varying angles. When they're not needed they fold down out of the way.

Accurate positioning

Angle adjustment is calibrated from 0° to 85° for accurate positioning.

Close access

The sturdy, curved frame gives foot room to the caregiver and close access for transfers.

Hygienic

Healthcare grade vinyl covers padded body support.

Easy transfers

Lateral supports slide out of the way to aid transfers.



Transfers made safe

Double-locking casters prevent roll and swivel during transfer.

The footboard adjusts,

bringing the client up to a comfortable height for the caregiver.