

ROHO® QUADTRO SELECT® Cushion

HIGH PROFILE® version



LOW PROFILE® version



MID PROFILE™ version not shown

APPLICATIONS

The **HIGH PROFILE QUADTRO SELECT** Cushion is most effective for individuals:

- have a history of skin/soft tissue breakdown
- currently have skin/soft tissue breakdown
- at risk for skin/soft tissue breakdown
- have normal or impaired sensation
- require accommodation of pelvic asymmetry, up to approximately 3 in (7.5 cm)
- require increased stability
- require correction of pelvic asymmetry (medial lateral and anterior posterior)

The **MID PROFILE QUADTRO SELECT** Cushion is most effective for individuals:

- have a history of skin/soft tissue breakdown
- currently have skin/soft tissue breakdown
- at risk for skin/soft tissue breakdown
- have normal or impaired sensation
- require accommodation of pelvic asymmetry, up to approximately 2 in (5 cm)
- require increased stability
- require correction of pelvic asymmetry (medial lateral and anterior posterior)

The **LOW PROFILE QUADTRO SELECT** Cushion is most effective for individuals:

- have a history of skin/soft tissue breakdown
- currently have skin/soft tissue breakdown
- at risk for skin/soft tissue breakdown
- have normal or impaired sensation
- require accommodation of pelvic asymmetry, up to approximately 1 in (2.5 cm)
- require increased stability
- require correction of pelvic asymmetry (medial lateral and anterior posterior)

SPECIFICATIONS

INCLUDES: QUADTRO SELECT HIGH PROFILE, MID PROFILE or LOW PROFILE Cushion, two-way stretch cover, hand inflation pump, repair kit, and operating instructions.
CONSTRUCTIONS: Flame resistant, Neoprene Rubber
SIZES: Wide range to fit chair widths 12" to 24"/30 cm to 61 cm on the HIGH PROFILE, 14" to 19"/36 cm to 47 cm on the MID PROFILE, 12" to 20"/30 cm to 51 cm on the LOW PROFILE.
Cell height is 4.25" on the HIGH PROFILE, 3.25" on the MID PROFILE, 2.25" on the LOW PROFILE.
WEIGHT: Varies by size, approximately 3.5* lbs./1.6 kg (based on 16" x 16" size cushion)
WEIGHT LIMIT: Unlimited*
WARRANTY: 36-Month Limited Warranty
U.S. MEDICARE CODE: E2624/E2625
- Adjustable skin protection and positioning wheelchair cushion.
*Cushion must be properly sized to the individual.

ADJUSTABLE: Fits specific seating needs and unique body types.

LIGHTWEIGHT: Easy to handle and carry.

WASHABLE: Cushion can be easily cleaned with soap and water. Cover is machine washable.

REPAIRABLE: Neoprene rubber cushion can be patched or repaired to extend use.

VENTILATED: Multiple air cells allow air circulation to reach the surface of the skin and provide a cooling environment.

We recommend consulting with a certified rehab professional when ordering and fitting ROHO Group products. Visit NRRTS.org for information on certified providers in your area.



LITG02083 Rev. 08/11, 2M SP 08/11

ROHO® CONTOUR SELECT® Cushion



APPLICATIONS


The **CONTOUR SELECT** Cushion is most effective for individuals:

- have a history of skin/soft tissue breakdown
- currently have skin/soft tissue breakdown
- at risk for skin/soft tissue breakdown
- have normal or impaired sensation
- require accommodation of pelvic asymmetry, up to approximately 1 in (2.5 cm)
- require increased stability
- require lower extremity alignment
- require correction of pelvic asymmetry (medial lateral and anterior posterior)

SPECIFICATIONS

INCLUDES: CONTOUR SELECT Cushion, two-way stretch cover, hand inflation pump, repair kit, and operating instructions.
CONSTRUCTIONS: Flame resistant, Neoprene Rubber
SIZES: Wide range to fit chair widths 14" to 20"/36 cm to 51 cm,
WEIGHT: Varies by size, approximately 3 lbs./1.4kg (based on 16" x 16" size cushion)
WEIGHT LIMIT: Unlimited*
WARRANTY: 36-Month Limited Warranty
U.S. MEDICARE CODE: E2624/E2625
- Adjustable skin protection and positioning wheelchair cushion.
*Cushion must be properly sized to the individual.

The latest evolution of ROHO is a natural selection.

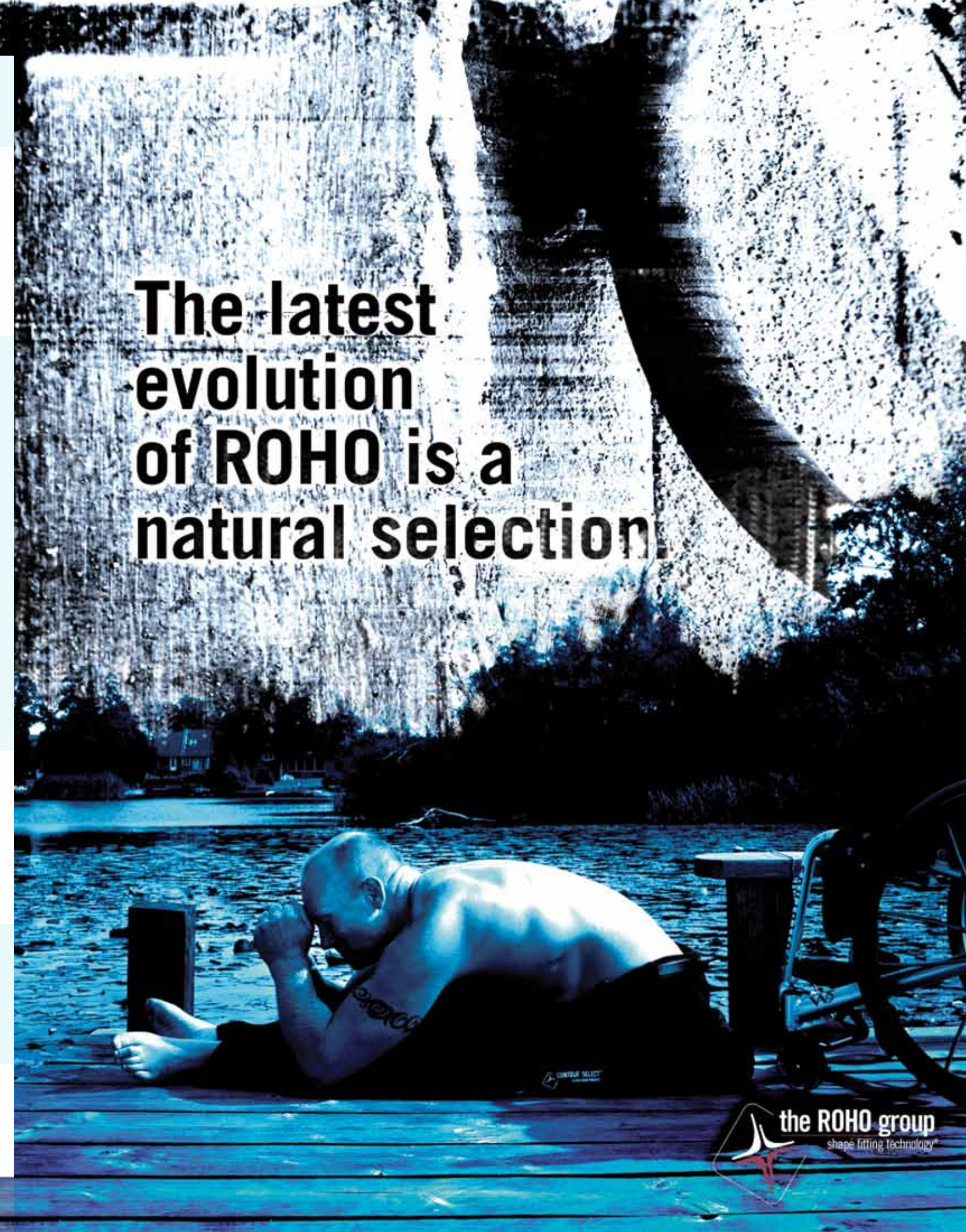

The SELECT Series from ROHO® Sit down and take notice.

With the ROHO® QUADTRO SELECT® HIGH PROFILE®, ROHO QUADTRO SELECT MID PROFILE™, ROHO QUADTRO SELECT LOW PROFILE® and ROHO CONTOUR SELECT® Cushions, you can now enjoy a choice of seating cushions that meet your needs for an active lifestyle.

SELECT Series Cushions set the standard in the industry for overall performance in wheelchair seating. With the benefit of a customized fit through the simple push of a knob, the revolutionary ISOFLO MEMORY CONTROL® offers shape fitting capabilities while you are seated, allowing quick and easy, on-demand adjustment to maximize function. With the built-in stability and simplicity of a SELECT Series Cushion, no longer will you have to sacrifice maximum skin/soft tissue protection to get stability, positioning or convenience.



QUADTRO SELECT HIGH PROFILE

Also available in MID PROFILE and LOW PROFILE



CONTOUR SELECT

Versatility and stability are the hallmarks of every SELECT Series cushion. From QUADTRO SELECT HIGH PROFILE to LOW PROFILE to CONTOUR SELECT, all cushions deliver a high level of skin protection and stability.

A perfect position to match any activity.

Now you can adjust positions throughout the day, as your changing activities demand—all while seated. Just push the ISOFLO MEMORY CONTROL® to the open position and adjust to the desired sitting position. Then switch it back to the closed position and this new posture is locked in for maximum comfort and function.



Stability is just a push away.

When open, the cushion is inflated through one air valve, adjusted for your lowest bony prominence and positioned for your specific seating needs.



When closed, the ISOFLO MEMORY CONTROL locks the cushion to your required seat position.



Color-coded control helps visually aid in the operation.

Superior skin protection has never felt better.

Built upon thirty years of cutting edge research and real-life experience, the patented ROHO® DRY FLOATATION® Technology provides comfort and protection by fitting, matching, and tracking your shape.

To experience how much better life is on a SELECT Series Cushion, see your certified rehab professional or visit us at www.therohogroup.com today!

