Reusable Overlay Cover Instructions for Use

For additional information about using this cover, refer to the instructions provided with a compatible ROHO Mattress Overlay.

Supplier: This manual must be given to the user of this product.
Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.
Incident Reporting: In the event of a serious incident or injury, contact Customer Support.
* "Warning" Indicates that not following the specified instructions may lead to hazardous conditions resulting in serious injury.


Intended Use: The Reusable Overlay Cover* is a one-piece fluid-resistant textile intended to protect a ROHO Mattress Overlay from solid debris or liquids.
* ROHO Reusable Mattress Overlay System Cover • ROHO Reusable Mattress Overlay Bariatric System Cover
Compatibility: Compatible mattress overlays are shown on the cover care label.
In addition to the intended use statement, a clinician should be consulted regarding use of this cover within the individual's overall care regimen and including any need for large-print instructions or other assistive technology.

Cover Placement

Warnings: Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The mattress overlay surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the overlay will contact unprotected skin. • DO NOT use the product as a transfer surface.

- Before the individual transfers to the bed, place the cover around the overlay and the bed mattress, referring to the directional labels to orient the cover.
- If needed, the following items may be placed on top of the Mattress Overlay and Reusable Overlay Cover: One flat or contoured sheet, arranged loosely • One incontinent pad • One draw sheet, for moving and transferring.

Warning: Placing too many linens between the individual and the product will reduce its effectiveness.
Cover Removal: The individual should transfer to a different surface. Pull each corner away from the bed mattress and overlay.

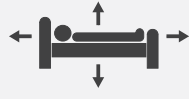Materials: Two-way stretch, moisture-resistant fabric with elastic. Refer to the cover care label for material content and for compliance to flammability requirements.

Cover expected lifetime: 1 year
Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incinerate with other healthcare waste. Incineration must be performed by a qualified, licensed waste management facility.

Cover Care

Warnings: Clean the product regularly and when it becomes soiled. If the product becomes soiled, or between use by different individuals: clean, disinfect, and check for proper functioning. • Follow all manufacturer bleach container and germicidal disinfectant safety guidelines. • This product does not require sterilization; DO NOT autoclave. • DO NOT dry clean.

Notes: Repeated washing at high temperatures can accelerate deterioration of the cover and is not recommended. • Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer.

To Clean: Remove the cover from the overlay. Machine wash cold with mild detergent, gentle cycle, or wipe clean with neutral detergent and water. Rinse thoroughly. Tumble dry low or air dry.

To Disinfect: Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Quaternary-type disinfectants may also be used in concentrations recommended by the manufacturer. Rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low or air dry.

Note about sanitization: Washing machines certified to NSF Protocol P172 are designed to heat water to 60°C. Recommendation: When washing by hand, use room-temperature water.

Symbols Glossary

	Model number		Fits		Date of manufacture		Manufacturer		Fall risk.		Machine wash hot, normal, at the temperature shown.		Hand wash.		Line dry.
	Medical Device		Size		Meets flammability		Entrapment risk		Machine wash cold, gentle.		Tumble dry low.		Cleaning instructions		Disinfecting instructions
	Batch code		Consult instructions for use.		Authorized representative in the European Community		Suffocation risk		Bleach (1 part bleach: 9 parts water).		Do not dry clean.				

Limited Warranty Term: 6 months; see also the Limited Warranty Supplement provided with the product, or contact Customer Support.

Contact Customer Support for previous versions of product information. • ROHO® is a registered trademark of Permobil. • ROHO, Inc. is a part of Permobil Seating and Positioning.

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