

rollz®



Rollz Motion Rhythm

EN - Module and app guide

NL - Gids van de module en app

DE - Leitfaden des Moduls und der App

FR - Guide du module et de l'application

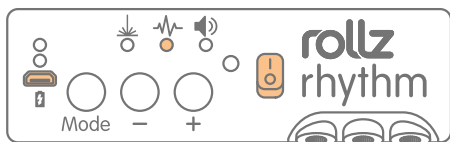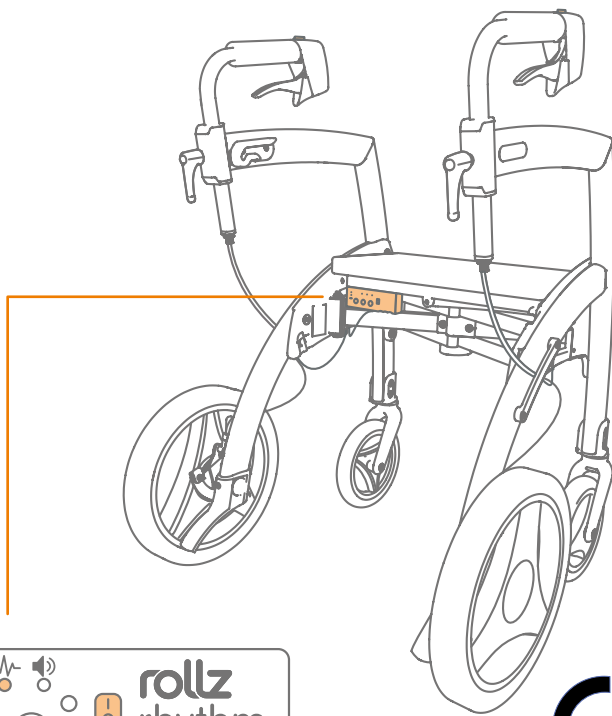

CE

MODULE AND APP GUIDE

rollz motion rhythm

EN

The newest version of this manual can be found on our website and in the settings area of the Rollz app.

BEFORE YOU START

This guide is an addition to the Rollz Motion user manual.

You can set up and use the Rollz Rhythm module in two ways: via the module on the rollator and/or via the app.



WARNING

The Rollz Motion Rhythm contains electronic components. Therefore, prevent the rollator from getting wet. Do not use it outside in heavy rain.

Use only the supplied micro USB adapter to charge the module.

TECHNICAL DETAILS OF THE MODULE

Operating temperature: -5°C to +45°C


Rated voltage and current: 5V DC - 0,5A

Battery capacity: (1S2P) 5200mAh

Wireless connection frequency: 2.4GHz - 2.5GHz

CHARGING THE ROLLZ RHYTHM MODULE

Before starting the module, make sure it is charged.
To charge, use the included micro USB adapter.
The USB connector is located on the left side of the module.

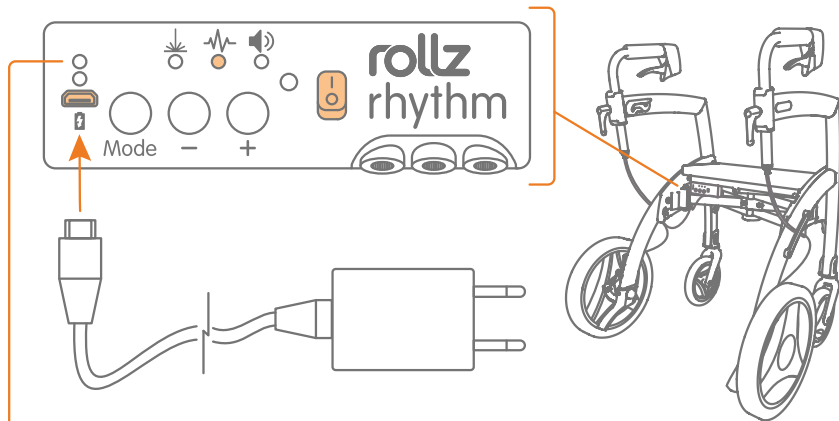

The module will display a red light while charging. When fully charged, this light will turn green. Charge the module only with an adapter of 5V and 1A.

Battery life depends on usage. Battery can be charged at any time. A fully depleted battery takes 10 hours to charge.

Note: When the battery is empty and the module can no longer boot, charge the battery with the included micro USB charger.


INSTALLING THE ROLLZ APP

The Rollz app can be downloaded for free on your smartphone or tablet from the Play Store (for Android) or the App Store (for iOS). Search for "Rollz".



CONTROLLING THE MODULE ON THE ROLLATOR

To turn the module on, slide the power switch upwards. Allow a few seconds to turn on.

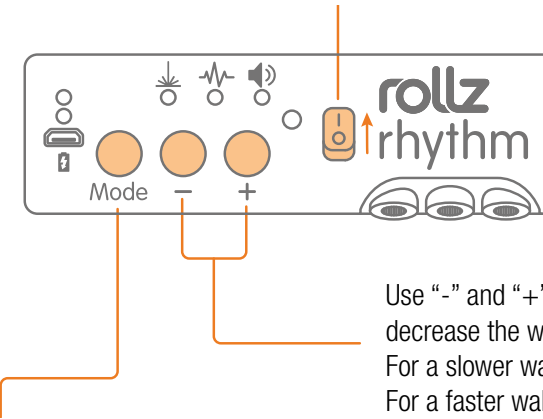

Use "-" and "+" buttons to increase or decrease the walking frequency in steps/min. For a slower walking speed, press "-". For a faster walking speed, press "+".

Use the "Mode" button to shift through the possible combination of cues: laser, vibration and sound. The active cues will be displayed by the LEDs.


- All cues
- Laser
- Vibration
- Laser & vibration
- Sound
- Laser & sound
- Vibration & sound

To reset to default settings, press and hold both "-" and "+" buttons for two seconds. Default settings: all cues are on, medium intensity, 16 repetitions and 66 steps/min.

Extra settings are available and can be set in the Rollz app.


START THE CUES WHILE WALKING

Use any of the two buttons on the handgrips to start or stop the cues. Press shortly to start the cues and again to stop.

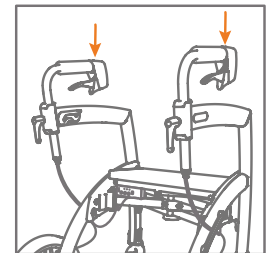


CONNECTING THE MODULE (with the app)

Turn on the module on the rollator and open the Rollz app to connect.

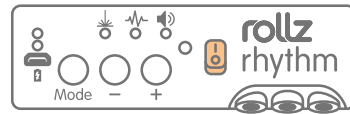

The module can be connected by Bluetooth.
When opening the app a few steps have to be taken to connect to the device:

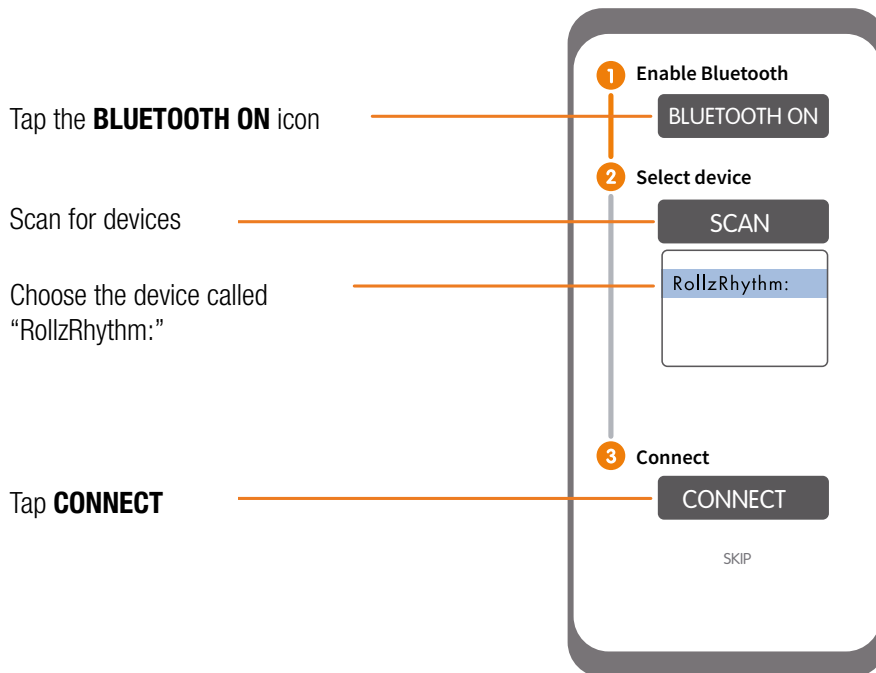

When the module is successfully connected, the Rollz app will display the "connected symbol" in the upper right corner. At the same time, the LED's of all active cues will flash on the module.



USING THE ROLLZ APP

With the app you can set up more options than via the module.

Toggle to activate/deactivate cues.



1. Laser
2. Vibration
3. Sound
4. Receive cues on the phone

Toggle to activate/deactivate cues on the phone

Intensity of the sound

Pattern length adjustment of 8, 16, 32 steps or continuous repetition


Scroll down in the Rollz app to adjust every step individually for the respective cues.
Tap to activate/deactivate each step.

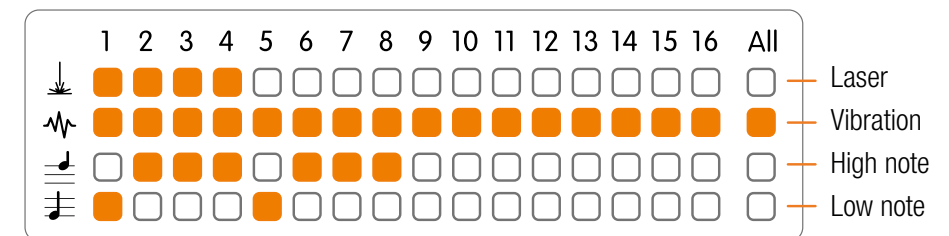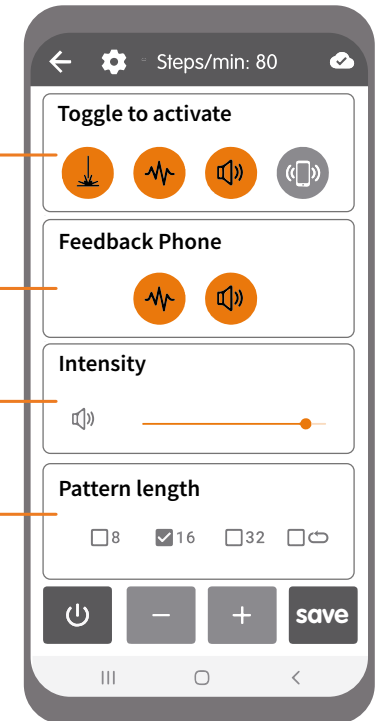

Once the desired settings are set, tap **save**. The Rollz Rhythm module will now execute an example of the newly saved settings.

Note: Settings changed in the app will only take place after tapping **save**.

The rhythm of the pattern can also be adjusted in the Rollz app by tapping “+” or “-”. The changing in amount of steps per minute can be seen on the top of the screen when these buttons are engaged. Press **save** after each adjustment.

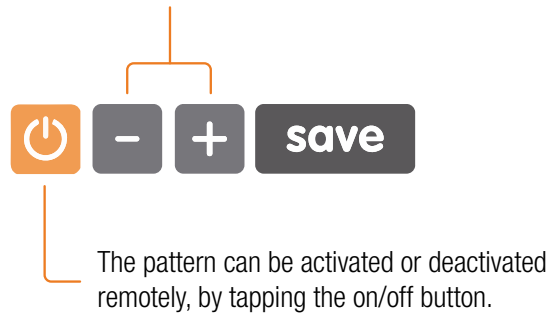

Note: Update the app when a new version is available.

PRIVACY PROTECTION

Rollz International treats your privacy with care. All of your settings are stored on the module and on your smartphone/tablet.

WARRANTY

Your Rollz Motion Rhythm rollator is covered by the warranty conditions in the Rollz Motion user manual. The specific parts of your Rollz Motion Rhythm are covered by the following warranty periods:

1. The Rollz Rhythm module is under warranty for a period of 24 months.
2. The Rollz Motion Rhythm handles containing the vibrating part are covered by 12 months warranty.
3. The battery in the Rollz Motion Rhythm module is fully warranted for 12 months.

ACCESSORIES

The Rollz Motion Rhythm is supplied as a rollator. If you want to be able to turn it into a transport chair on the go, you can purchase the transport chair package. All Rollz Motion accessories also fit onto the Rollz Motion Rhythm.

USER TIPS

Tips for the optimal use of your rollator can be found at www.rollz.com.