



# Repositioning Sling

## SHEET STYLE

- The sling can be used in a side-lying position to facilitate a 'log-roll' movement which will give access for personal care, to change patient's position for help in assisting with pressure management and is also useful for changing soiled bedding. Can be used with a slide sheet positioned underneath to help with the glide and to ensure that the patient is positioned correctly back into the center of the bed after the turn.
- With sling attached to the spreader bar, patient can be lifted slowly and leveled correctly before repositioning. This procedure will help to assist the caregiver in repositioning the patient higher up in bed.
- The positioning sling comes with eight tapes plus two head support tapes. When fully attached to the lift, this creates a supine transfer sling useful for lateral transfers from a bed/stretcher to an operating theater or diagnostic table.
- 100% Polyester Mesh (High density open weave mesh, durable, used for bathing as quick drying.)



### Product Codes

One size	M921506-L
----------	-----------

### Guidelines

- SWL 454kg / 1000lb
- Wash 95°C / 203°F
- Low Tumble Dry

