

Instructions of Use

In-Situ Recline Leg

IN-SITU RANGE

Reference: **IS-RS-IOU**

Product Code/s: **M921507 - Silva-Superfine PLUS®**,
M921508 - Mesh

Revision: **07.08.2020**

Copyright & Design Rights: **Silvalea.**


Silva-Superfine® PLUS is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark.




WARNING

Do NOT use clip sling attachments on a loop type spreader bar and Do NOT use loop sling attachments on a stud type spreader bar.

SILVA-SUPERFINE PLUS®

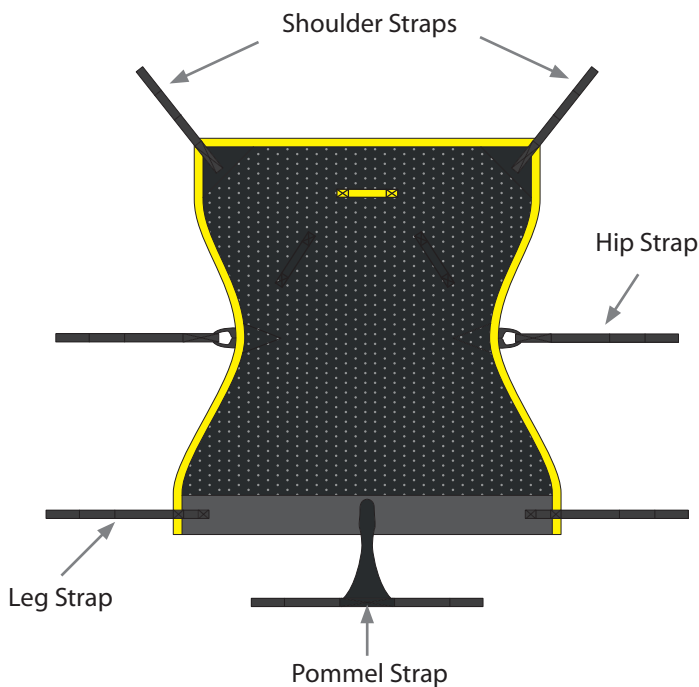


MESH

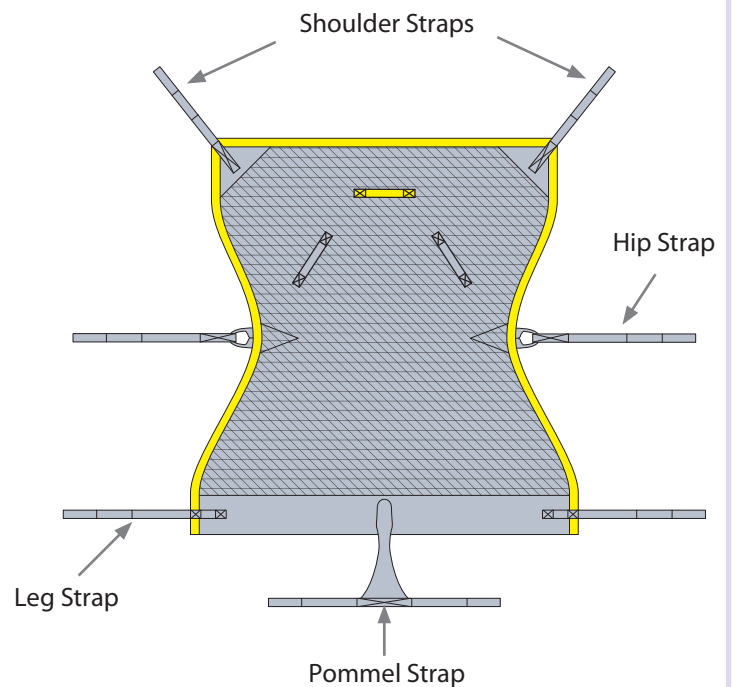


Loop Sling Attachments - Loop Type Spreader Bars:


Please note there are many variations of the loop type spreader bar but they should all be used exclusively by slings with: **LOOP SLING ATTACHMENTS**

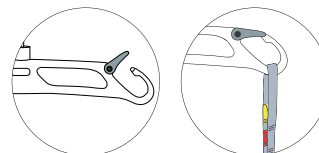


Sling Application:

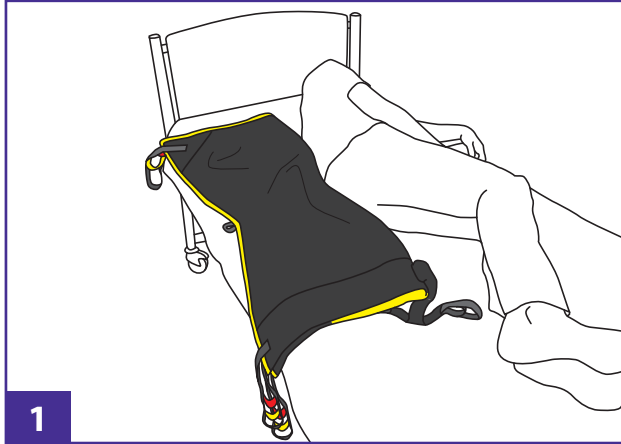


Fold the sling in half with the label inside the sling. Turn patient on their side and place folded edge of the sling against spine area.

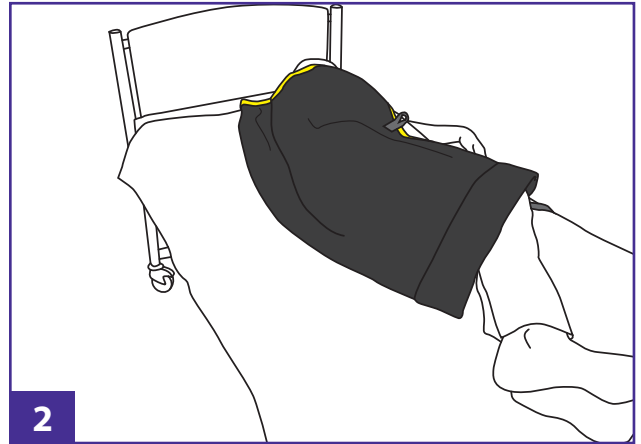


Open sling out and lay half over the patient then roll up the other half of the sling underneath against the spine area.

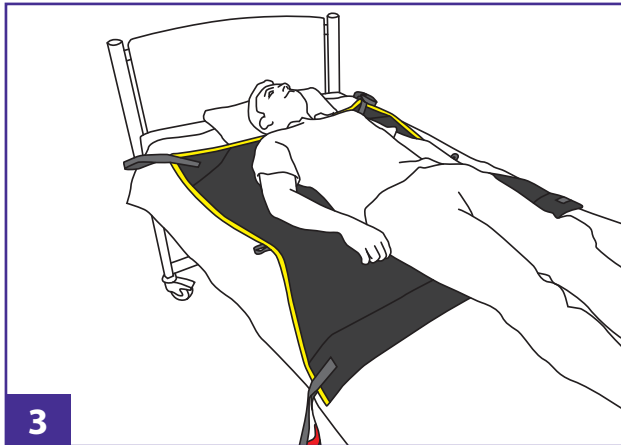

Roll patient onto the opposite side and pull through the underside of the sling. Lay them flat on their back and central to the sling.

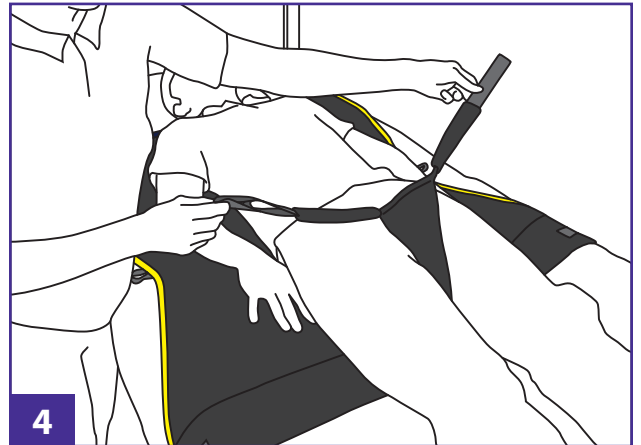


Pull the pommel strap through between the legs.

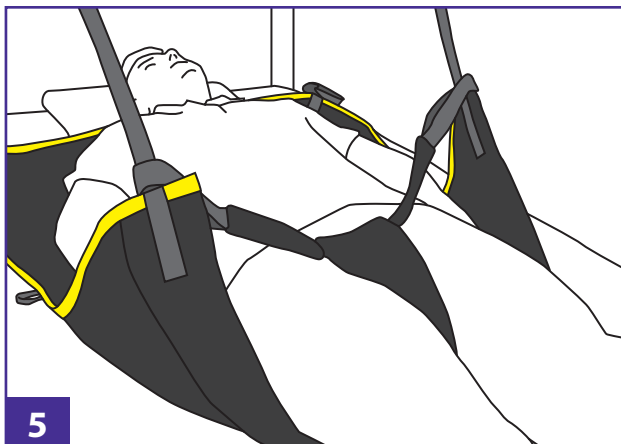

Once the sling has been fitted correctly, thread the leg straps through the pommel strap loops before attaching to the lift. Then attach the shoulder straps to the spreader bar of the lift.

Color coding on the pick up loops are provided for ease of reference and position.


Attaching Hip Straps:

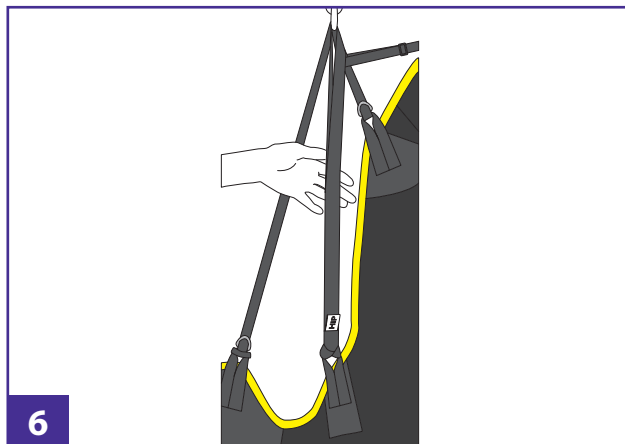

When attaching hip straps, the ideal pick up should enable the pelvic area and the legs to pickup simultaneously.

Adjustment can be made by using the color coded connections to enable the best pick up.

Ensure that all straps are not twisted and are attached securely before commencing the transfer.

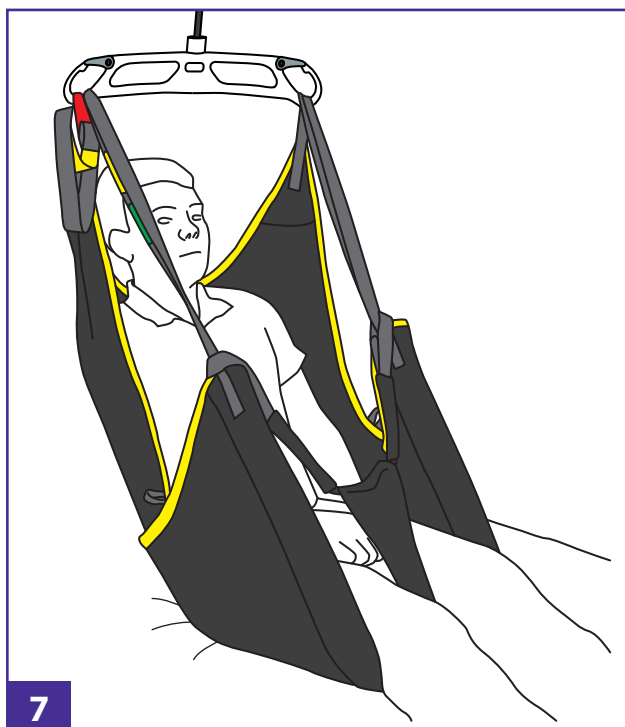

You can now begin the transfer. Once the sling takes the weight of the patient you may need to re-adjust the position of the sling for the patient's comfort.



Lower the patient back onto the chair/bed to make any adjustments before continuing the transfer.



General Instructions: Warning

Do not attempt to transfer the patient until all appropriate sling attachments are secured to the lift spreader bar and you are satisfied that the sling is the correct style, size and fit for the patient.

Correct sizing of a sling is important as a sling that is too large could cause the patient to slip through the aperture.

While every attempt is made to instruct a safe move you must conform to the current moving and handling procedures/regulations. If in any doubt, consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products should be checked each time they are used for any signs of deterioration.

The service label is a visual guide to the product's check. Please conduct a thorough sling check in line with current safety requirements in your country. If in doubt consult your distributor for current sling checking protocols.

A risk assessment must be carried out to ensure that the correct size and style of sling has been met before use.

Pre-use sling check list:

- braiding/straps for deterioration and fraying
- material for holes/snagging
- compatibility with the lift
- safe working load of both the lift and sling/product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or straps or the label is illegible.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- DO NOT IRON

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the patient.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convector heater or steam pipes.



SWL 272kg/600lb/42st