


Q-Series 42" Expander Kit

(Kit# Q6709 for Rexx and Advantage beds)

The Span MC Expanders Kit part no Q6709 is supplied with form-fitted packaging for protective storage and reuse. The kit contains 6 expander units and two mattress retainers, one each for the center of the head and foot decks for additional stability with wider mattresses in some applications.

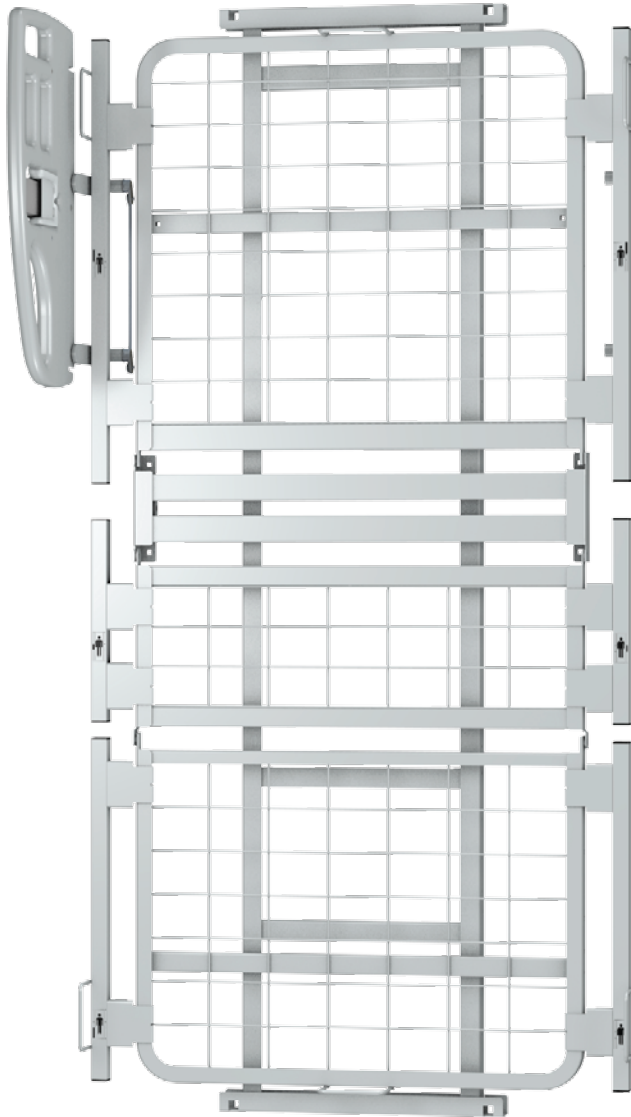

Bed frame, overhead view.

STEP 1

If the bed is fitted with half head assist rails it is necessary to remove them and the four corner mattress retainers prior to installation of the expanders. The six expander units will then install toollessly, converting the sleep surface to a 42" width. We recommend storing the original corner mattress retainers in the kit for reuse.



STEP 2

Note the placement decals on each of the six unique expander units and prepare to install them on either side of the head, seat and foot decks.



STEP 3

Using the figure (upper left) as a guide, connect each expander unit to the upper surface of the grid deck and then lower the expander until the spring loaded retaining latch attaches securely to the lower portion of the deck.



SPAN
better science. better care.

MC
BEDS AND FURNITURE

The expanders are fully compatible with Span MC half head assist rails. The rails should be fitted to the head deck expander units prior to installation. See p. 2 for instructions.

