



jstanzme

INSTRUCTION MANUAL STANZME PRONE STANDER

Read and follow all instructions for assembly
and use before using this product



Designed to provide the child with the ability to stand and bear the weight of the body in an upright or forward-leaning position that allows the user to relieve pressure created by long hours of seating for better digestion and bowel movements. The easy adjustment combinations and support accessories of the stander hold the child in good postural alignment and provide overall perks of standing

| Warnings

The stander should only be used and adjusted according to the guidance of the certified therapist or physician. The supervision of an adult is needed when the child is using the stander. Stanzme will not be responsible for any damage or injuries resulting from the modification of the stander by anyone other than the manufacturer. The company takes no responsibility for the damage and injuries caused by any neglect or misuse of the product.

***Never leave a child unattended when the product is in use.**

Use

The stanzme Prone stander is designed to provide the child with ability to experience weight bearing in an upright or forward leaning position while the combination of adjustments and supports hold the child in good postural alignment.

Never leave a child unattended.

Care

Keep the regular check on the product to detect the loose, missing hardware or any signs of cracked or broken parts to avoid unwanted damage to the user and the stander.

Contact the manufacturer immediately in case of any faults or product failure.

All components can be clean with mild soapy water or non-abrasive commercial cleaner approved for use on vinyl, wood, and metal.

Do not use excessive amounts of water, and make sure to wipe the woods dry after cleaning.

Warranty


The product warranty is of one year from the date of the purchase.

If any part of the product break or fail, the manufacturer will repair or replace it at no charge to the purchaser, except for the pneumatic tilt piston or if there is abuse on use by the client or caregiver.

FITTING AND ADJUSTING THE STANZME PRONE STANDER

Height Adjustment

The height of the stander generally adjusted with the child's mid-chest length. The top of the chest pad should come up to about in line with the child's armpit or ax illa. According to the body structure of the child, adjustments should determine by the therapist or the physician. The user can change the height of the stander by losing the height adjustment knobs on the verticle portion of the stander. The whole upper part of the stander can then elevated or reduced to the appropriate height. Be sure to re-tighten the knobs securely. The change in the height of the stander will also alter the other components/parts of the product.

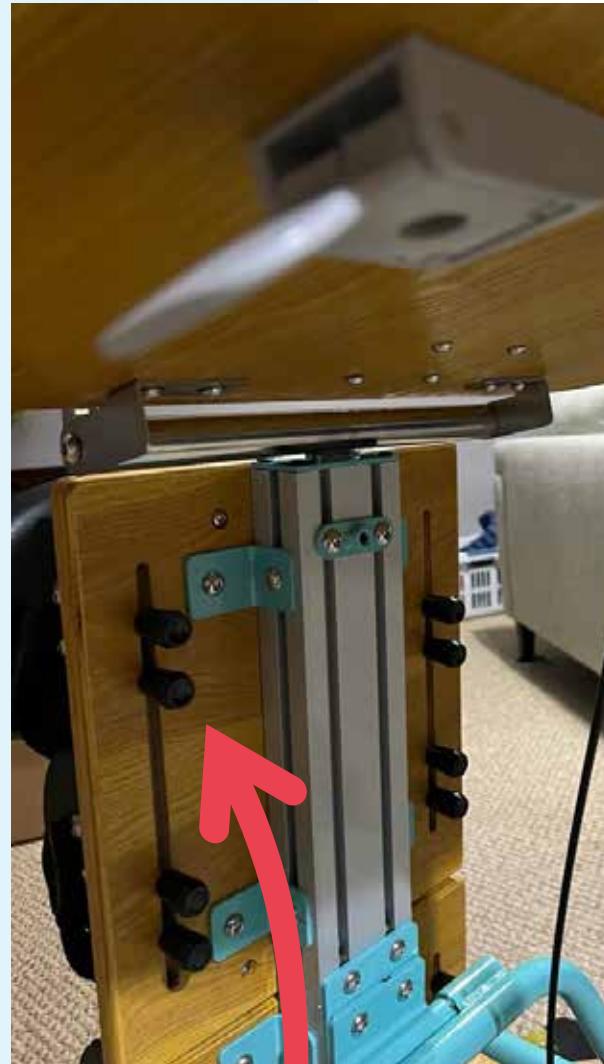

Height adjustment knobs for growth

Height Adjustment

The angle adjustment of the stander ranges from an upright 90° position to approximately 30° forward tilt. To change the angle of the stander press, or thumb the plate towards the stander to the Pneumatic pump plunger and move the stander to the desired angle.

The easy Pneumatic plunger allows a swift anterior infinite position with ease and gravity of the client hold the stander while adjusting the anterior angle incline.

Tray Adjustment

The tray is adjustable for both height and angle. To adjust the tray height/angle, loosen the knobs of the square center bar of the stander and raise or lower the whole tray assembly to the desired height.



**Angle
Adjustment Plate**

**Tray Adjustment
Knob**