

Warning Labels

6.1

Warnings are included for the safety of the user, client, operator and property. Please read and understand what the signal words **SAFETY, NOTICE, CAUTION, WARNING** and **DANGER** mean, how they could affect the user, those around the user, and property.

DANGER

Identifies an **imminent** situation which (if not avoided) will result in **severe injury, death, and property damage**.

WARNING

Identifies a **potential** situation which (if not avoided) will result in **severe injury, death, and property damage**.

CAUTION

Identifies a **potential** situation which (if not avoided) will result in **minor to moderate injury, and property damage**.

NOTICE

Identifies important information not related to injury, but possible **property damage**.

SAFETY

Indicates steps or instructions for safe practices, reminders of safe procedures, or important safety equipment that may be necessary.

Limited Liability

6.2

~~ODDDEW/0000~~ accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this manual.

~~ODDDEW/0000~~ does not hold responsibility of final integration of final assembly of product to end user. ~~ODDDEW/0000~~ is not liable for user death of injury.

Testing

6.3

Initial setup and driving should be done in an open area free of obstacles until the user is fully capable of driving safely.

The Tarta back should always be tested without any person sitting in the wheelchair until every alteration of the physical installation or adjustment is complete.

The Tarta System

7.1

The Tarta system is a backrest devised to be adaptable to each individual and their unique positioning needs. Using a combination of design, materials, and technology, Tarta delivers a product that is highly configurable but easy to use. It is a product that offers a high degree of comfort while introducing a uniquely functional design following the modular build-a-back philosophy.

The materials used in Tarta have been designed to achieve the highest levels of performance. The pre-stressed steel used for the vertical spine column flexes to follow the movement of the person, giving freedom of movement, and then returns to its original position. The padding is made of washable, breathable material.

The Tarta backrest consists of an exoskeleton similar to the rib cage in which aluminum ribs, (which are adjustable) and can be configured to conform to the position of the person, are attached to central "vertebra" (that can also be tailored in shape and size) that simulate a spinal column.

Parts Included

8.1


The Tarta system consists of a series of “vertebra” that, joined together by elements made of pre-stressed steel, reproduce the shape and characteristics of a spinal column.

The following items are included with each Tarta system:

- Vertebra
- Ribs
- Padding

Vertebra


Each column is made up of a series of vertebrae, from size ranges of 4 to 6 vertebrae, but the custom build-a-back nature of the product allows for virtually any height back configuration.

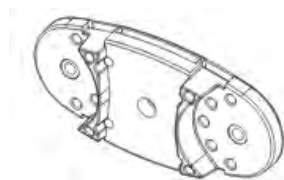

Each vertebra has a height of 50mm (1.97in) and a digression to 10mm (.39in).

Ribs


On each vertebra it is possible to fit ribs that are available in 4/5 different sizes, (S,M,L,XL) . These ribs provide both posterior and lateral support. Made of malleable aluminum, these ribs can be manually contoured in the field to create a more intimate fit.

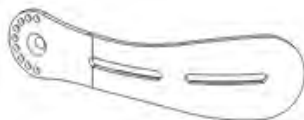

Each rib can be adjusted by moving the locking plug in order to find the exact position necessary.

Padding

Once installed, the padding is fitted and easily adjusted by means of the Velcro fasteners.

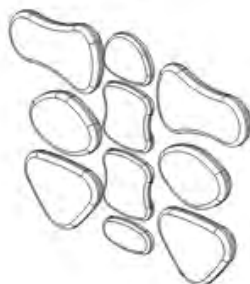

Hardware

Once the final configuration has been established, the pads can be permanently mounted to the ribs using the provided t-nuts.

Choosing Your Tarta Back

8.2

Reference the part number key below in order to help create your customized Tarta back.

Part Number Key

First 4 digits	Style	TRTB	Tarta Back
5th digit	Size	S	Small
		M	Medium
		L	Large
		XL	X-Large
6th digit	Vertebra Size	4	4 Vertebra
		5	5 Vertebra
		6	6 Vertebra
7th digit	Contour	A	Active

Available Tarta Back-in-a-Box packages:

- TRTBS4A
- TRTBS5A
- TRTBS6A
- TRTBM4A
- TRTBM5A
- TRTBM6A
- TRTBL4A
- TRTBL5A
- TRTBL6A
- TRTBX4A
- TRTBX5A
- TRTBX6A

The charts in the following pages show each set of ribs and pads that will come with the a standard Back-in-a-Box package ordered. These parts are for an **ACTIVE Contour Package** configuration of a Tarta back. Extra ribs and pads are provided if the user needs to change from an Active Contour to a **DEEP Contour Package**.

NOTICE

A punch is provided in each Back-in-a-Box package.

How To Determine Your Tarta Size

8.3

Back Support Height

8"	9"	10"	11"	12"	13"	14"	Varies depending on # of vertebrae
4 Vertebra		5 Vertebra		6 Vertebra		Custom– As many vertebrae as necessary	

NOTICE

The height of a custom Tarta back is dependent upon the number of vertebrae requested in the order.

Trunk Width

Small	Medium	Large	Extra Large
<10"	10"-12"	12"-14"	14"-17"

Hardware Selection

Tarta (TRT)			Hardware (HW)		Single (1) or Double Point (2)	Hardware Type (A, F, R)	Size	Marketing Part Number
T	R	T	H	W	1	A	L	TRTHW1AL
T	R	T	H	W	1	A	S	TRTHW1AS
T	R	T	H	W	1	F	L	TRTHW1FL
T	R	T	H	W	1	F	S	TRTHW1FS
T	R	T	H	W	1	R	L	TRTHW1RL
T	R	T	H	W	1	R	S	TRTHW1RS
T	R	T	H	W	2	R	L	TRTHW2RL
T	R	T	H	W	2	R	S	TRTHW2RS

A– Adjustable

F- Fixed



R– Removable

S– Small/Medium

L– Large/X-Large



Tarta Back Specs

8.4

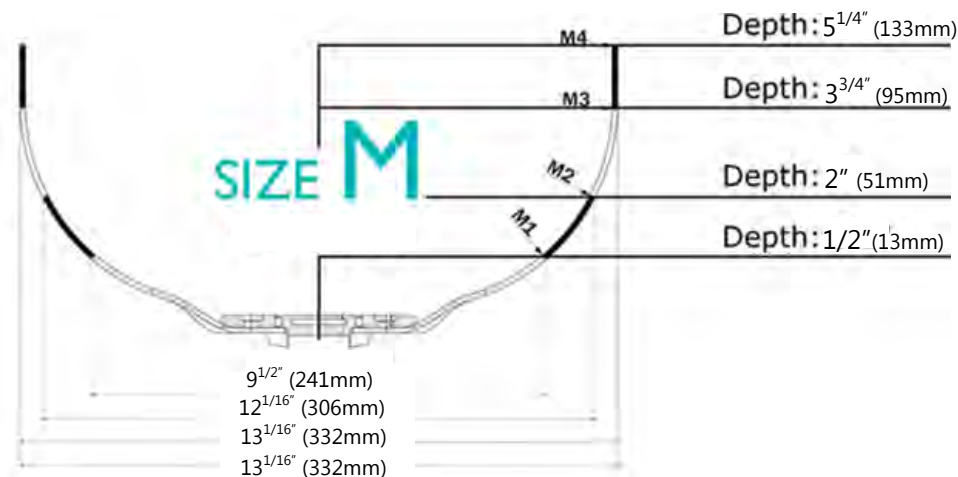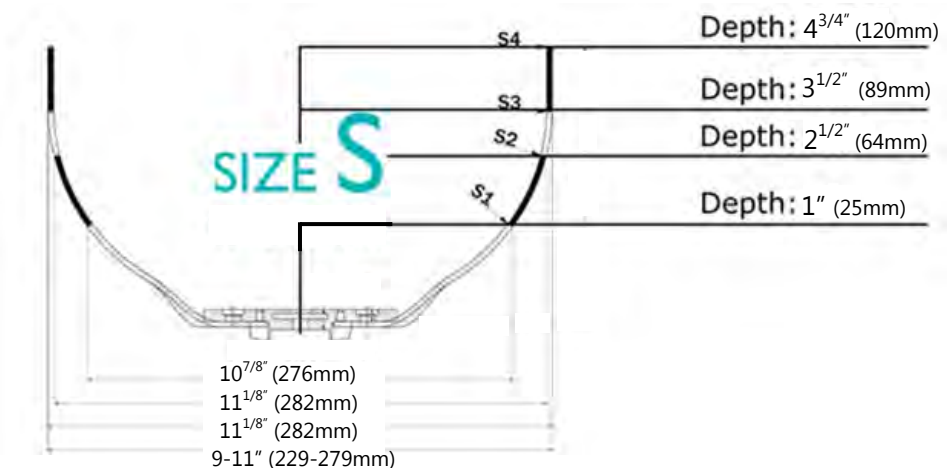

NOTICE

All measurements are approximate.

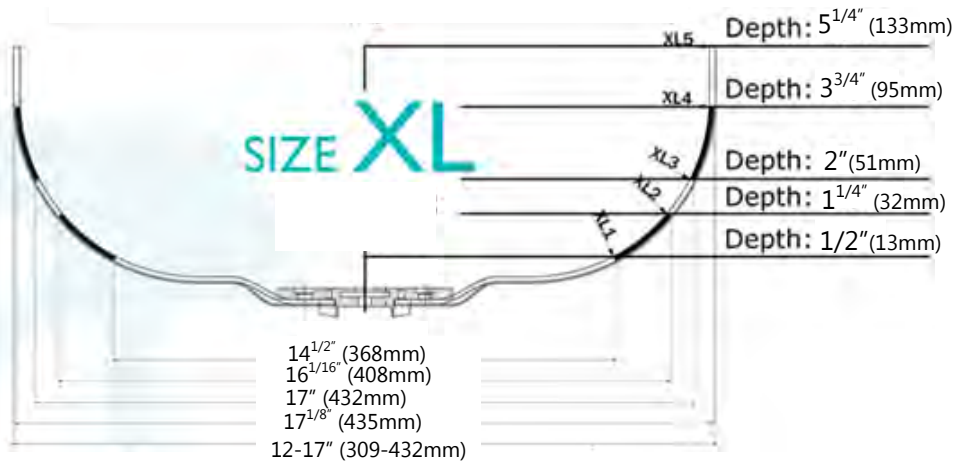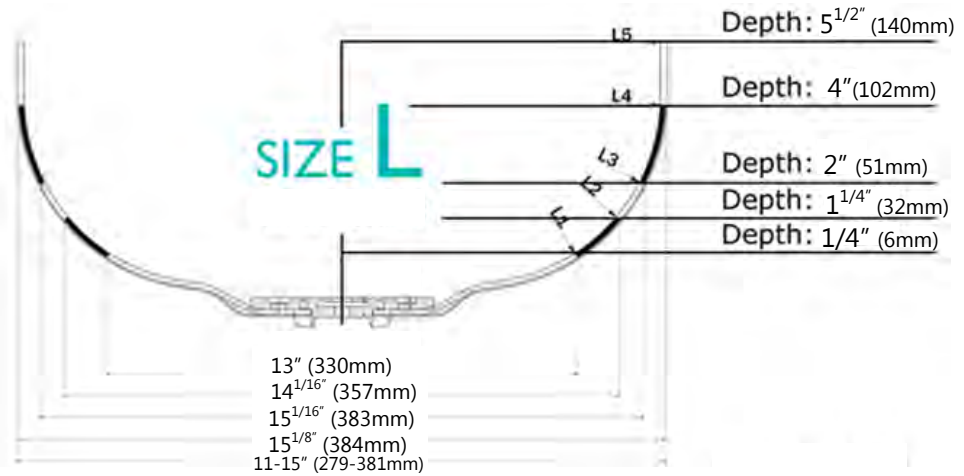










TRTBS4A Tarta Package

Height Range

8" - 9"


Trunk Width Range

<10"

Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-S1		TRT-RIB-S1	A 
B	 TRT-RIB-S2		TRT-RIB-S2	B 
C	 TRT-RIB-S2		TRT-RIB-S2	C 
D	 TRT-RIB-S1		TRT-RIB-S1	D 

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-056 	TRT-PAD-062 	TRT-PAD-056 
TRT-PAD-059 		TRT-PAD-051 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:

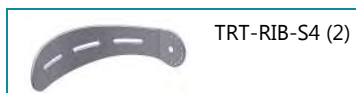

To create a deep contour on your Tarta back:

- Remove the first set of S2 ribs (B) and replace with the S4 ribs.
- Remove the 056 pads and replace with the 053 pads.
- 2 extra S3 ribs and 2 extra 051 pads are provided for the user.










TRTBM4A Tarta Package

Height Range

8" - 9"

Trunk Width Range

10" - 12"

Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-M1		TRT-RIB-M1	 A
B	 TRT-RIB-M2		TRT-RIB-M2	 B
C	 TRT-RIB-M3		TRT-RIB-M3	 C
D	 TRT-RIB-M2		TRT-RIB-M2	 D

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-056 	TRT-PAD-062 	TRT-PAD-056 
TRT-PAD-060 	TRT-PAD-051 	TRT-PAD-060 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on your Tarta back:

- Remove the M1 (A) and first set of M2 ribs (B). Shift the M2 ribs up (spot A) and in place of the M1 ribs.
- Attach the M4 ribs in place of M2 ribs (spot B).
- Remove the 056 pads and replace with 054 pads.
- 2 extra 051 pads are provided for the user.










TRTB4A Tarta Package

Height Range

8" - 9"

Trunk Width Range

12" - 14"

Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-L1		TRT-RIB-L1	 A
B	 TRT-RIB-L3		TRT-RIB-L3	 B
C	 TRT-RIB-L3		TRT-RIB-L3	 C
D	 TRT-RIB-L2		TRT-RIB-L2	 D

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-057 	TRT-PAD-062 	TRT-PAD-057 
TRT-PAD-060 	TRT-PAD-051 	TRT-PAD-060 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on your Tarta back:

- Remove the first set of L3 ribs (B) and replace with the L4 ribs.
- Remove the 057 pads and replace with the 054 pads.
- Two extra L5 ribs and 051 pads are provided for the user.










TRTBX4A Tarta Package






Height Range

8" - 9"



Trunk Width Range

14" - 17"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-XL1		TRT-RIB-XL1  A
B  TRT-RIB-XL3		TRT-RIB-XL3  B
C  TRT-RIB-XL3		TRT-RIB-XL3  C
D  TRT-RIB-XL2		TRT-RIB-XL2  D

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-058 	TRT-PAD-062 	TRT-PAD-058 
TRT-PAD-060 		TRT-PAD-051 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:

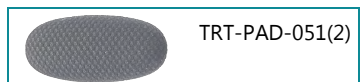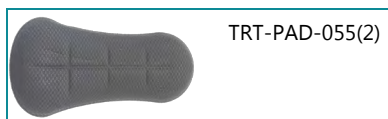

To create a deep contour on your Tarta back:

- Remove the first set of XL3 ribs (B) and replace with the XL4 ribs.
- Remove the 058 pads and replace with the 055 pads.
- Two extra XL5 ribs and 051 pads are provided for the user.


TRTS5A Tarta Package








Height Range

10" - 11"

Trunk Width Range

<10"

Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-S1		TRT-RIB-S1	
B	 TRT-RIB-S2		TRT-RIB-S2	
C	 TRT-RIB-S2		TRT-RIB-S2	
D	 TRT-RIB-S2		TRT-RIB-S2	
E	 TRT-RIB-S1		TRT-RIB-S1	

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-056 	TRT-PAD-062 	TRT-PAD-056 
TRT-PAD-051 		TRT-PAD-052 
TRT-PAD-059 		TRT-PAD-059 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



TRT-RIB-S3(2)



TRT-RIB-S4(2)



TRT-PAD-053(2)

To create a deep contour on your Tarta back:

- Remove the first set of S2 ribs (B) and replace with the S4 ribs.
- Remove the 056 pads and replace with the 053 pads.
- Two extra S3 ribs are provided for the user.









TRTM5A Tarta Package









Height Range

10" - 11"

Trunk Width Range

10" - 12"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-M1		TRT-RIB-M1  A
B  TRT-RIB-M2		TRT-RIB-M2  B
C  TRT-RIB-M2		TRT-RIB-M2  C
D  TRT-RIB-M3		TRT-RIB-M3  D
E  TRT-RIB-M2		TRT-RIB-M2  E

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-056 	TRT-PAD-062 	TRT-PAD-056 
TRT-PAD-051 		TRT-PAD-051 
TRT-PAD-060 	TRT-PAD-052 	TRT-PAD-060 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on your Tarta back:

- Remove the first set of M2 ribs (B) and replace with the M4 ribs.
- Remove the 056 pads and replace with the 054 pads.












TRTBLSA Tarta Package









Height Range

10" - 11"

Trunk Width Range

12" - 14"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-L2		TRT-RIB-L2  A
B  TRT-RIB-L3		TRT-RIB-L3  B
C  TRT-RIB-L2		TRT-RIB-L2  C
D  TRT-RIB-L3		TRT-RIB-L3  D
E  TRT-RIB-L2		TRT-RIB-L2  E

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-057 	TRT-PAD-062 	TRT-PAD-057 
TRT-PAD-051 		TRT-PAD-051 
TRT-PAD-060 	TRT-PAD-052 	TRT-PAD-060 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on your Tarta back:

- Remove the first set of L3 ribs (B) and replace with the L5 ribs.
- Remove the second set of L3 ribs (D) and replace with the L4 ribs.
- Remove the bottom set of L2 ribs (E) and replace with the L3 ribs.
- Remove the 057 pads and replace with the 054 pads.












TRTBX5A Tarta Package









Height Range

10" - 11"

Trunk Width Range

14" - 17"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-XL1		TRT-RIB-XL1  A
B  TRT-RIB-XL3		TRT-RIB-XL3  B
C  TRT-RIB-XL2		TRT-RIB-XL2  C
D  TRT-RIB-XL3		TRT-RIB-XL3  D
E  TRT-RIB-XL2		TRT-RIB-XL2  E

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
TRT-PAD-058 	TRT-PAD-062 	TRT-PAD-058 
TRT-PAD-051 		TRT-PAD-051 
TRT-PAD-060 	TRT-PAD-052 	TRT-PAD-060 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



TRT-RIB-XL4(2)



TRT-RIB-XL5(2)



TRT-PAD-055(2)



TRT-PAD-061(2)

To create a deep contour on your Tarta back:

- Remove the first set of XL3 ribs (B) and replace with the XL5 ribs.
- Remove the bottom set of XL3 ribs (D) and replace with the XL4 ribs.
- Remove the 058 pads and replace with the 055 pads.
- Remove the 060 pads and replace with the 061 pads.

TRTBS6A Tarta Package










Height Range

12"

Trunk Width Range

<10"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-S1		TRT-RIB-S1  A
B  TRT-RIB-S2		TRT-RIB-S2  B
C  TRT-RIB-S2		TRT-RIB-S2  C
D  TRT-RIB-S1		TRT-RIB-S1  D
E  TRT-RIB-S2		TRT-RIB-S2  E
F  TRT-RIB-S1		TRT-RIB-S1  F

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
G  TRT-PAD-056	TRT-PAD-062 	TRT-PAD-056  G
H  TRT-PAD-056	TRT-PAD-052 	TRT-PAD-056  H
I  TRT-PAD-059	TRT-PAD-051 	TRT-PAD-059  I

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on your Tarta back:










- Remove the first set of S2 ribs (B) and replace with the S4 ribs.
- Remove the bottom set of S2 ribs (E) and replace with the S3 ribs.
- Remove the bottom set of S1 ribs (F) and replace with the S2 ribs.
- Remove the first set of 056 pads (G) and replace with 053 pads.
- Two extra 051 pads are provided for the user.

TRTBM6A Tarta Package

Height Range
12" - 13"

Trunk Width Range
10" - 12"

Ribs Sequence (Right)	Spine	Ribs Sequence (Left)
A  TRT-RIB-M1		TRT-RIB-M1  A
B  TRT-RIB-M2		TRT-RIB-M2  B
C  TRT-RIB-M2		TRT-RIB-M2  C
D  TRT-RIB-M1		TRT-RIB-M1  D
E  TRT-RIB-M3		TRT-RIB-M3  E
F  TRT-RIB-M2		TRT-RIB-M2  F

Lateral Pads (Right)	Central Pads	Lateral Pads (Left)
G  TRT-PAD-056	TRT-PAD-062 	TRT-PAD-056  G
H  TRT-PAD-056	TRT-PAD-052 	TRT-PAD-056  H
I  TRT-PAD-060	TRT-PAD-051 	TRT-PAD-060  I

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



To create a deep contour on a Tarta back:

- Remove the first set of M2 ribs (B) and replace with the M4 ribs.
- Remove the second set of M1 ribs (D) and replace with the M3 ribs.
- Remove the first set of 056 pads (G) and replace with the 054 pads.
- Remove the bottom set of 056 pads (H) and replace with the 057 pads.
- Two extra 051 pads are provided for the user.

TRTB6A Tarta Package







Height Range

13"

Trunk Width Range

12" - 14"


Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-L2		TRT-RIB-L2	A
B	 TRT-RIB-L3		TRT-RIB-L3	B
C	 TRT-RIB-L2		TRT-RIB-L2	C
D	 TRT-RIB-L3		TRT-RIB-L3	D
E	 TRT-RIB-L3		TRT-RIB-L3	E
F	 TRT-RIB-L2		TRT-RIB-L2	F

Lateral Pads (Right)		Central Pads	Lateral Pads (Left)	
G	TRT-PAD-057 	TRT-PAD-062 	TRT-PAD-057	G
H	TRT-PAD-057 	TRT-PAD-052 	TRT-PAD-057	H
I	TRT-PAD-060 	TRT-PAD-051 	TRT-PAD-060	I

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:



TRT-RIB-L4(2)

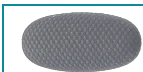

TRT-PAD-051(2)



TRT-RIB-L5(2)



TRT-PAD-054(2)

To create a deep contour on a Tarta back:

- Remove the first set of L3 ribs (B) and replace with the L5 ribs.
- Remove the last set of L3 ribs (E) and replace with the L4 ribs.
- Remove the bottom set of L2 ribs (F) and replace with the L3 ribs.
- Remove the first set of 057 pads (G) and replace with the 054 pads.
- Two extra 051 pads are provided for the user.


TRTBX6A Tarta Package










Height Range

14"


Trunk Width Range

14" - 17"

Ribs Sequence (Right)		Spine	Ribs Sequence (Left)	
A	 TRT-RIB-XL1		TRT-RIB-XL1	A 
B	 TRT-RIB-XL3		TRT-RIB-XL3 	
C	 TRT-RIB-XL2		TRT-RIB-XL2 	
D	 TRT-RIB-XL1		TRT-RIB-XL1 	
E	 TRT-RIB-XL3		TRT-RIB-XL3 	
F	 TRT-RIB-XL2		TRT-RIB-XL2 	

Lateral Pads (Right)		Central Pads	Lateral Pads (Left)	
G	TRT-PAD-058 	TRT-PAD-062 	TRT-PAD-058	G 
H	TRT-PAD-057 	TRT-PAD-052 	TRT-PAD-057	H 
I	TRT-PAD-060 	TRT-PAD-051 	TRT-PAD-060	I 

The following ribs and pads are also included in the package to transform a standard active back configuration into a **DEEP** contour back:

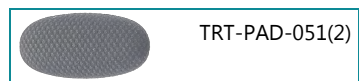


To create a deep contour on a Tarta back:





- Remove the first set of XL3 ribs (B) and replace with the XL5 ribs.
- Remove the second set of XL1 ribs (D) and replace with the XL3 ribs.
- Remove the second set of XL3 ribs (E) and replace with the XL4 ribs.
- Remove the 058 pads (G) and replace with the 055 pads.
- Remove the 057 pads (H) and replace with the 058 pads.
- Remove the 060 pads (I) and replace with the 061 pads.






Rib Selection






8.5

The charts below list the ribs available to create a custom Tarta back:

Rib Size	Product	Part Number
S		TRT-RIB-S1
S		TRT-RIB-S2
S		TRT-RIB-S3
S		TRT-RIB-S4

Rib Size	Product	Part Number
M		TRT-RIB-M1
M		TRT-RIB-M2
M		TRT-RIB-M3
M		TRT-RIB-M4













Rib Size	Product	Part Number
L		TRT-RIB-L1
L		TRT-RIB-L2
L		TRT-RIB-L3
L		TRT-RIB-L4
L		TRT-RIB-L5

Rib Size	Product	Part Number
XL		TRT-RIB-XL1
XL		TRT-RIB-XL2
XL		TRT-RIB-XL3
XL		TRT-RIB-XL4
XL		TRT-RIB-XL5

Pad Selection

8.6

12 pads of different sizes are available to create a custom Tarta back:

Product	Part Number	Product	Part Number
 $4^{3/8} \times 2^{1/16}$	TRT-PAD-051	 $6^{1/8} \times 4^{3/8}$	TRT-PAD-057
 $4^{3/8} \times 4^{1/2}$	TRT-PAD-052	 $7 \times 4^{1/2}$	TRT-PAD-058
 $7^{1/16} \times 4^{7/8}$	TRT-PAD-053	 $5^{1/16} \times 4^{7/8}$	TRT-PAD-059
 $8^{1/4} \times 4^{7/8}$	TRT-PAD-054	 $6^{3/4} \times 5$	TRT-PAD-060
 $9^{1/2} \times 4^{7/8}$	TRT-PAD-055	 $4^{1/2} \times 5^{1/8}$	TRT-PAD-061
 $5^{3/8} \times 4^{1/8}$	TRT-PAD-056	 $4^{1/2} \times 7^{5/8}$	TRT-PAD-062

Mounting Hardware Options

8.7



Description	Part Number	12	13	14	15	16	17	18	19	20
Tarta Hardware Kit, Single Point, Adjustable Height, Small-Medium	TRTHW1AS	X	X	X	X	X				
Tarta Hardware Kit, Single Point, Adjustable Height, Large-X Large	TRTHW1AL						X	X	X	X



Description	Part Number	12	13	14	15	16	17	18	19	20
Tarta Hardware Kit, Single Point, Fixed, Small-Medium	TRTHW1FS	X	X	X	X	X				
Tarta Hardware Kit, Single Point, Fixed, Large- X Large	TRTHW1FL						X	X	X	X



Description	Part Number	12	13	14	15	16	17	18	19	20
Tarta Hardware Kit, 2 Point, Removable, Small-Medium	TRTHW2RS		X	X	X					
Tarta Hardware Kit, 2 Point, Removable, Large- X Large	TRTHW2RL					X	X	X	X	X



Description	Part Number	12	13	14	15	16	17	18	19	20
Tarta Hardware Kit, Single Point, Removable, Small-Medium	TRTHW1RS			X	X					
Tarta Hardware Kit, Single Point, Removable, Large-X Large	TRTHW1RL					X	X	X		

Dealer Assistance

9.1

During first time use by the client, it is advised that the dealer or service technician assists and explains the different configurations to the client. If needed, the dealer can make final adjustments.

User Testing

9.2

It is important that the customer is fully aware of the installation and how to adjust the position of the ribs and vertebrae in order to have the Tarta back in the best position for the user.

Have the user test the position of the back:

- Is the placement of the Tarta back in an optimal position for the user?
- Can the user safely operate chair with the placement of the back?

If needed, adjust the Tarta back to the best position for the user.

Explain to the client the possible problems that could arise with the hardware and how to address them.

Conditions of Use

9.3

The Tarta back is intended for use as installed by the dealer, in accordance to the installation and instructions in this manual.

- The foreseen conditions of use are communicated by the dealer or service technician to the user and/or attendant during the first time use.
- If the usage conditions change significantly, please contact your dealer or a qualified service technician to avoid excessive wear and tear or unintended damage.



Failure to comply with these warnings could result in serious bodily harm.

Cleaning

10.1

- The individual parts of the device can be cleaned with a soft, damp cloth.
- Do not immerse the device in water.
- The pads are machine washable. Do not wash with water warmer than 86°F (30°C).

SAFETY

The Tarta back should never come into direct contact with the client's skin. A garment or protective material should be worn to avoid skin irritation.

Maintenance

10.2

- Avoid direct contact with fire. The flames could damage the device and cause injury to the user.
- Do not leave the backrest inside the car when exposed to the sun. Excessive temperatures can damage the adhesive applied on the Velcro strap slats.
- Avoid contact with sharp and/or pointed objects or instruments that might damage the device and/or the padding.
- The Tarta backrest has been designed to support a maximum weight of 220 pounds (100kg).

SAFETY

Periodically check that the mechanical parts that make up the device have not become loose or have become misaligned.

