

Please read all of the following information before using this product.

TherAdapt[®] Products Inc. has designed and manufactured this product with the well being of the user and caregiver(s) in mind. It is our endeavor to provide products which are attractive, comfortable, useful, highly adjustable, long lasting and safe. As when using any product, precautions must be taken.



WARNING: This product must only be used under the recommendation of a physician and / or licensed therapist and with adult supervision. Never leave user unattended. It is your responsibility to properly use and maintain your Low Back Insert. Failure to follow USE and CARE instructions or ASSEMBLE the insert properly as outlined may result in bodily injury.

THE MANUFACTURER IS NOT RESPONSIBLE FOR INJURIES RESULTING FROM ANY CUSTOMIZATION OR MODIFICATION OF THE EQUIPMENT THAT WAS NOT COMPLETED BY THE MANUFACTURER.

USE: The Low Back Insert is designed for a single purpose: as a positioning support attached to a TherAdapt[®] Adjustable Positioning Chair or TherAdapt[®] Bolster Chair **only**. **No other use is permitted.**

CARE: Take care to inspect the ongoing condition of all parts of your Low Back Insert. Verify that all pads, hook and / or loop fastener, knobs, nuts, bolts, and washers are properly positioned and secure. Contact the manufacturer for replacement or repair of necessary items. All components may be cleaned with mild soap and water, or a gentle sanitizing spray. **Do not** use abrasive cleaners. Wipe wood components dry after cleaning. Excess amounts of liquid on the wood may damage the finish. Always store indoors.

WARRANTY: The manufacturer provides a full one (1) year warranty, beginning the date shipped by the manufacturer. The warranty provides full replacement of all parts against breakage when used in compliance with the manufacturer's USE and CARE instructions. Use or modification beyond the normal capacity or range of adaptability designed into the products will void the warranty. The foregoing is TherAdapt's sole liability for consequential damages.

PLEASE COMPLETE AND RETAIN FOR YOUR RECORDS

Low Back Insert LBI- _____

Serial Number: _____

Purchase date: _____

TherAdapt[®] Low Back Insert (LBI-100/200/300) Instructions

Packing List:

- (1) Insert with (2) large back liner pads, and (1) small back liner pad
- (2) Thoracic pads with (2) D-rings and (4) ¼ -20 hanger bolts
- (1) Wooden “Yoke” *
- (3) 6” ¼ -20 carriage bolts *
- (3) 4” ¼ -20 carriage bolts *
- (2) 1 ¾” ¼ -20 hex head bolts *
- (6) 7/16” ¼ -20 hex nuts *
- (6) Metal washers *
- (3) “Flower” knobs with ¼ -20 threaded inserts *
- (2) Thru-knobs with ¼ -20 threaded inserts *
- (6) Nylon washers *

Tools Needed:


- (1) 7/16” open end wrench


* Included in Hardware Bag

The Low Back Insert is designed to interface with the *TherAdapt*[®] Adjustable Positioning Chair or *TherAdapt*[®] Bolster Chair only. Assemble, attach and adjusts as follows:

1. Assemble the Low Back Insert:

- A. Select either the 4” (for up to 3” of seat depth reduction) or the 6” (for up to 5” of seat depth reduction) carriage bolts for use. (Keep the unused set on hand.) Insert (2) of the bolts into the two holes at the bottom of the insert. Place a metal washer, followed by a nut, on each bolt and secure it in place. **Important:** With a 7/16” wrench (not included) tighten the nuts securely to “pull” the square neck of the bolt into the wood on the front side of the insert.
- B. Insert (1) bolt through the center slot. Place a metal washer, followed by a nut, on the bolt and secure it in place loosely until final position is determined. Tighten as above once height adjustment is finalized.
- C. Place a 2nd nut, followed by a metal washer, on the bolt. **Note:** The greater the distance between the nuts the shorter the seat depth on the Adjustable Positioning Chair.

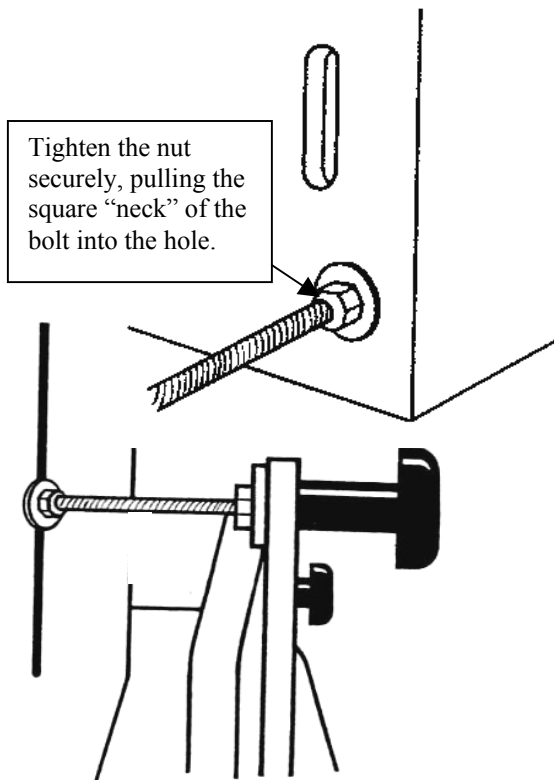

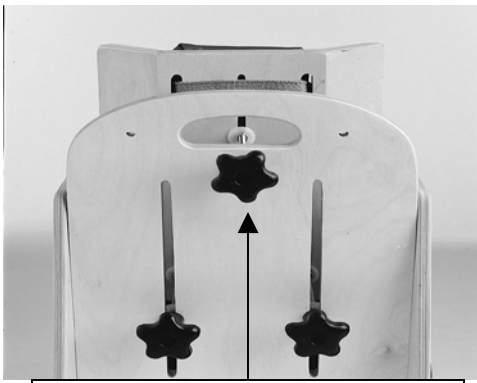


Note: For smaller users DO NOT use the “yoke”. Attach the 3rd bolt of the Corner Back Insert here.

2. **Attach the wooden “yoke” to the back of the chair.** This can be placed on the “inside” or the “outside” of the chair back depending on the depth of the seat needed. Place the (2) hex bolts through the pre-drilled holes. Place a nylon washer between the wood and the knob. Secure with small knobs. **Tighten securely.**
Note: For smaller users DO NOT attach the “yoke”.


3. **Attach the Insert to the chair.**

- A. Place the lower carriage bolts through the slots in the back of the chair. Place the upper carriage bolt through the hole in the top of the yoke. For the smaller user place the upper bolt through the hole (see note to left) in the chair back and store the yoke for later use. **Note:** The washer-nut / nut-washer combination should be on the “inside” of the chair. Secure by placing the “flower” knobs (see illustration) on the ends of the carriage bolts. **Tighten all knobs securely before use.**

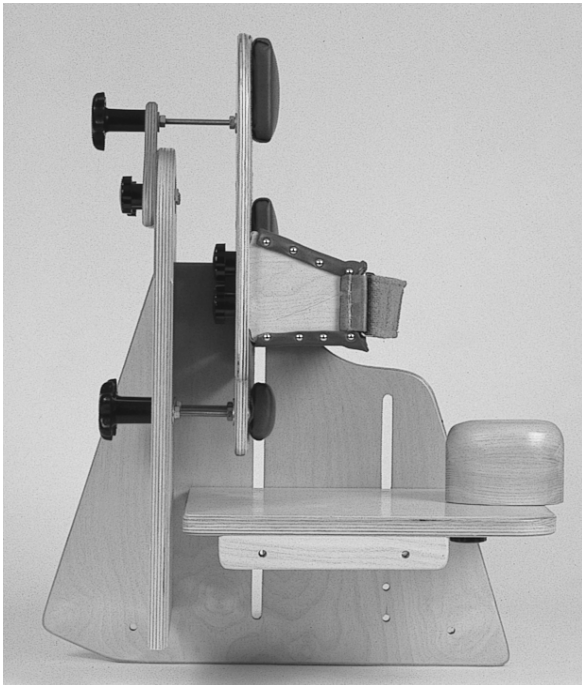

- B. The Insert is adjusted vertically by loosening the 1st nut on the top bolt to allow the bolt to slide in the insert slot. Loosening the lower pair of “flower” knobs allows the bolts to slide in the chair slots. **Re-tighten the nut and knobs securely** once the appropriate height is obtained.

- C. The seat depth (of the Adjustable Positioning Chair) can be reduced by up to 3” (with the 4” bolts) or 5” (with the 6” bolts) by changing the position of the 2nd nut in relation to the 1st nut. **Note:** The greater the distance between the two (2) nuts, the greater the seat depth reduction.

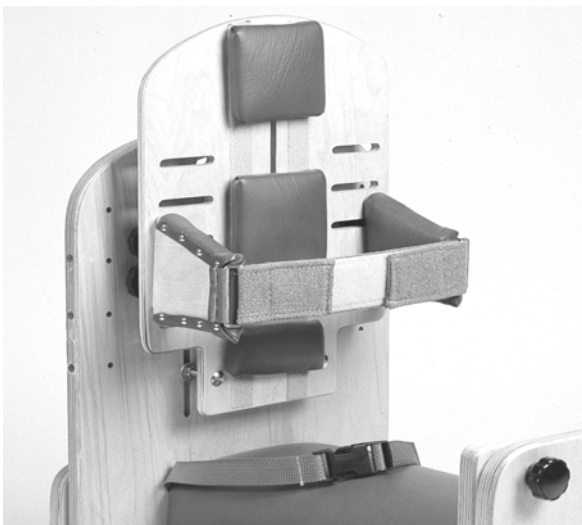

- D. When attaching the Low Back Insert to the Bolster chair use the 4” bolts and place the insert close to the back of the chair.