


Front Pull Static Hip Harness

- ⚠ Caution: Read Instructions before operating device.
- ⚠ Caution: Device should be installed by a trained wheelchair technician.
- ⚠ Caution: Use caution when installing, removing or adjusting device to avoid pinching or trapping fingers.
- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.
- ⚠ Warning: This device is NOT intended as an occupant restraint in a moving vehicle. This device is for positioning use only and should not be relied upon as a transit safety restraint. Use only WC-3 tested products for occupant restraint in a moving vehicle.

Product Identification


Static Hip Harness (Small, Medium, or Large)

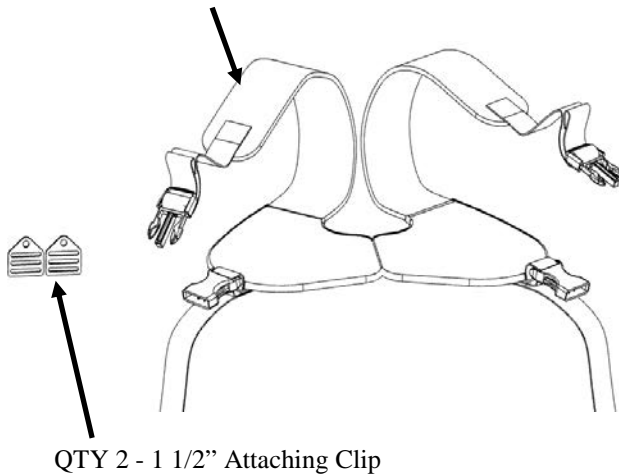


To Install:

Step 1: Mount attaching clips to chair as desired and thread strap as shown.


Metal Attaching Clip & 1" Or 1.5" Strap

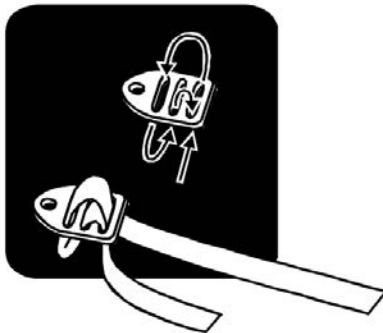

Step 2: Place harness in chair as shown below and wrap pads around user's legs.

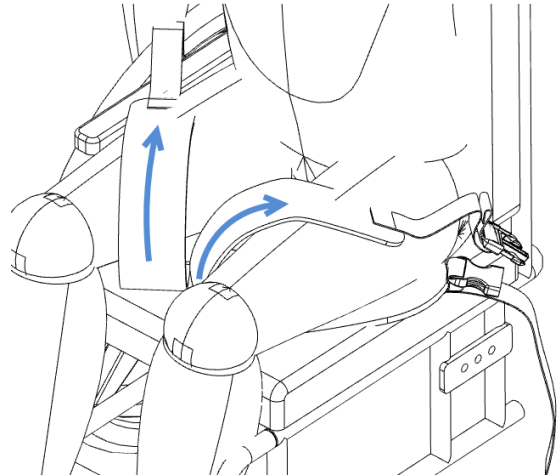

Step 3: Fasten buckles around either leg of the user and pull strap to tighten.

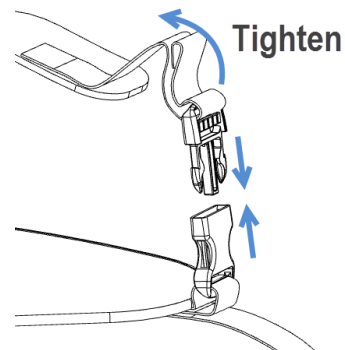

Care and Cleaning

- ⚠ Caution: Always tighten all fasteners securely and inspect fasteners regularly for tightness.

1. Clean with mild detergent and water using damp cloth. Air Dry.



2. Check all fasteners and retighten as needed.

Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.

Document 08571
Revision A 04-2018

