

# ThevoautoActiv

DE	Bedienerhandbuch – ThevoautoActiv .....	3
GB	User manual – ThevoautoActiv.....	6
FR	Manuel d'utilisation – ThevoautoActiv .....	9
ES	Manual de usuario – ThevoautoActiv .....	12
IT	Manuale d'istruzioni – ThevoautoActiv .....	15
NL	Gebruiksaanwijzing – ThevoautoActiv.....	18
NO	Bruksanvisning – ThevoautoActiv .....	21
SE	Bruksanvisning – ThevoautoActiv .....	24
DK	Brugsanvisning – ThevoautoActiv .....	27
FI	Käyttäjän käsikirja – ThevoautoActiv.....	30
PL	Instrukcja obsługi – ThevoautoActiv.....	32

**Dear User,**

You have just acquired a product - the ThevoautoActiv mobility bed - that meets the highest quality standards. ThevoautoActiv features the latest materials for an extended service life that's guaranteed. Please read through this manual carefully before using ThevoautoActiv. It contains lots of useful information and tips. If you have any further questions or suggestions concerning ThevoautoActiv give us a call.

**Contents**

	Page
Technical data .....	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning information .....	7
Application risks and contraindications.....	8
Warranty conditions.....	8

**Pressure ulcer prevention and treatment**

ThevoautoActiv has been developed for **the prevention and treatment of pressure ulcers up to and including the stage III according to EPUAP** (European Pressure Ulcer Advisory Panel). ThevoautoActiv is especially suited for patients with neurological diseases and pain. Besides an excellent pressure distribution the system supports perception and movement. The BVMed (Bundesverband Medizintechnologie e.V. – German association medical technology) developed a provision guideline for the selection of devices against decubitus.

**Technical data**

ThevoautoActiv external dimensions	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")
Weight ThevoautoActiv 50	20 - 25 kg / 44 - 55 lbs*	
Weight ThevoautoActiv 80		
Weight ThevoautoActiv 135		
Hight ThevoautoActiv 135	19.0 cm (7.5")	19.0 cm (7.5")

**Other dimensions available on request.**

**Scope of delivery**

- ThevoautoActiv with jersey or incontinence cover
- User manual

**Usage**

The product is compatible for use with every type of bed. Lay ThevoautoActiv with the red side facing upwards on the existing bed slats or bed base.

**Important hint:** ThevoautoActiv can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm (5,1 ") wide.

**Attention:** Beds with side-rails may require additional side-rail height extenders.

**Disinfection methods and cleaning information**

**Mattress**

The washable covers are easy to remove thanks to an integrated zip.

**Jersey cover (fabric cover)**

- washable at 65 °C (149 °F) • suitable for tumble dryers

**Incontinence cover (rubberised cover)**

- washable at 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

**Disinfection**

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

**Important hint:** A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

## Application risks and contraindications

The use of ThevoautoActiv does not require any special instructions. Referring to the previous practical use and several tests executed, we are not knowledgeable of any problem in the use of this product. When used properly, risks can be excluded.

**Important hint:** For the correct supply it is important to ensure that the mattress is selected according to the body weight of the user in question.

- ThevoautoActiv 50 up to a body weight of 50 kg (110.2 lbs)
- ThevoautoActiv 100 from 50 kg (110.2 lb) up to a body weight of 100 kg (220.4 lbs)
- ThevoautoActiv 135 from 100 kg (220.4 lb) up to a body weight of 135 kg (297.6 lbs)

ThevoautoActiv 100 may also be used in the border zone of patients weighing 50 kg/110.2 lb (+/- 2 kg / +/- 4.4 lb).

ThevoautoActiv 135 may also be used in the border zone of patients weighing 100 kg/220.4 lb (+/- 2 kg / +/- 4.4 lb).

If the body weight is distributed very unequally (e.g. strong pelvis or stomach, leg amputation), a mattress for higher weight should be used.

## Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoautoActiv mattresses are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoautoActiv receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

**Important hint:** ThevoautoActiv is autoclavable and may be re-used. ThevoautoActiv should be cleaned and disinfected using one of the cleaning methods stated on page 7 with every change of user. Servicing by the manufacturer is not necessary.