

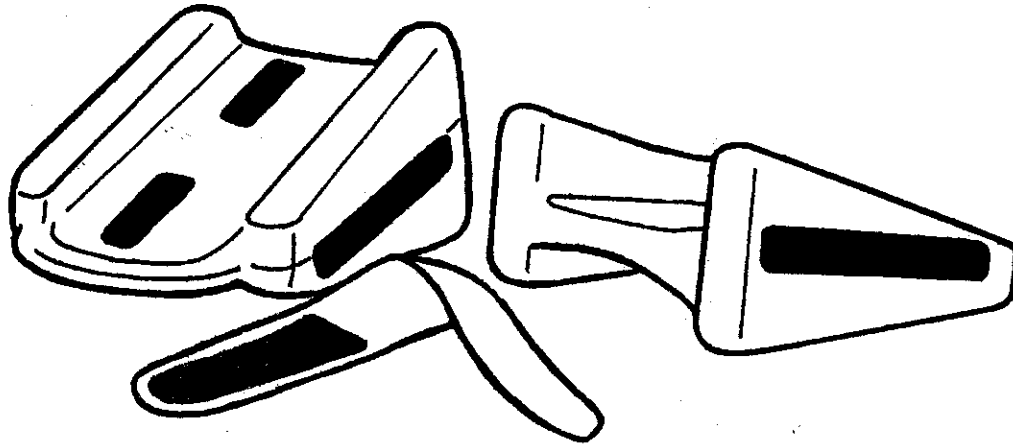


**Tumble  
Forms<sup>2</sup>**  
With Anti-Microbial  
Protection

**biform wedge**®

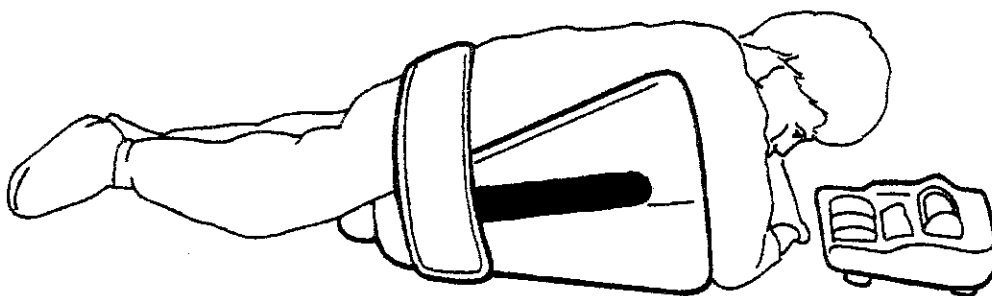


The two sided Biform Wedge is actually two wedges in one. The wedge is designed to offer two different heights for positioning simply by flipping the wedge over.

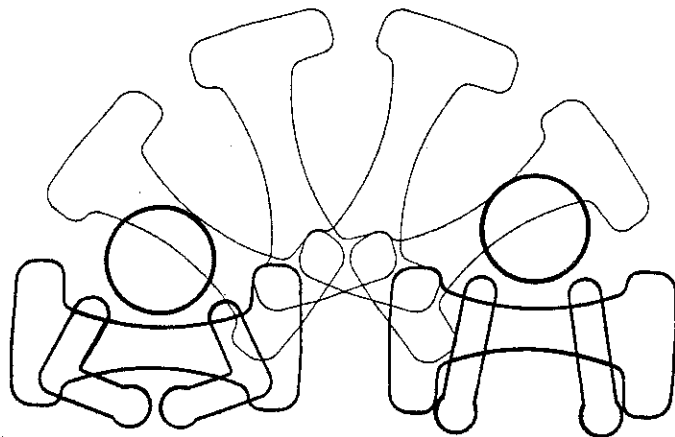

The wedge features a unique shape contoured to the chest of the child which passively brings him or her into a midline position and permits better chest expansion. In addition, lateral supports help hold the child in place. If the contour and lateral shapes are not sufficient, the child can be secured with the wide strap provided.



Each wedge is outfitted with Velcro® loop on both contoured surfaces as well as the sides. Use the Velcro® loop on the contoured surfaces to attach abductor blocks or head supports.

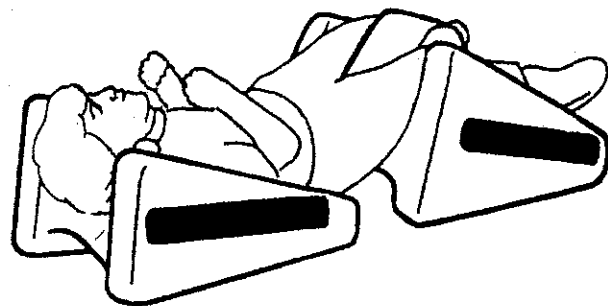

The wide straps provided can also be used to connect Biform Wedges together in a number of different positions.



In addition, attach the straps with Velcro® hook onto the Velcro® loop on the wedges to make easy carry handles.

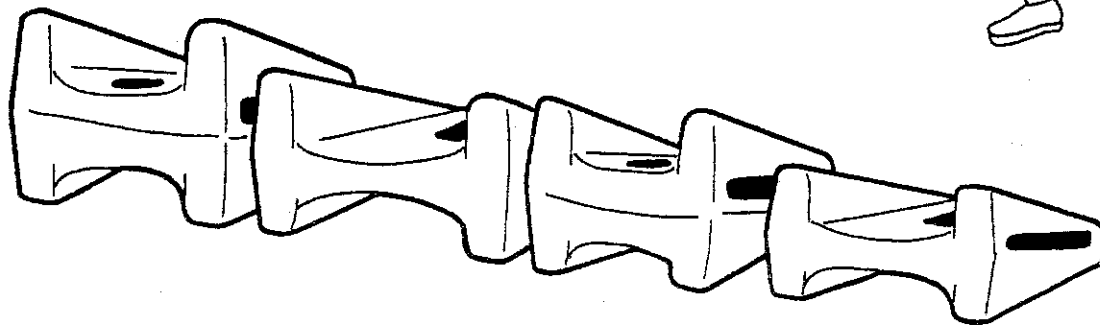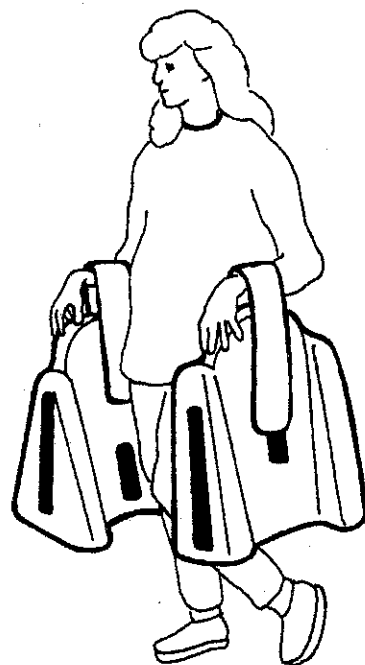

Tumble Forms 2® makes four different Biform Wedges: two small and two large. The heights of the two small wedges measure 4 & 6 inches and 8 & 10 in. The heights of the two large wedges measure 6 & 8 in. And 10 & 12 in.