


Square ModuleTM Seating System

The Square Module Seating System consists of the "box" seat and a set of blocks. Each module shape is used for adaptive positioning. They are a tremendous asset because they can be used for a thousand helpful ways of positioning. Their successful use depends largely on your creativity.

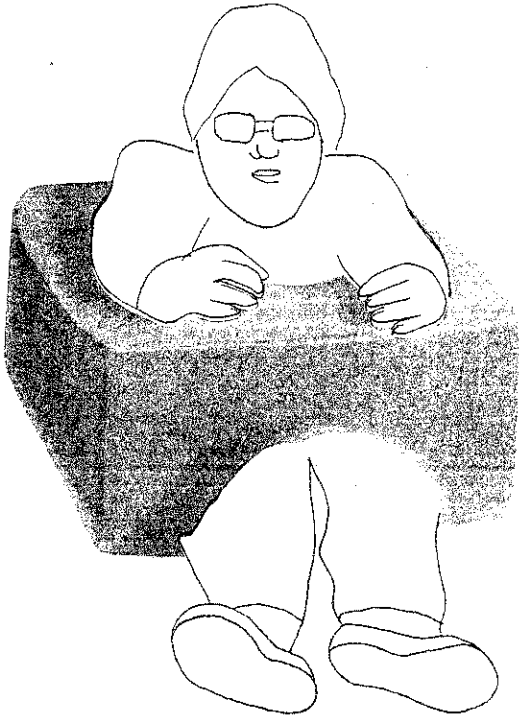

APPLIED USED

1. The square "box" seat can be turned upside down and used as an object to pull up to. Also, in this position the round hole provides good support for beginning sitting tolerance. Inverted, the "box" is an object to sit under, putting the child's torso through the hole with legs out the front opening, or flexed cross-legged inside the "box".

2. The ring can be used to practice dressing skills. The child can step into the ring and pull it up to the waist, or pull it over his head. This is an excellent way to develop body awareness.

