


## [A] Repair of Damaged Tumble Forms<sup>®</sup> Upholstery

1> Begin your Tumble Forms<sup>®</sup> upholstery repair by cleaning the area with an ammonia based window cleaner. This will remove dirt, grease, and prepare the surface for a better bond.

2> If damage warrants you may use any foam compatible adhesive to repair. Open the crack, slit, or tear and spread the adhesive uniformly and generously onto the area to be glued.


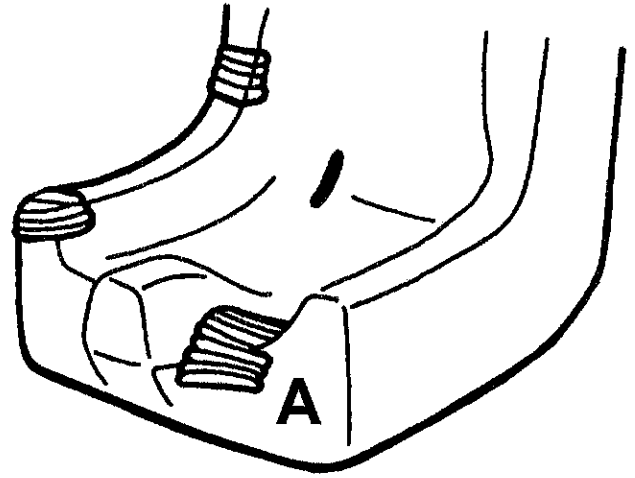


3> Cover the damaged area with the colored press patches in an overlapping sequence as shown. (A) Peeling off the paper backing is easier if you slightly tear an edge of the patch far enough to get a fingernail under the paper. Apply the patches as you would adhesive bandages.

## [B] Repair Velcro<sup>®</sup> Hook and Loop Fastener

1> If a Velcro<sup>®</sup> hook and loop fastener has peeled away from the Tumble Forms<sup>®</sup> upholstery, it can be replaced with the supplied self stick Velcro fastener.

Clean area where Velcro is to be positioned with an ammonia based window cleaner. Then simply remove backing from the self adhesive Velcro and press into place. (B)

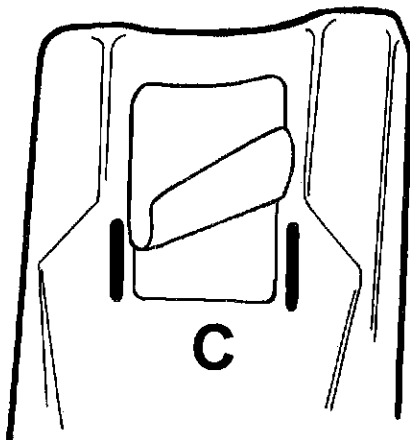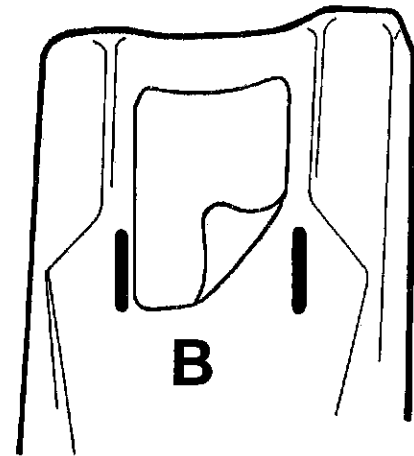

2> If Velcro<sup>®</sup> hook and loop fasteners are exhausted and have lost their holding power, a new Velcro<sup>®</sup> patch of larger dimensions may be adhered on top of the old Velcro<sup>®</sup>. Adhesive may be used if a stronger bond is required. (C)