
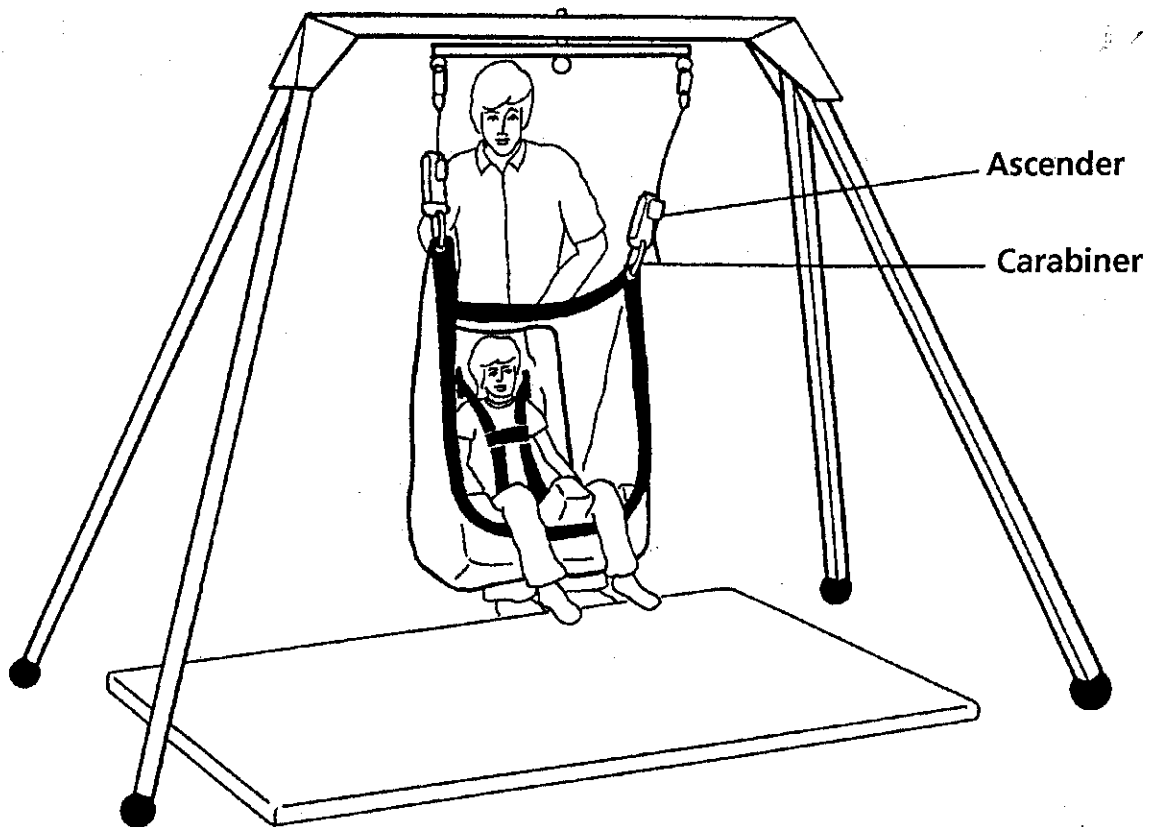


**Set up Instructions**

- 1> Connect one carabiner snap ring to each ascender.
- 2> Recline the Feeder Seat® in the net with the lower front edge of the net covering the lower front edge of the Feeder Seat®. The back of the seat should be fully supported by the net as shown.
- 3> Lower the unit, using the ascenders, to a point where your client has easy access. Use the positioning straps to maintain your clients posture.

**Net Swing Safety Notes**

The Tumble Forms® 2772S Net Swing should only be used with the Tumble Forms® Vestibulator II® frame. The client should not exceed 100 lbs in weight. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your clients safety, don't take a chance.

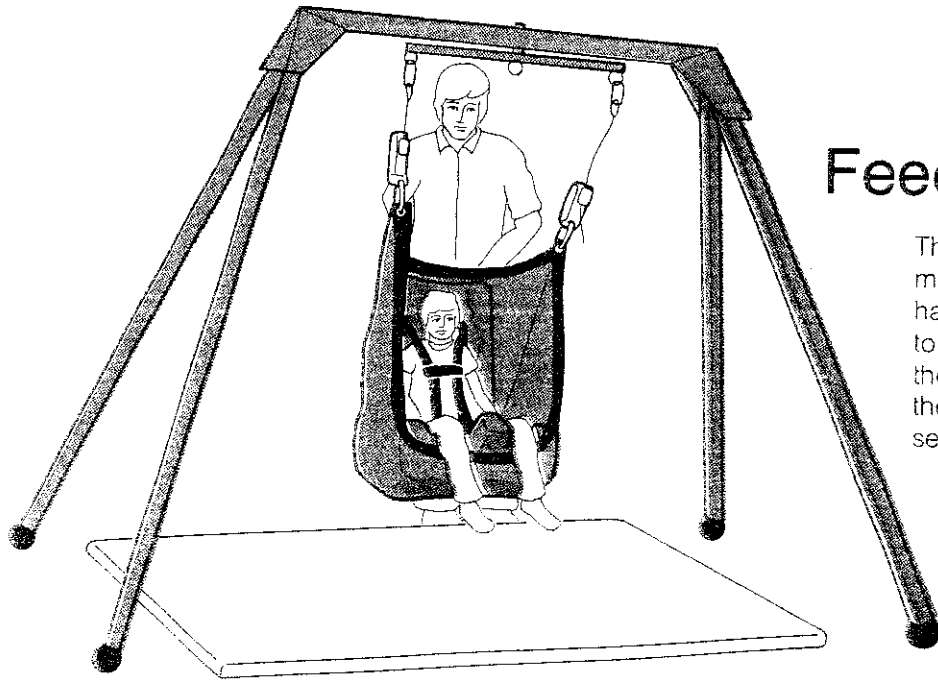

## Feeder Seat® Swing

The Feeder Seat Swing provides linear movement through space for the multiply handicapped child who is not usually able to have this experience. The child is therapeutically positioned and supported in the Feeder Seat positioner which is nestled securely in the net hammock.