

The ProForm NX™ Cushion




ProForm NX™

- A cost-effective, customizable solution from an off-the-shelf product
- For clients with challenging positioning and postural support requirements
- Incorporates Air-Foam Floatation technology with modifiable components
- Components can be easily modified using common everyday tools
- HCPCS Code E2624/E2625 Skin Protection and Positioning Wheelchair Seat Cushion, adjustable




Contoured Base

- Easily modifiable with standard, everyday tools
- Abductor Laterals
- Medial Thigh Separator
- Pre-ischial Bar
- Deep Ischial Pan

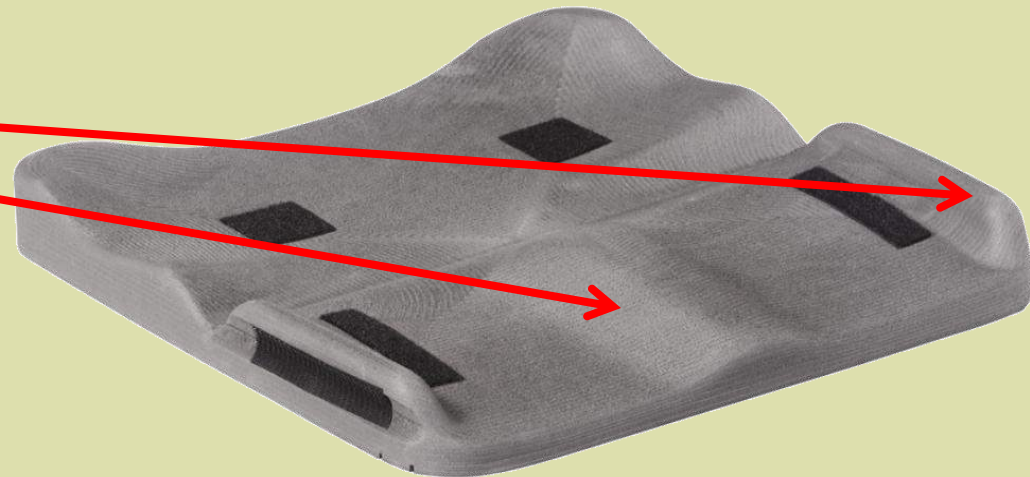

Contoured Base

- Easily modifiable with standard, everyday tools
- Adductor Laterals
- Medial Thigh Separator
- Deep Ischial Pan
- Pre-ischial Bar

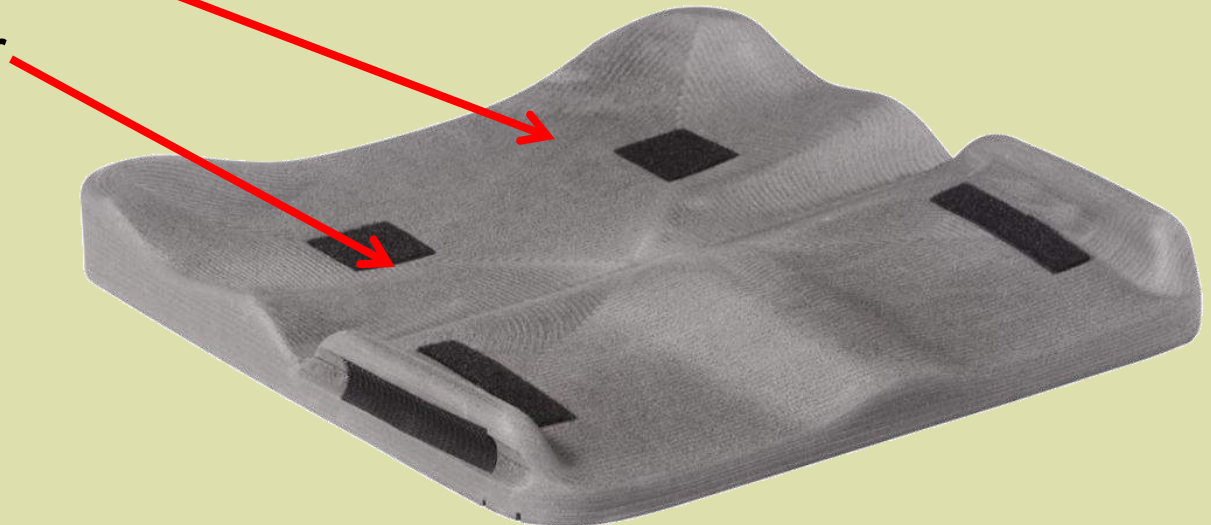


Contoured Base

- Modification Guides
- Flex Hinge



Modifiable Wedge

- Modifiable Wedge can be cut to any length for desired amount of “dump” or “squeeze”
- Because the Contoured Base is hinged, the seat-to-back angle does NOT change when Modifiable Wedge is utilized
 - The pelvis remains neutral while unloading the ITs and loading the thighs
- Can be placed inside the cushion cover

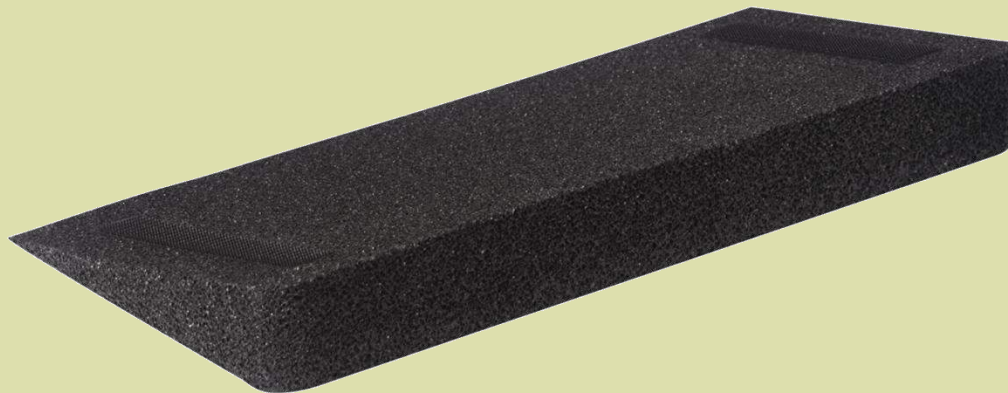

Air-Foam Floatation Cushion

- Self-inflating, air and foam cushion
- Two-way valve releases air to immerse client into the cushion
- Dual-chambered cushion
 - Patent pending technology
 - Provides accommodation of pelvic asymmetries
- Single-chambered cushion
 - Provides symmetrical pelvic and postural support



Modifiable Thigh Cushion

- Closed-cell foam provides postural support and comfort through the thigh region
- Contours over the medial thigh separator and adductor laterals
- Score lines on bottom for easy modification in 1" and 2" increments
- Hook and loop patches for secure attachment to the contoured base



Cover

- Available in mesh and incontinence versions
- Machine washable
- Non-skid surface on bottom
- Reticulated foam inside promotes air exchange and improves pressure distribution
- Hook and loop aligned on the bottom allows for cinching of the cover when making modifications



Modifying the ProForm NX™

- Modify Contoured Base and Modifiable Wedge using common blade or electric knife
- Cinch cover using provided hook and loop
- Alter Modifiable Thigh Cushion using common blade or electric knife



ProForm NX™ cushion is the seating solution for:



Pelvic Obliquities

ProForm NX™ cushion is the seating solution for:



Upper Leg Length Discrepancies

ProForm NX™ cushion is the seating solution for:



Wind-Swept Deformities

ProForm NX™ is the seating solution for:



CVA/ Unilateral Scooters

ProForm NX™ cushion is the seating solution for:



Above Knee Amputees (AKA)

ProForm NX™ cushion is the seating solution for:



Symmetrical sitters requiring extra support

ProForm NX™ cushion is the seating solution for:



Clients with limited hip flexion

ProForm NX™ cushion is the seating solution for:



Clients with hot spots